

Safety Information [PintoMed – Sacral Silicone Foam Dressing with Adhesive Border]

Version: 1

Date: 2024/11/08

Sacral Silicone Foam Dressing with Adhesive Border

While they are generally safe and effective when used correctly, improper use can lead to serious complications. By following these guidelines, you can help ensure that this product provides its intended benefits without causing harm.

Please read and follow these safety guidelines carefully:

1. Use each dressing only once.
2. Store in a cool, dry place, away from direct sunlight, to maintain the adhesive properties.
3. Do not use it if the package is damaged.
4. Do not use it after the expiration date.
5. Contents are sterile until the pouch is opened, damaged or wet.
6. Not for use on infected wounds unless under the supervision of a healthcare professional.
7. Apply and remove carefully on sensitive or fragile skin.
8. Seek medical attention immediately for severe wounds and burns. If redness or discomfort occurs, discontinue use and consult a physician.
9. Dispose Responsibly: Follow local guidelines for disposing to avoid contamination and environmental damage.
10. Seek the advice of a healthcare professional if you are unsure about using the dressings or if you experience any adverse effects. Proper use and regular monitoring are key to ensuring safety and effectiveness. This product is not a substitute for direct medical care from a healthcare professional.

By following these guidelines, you can help ensure that these dressings provide their intended benefits without causing harm.

How to Use

1. Select the appropriate dressing size to ensure the foam pad is larger than the wound area.
2. After cleaning and disinfecting the wound, remove the dressing from the pouch.
3. Peel back the liner, exposing the adhesive surface approximately halfway.
4. Center the dressing on the wound.
4. Press down firmly while removing the film.
5. Press around the edges to ensure a secure waterproof seal.
6. Replace as needed (Dressing change frequency will depend on wound type, exudate volume, and clinical situation. When exudate extends to the edges of the dressing or the dressing begins to leak, a dressing change is indicated.)
7. Store Properly: Keep the dressings in a cool, dry place, away from direct sunlight, to maintain their adhesive properties.
8. Dispose Responsibly: Follow local guidelines for disposing of used dressings to avoid contamination and environmental damage.
9. Seek the advice of a healthcare professional if you are unsure about using the dressings or if you experience any adverse effects. Proper use and regular monitoring are key to ensuring safety and effectiveness.

By following these guidelines, you can help ensure that these dressings provide their intended benefits without causing harm.