

Instructions

The FRIO cooling wallet is an evaporative unit purposely designed to keep medication* cool within safe temperatures of **18-26°C (64-78°F)** for a **minimum of 45 hours**, even in a constant environmental temperature of 37.8°C (100°F).

FRIO cooling wallets are reusable and can be reactivated, ensuring a consistent and secure storage environment for 45 hours or more.

The FRIO cooling wallet consists of two parts: The inner cooling wallet, which requires activation with water and the Cambrelle outer cover, which protects the inner cooling wallet.

* Medication must be maintained at its recommended safe temperature. Consult the manufacturer's storage instructions to ensure suitability for storage in a FRIO cooling wallet.

How to use:

1

Submerge the inner wallet (containing the crystals) in **cold water** and allow it to soak until it swells, yet remains pliable. This should take approximately **15-19 minutes** when soaking a dry inner pouch and **2-5 minutes** when reactivating a partially activated inner pouch.

2

Gently towel **dry** and then **allow the wallet to air dry for 5 minutes**, this allows the crystals to absorb any excess water.

3


When semi-dry to the touch place the activated wallet in its outer cover.

4

Place medication* in the activated wallet.

Drying Out and Cleaning

- When not in use, remove the inner wallet and allow to **completely air dry** until it returns to crystal form.
- To accelerate drying, position the wallet near a heat source, or on a sunny windowsill. Periodically shake the wallet to distribute the crystals evenly, making future reactivation more convenient.
- The outer wallet can be washed in warm water using a very mild detergent. The inner wallet can be spot cleaned by using a soft bristle brush and non-bleach antibacterial spray.

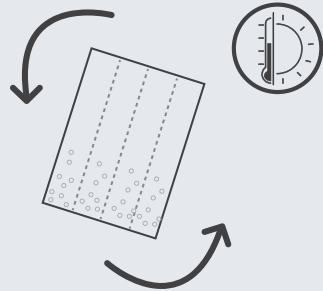


Continuous Use

To ensure continuous safe storage of medication*, re-submerge the FRIO inner wallet in water every 45 hours or when the gel quantity diminishes, causing it to revert to its crystal form.

When consistently used, reactivation only takes 2-4 minutes as the crystals remain partially activated.

Please note: The inner wallet does not need to be fully dry when reactivating your FRIO cooling product.

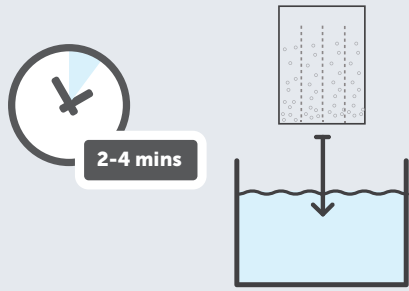

DO NOT



DO NOT carry an activated FRIO in an airtight container. This evaporative cooler needs air circulation to cool



DO NOT store your FRIO in an airtight container. Moisture combined with a lack of air circulation can lead to mold growth

DO NOT REFRIGERATE OR FREEZE



DO NOT place in a microwave or oven



DO NOT place in a tumble dryer or washing machine



DO NOT place in a cool bag