

KNEE PILLOW



PRODUCT USE & CARE


Please, be aware that due to the memory foam filler the pillow may need up to 24 hours to retrieve its operation size and properties. To get the pillow work faster just follow next few steps:

1. Unbox the product immediately.
2. Unfold the product and lay it flat.
3. Make sure the room is warm with proper air circulation.


The cushion will adapt to your body within a few days. It may feel firm initially, but it will soften once it absorbs heat from your body.

How to Use


Set the cushion horizontally its notch in the middle for one of the following positions:

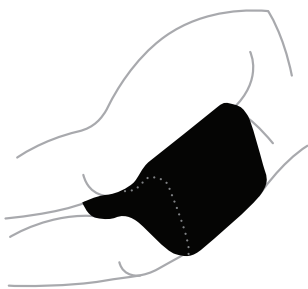

Between the knees
or thighs.

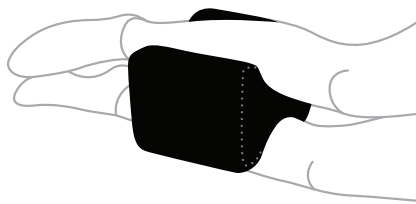


Between the shins.

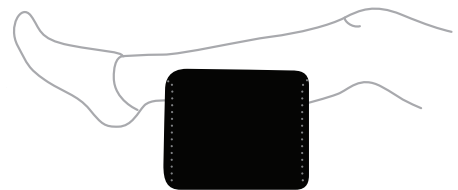

Under the calves.




Under the thigh.

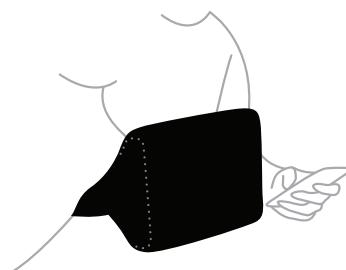


Under the chest.

How to Clean

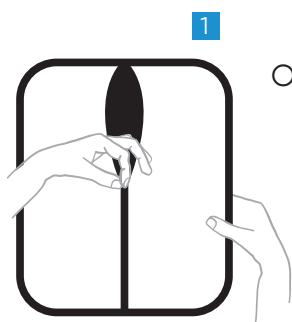


1 Open the cushion bottom zipper.

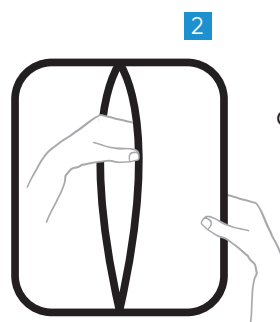

2 Take the pad out of the cover.



3 Fold the pad to remove the cover more easily if there is any resistance.

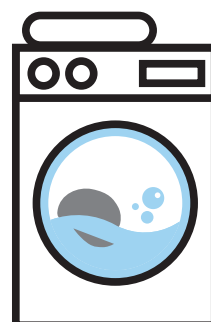

4 Use cold water only (30°) to wash the cover. Dry the cover flat or hang it up.

CAUTION
Never wash or dry the memory foam pad!

Washing Instructions



Do not dry clean



Do not bleach



Hand wash or Machine wash in cold water



Line or Flat dry



Do not iron



5 STARS
UNITED