We take breastmilk storage seriously.

Reinforced, double-sealed side seams to help prevent leaks.

Tamper-evident, **fully sealed tear-away top** for safety.

Patented double, Click 'n' Secure™ seal – a difference you can see, hear, and feel.

Designed to be frozen flat for more **space-efficient** freezer storage. Bags also stand for short-term storage.

BPA and BPS free.

Available in **25, 50** and **100 count** boxes.

Breastfeeding Questions?

Join the Lansinoh Moms' Club!

Sign up today at Lansinoh.com/AboutLMC

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BREASTMILK STORAGE BAGS

Lansinoh®

BREASTMILK STORAGE BAGS Pre-Sterilized

Directions for use:

- Open individual storage bag by tearing away tamper-evident safety seal and pulling zipper seal apart.
- Pour breastmilk into bag while holding bag open with one hand.
Caution: To avoid spilling, do not allow bag to stand on its own until bag has been filled and Click 'n' Secure™ top has been sealed.
Caution: Do not fill bag past 6 oz. Breastmilk will expand when frozen.
- Before sealing, carefully squeeze air out of the bag to preserve freshness. Seal the bag.
- Use a ballpoint or Sharpie® pen to fill in the name, date, time, and volume.
(Note: a felt tip marker can smear when wet.)

- When freezing storage bag, lay flat in a container. To avoid sticking, don't place directly on freezer surface.

Thawing instructions:

- Place sealed bag in warm tap water or in a clean, empty container. Do not put in boiling water, microwave, or on stovetop.
- Milk will separate: this is normal.
- Gently swirl after opening.
- Transfer milk into desired feeding container using convenient pour spout feature. Do not reuse bag.
- Dispose of used plastic bag safely.
- Use only as directed.

For additional storage tips and suggestions, visit Lansinoh.com

Breastmilk Storage Guidelines

WHERE	TEMPERATURE	TIME
At Room Temperature	66-78°F (19-26°C)	4 hours (ideal) up to 6 hours (acceptable)
In a Refrigerator	<39°F (<4°C)	72 hours (ideal) up to 8 days (acceptable)
Freezer	0°F (-18°C)	3 months (ideal) up to 6 months (acceptable)
Deep Freezer	-4°F (-20°C)	6 months (ideal) up to 12 months (acceptable)

Source: La Leche League International 2014

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Lansinoh® is a registered trademark of Lansinoh Laboratories, Inc.
©2015 Lansinoh Laboratories, Inc.
All Rights Reserved
Made in Thailand

BN20425EN1017

Lansinoh.com

0 44677 20425 5