






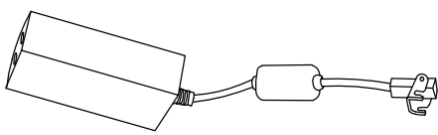
Assembly Instruction

POWER LIFT CHAIR

ITEM NO. 8545-1LT

NOTE: Do not use power tools for assembly. Power tools increase the risk of over tightening which lead to splitting or cracking the wood.


COMPONENT LIST(CHAIR)

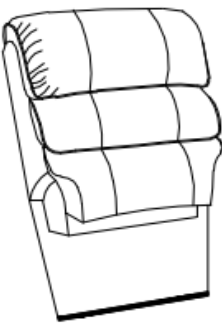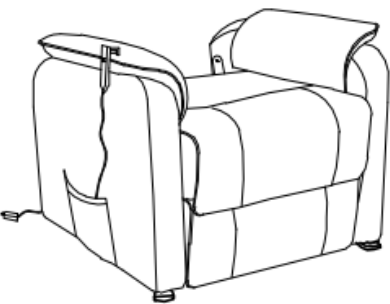NO.	DRAWING	QTY	DESCRIPTION
A		1	BACK REST
B		1	SEAT
C		1	PLUG
D		1	ADAPTOR

STEP #1


Lift up the back rest cover, align the female brackets at the back onto the male brackets at the seat, insert and push it to the end position.

Attach the plug to the adaptor, fix the wire properly to prevent any damages.

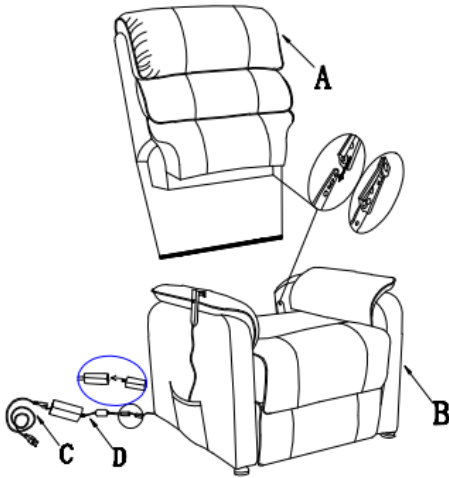

STEP # 2

Pull down the back cover onto the seat and fix the Velcro on the correct position.

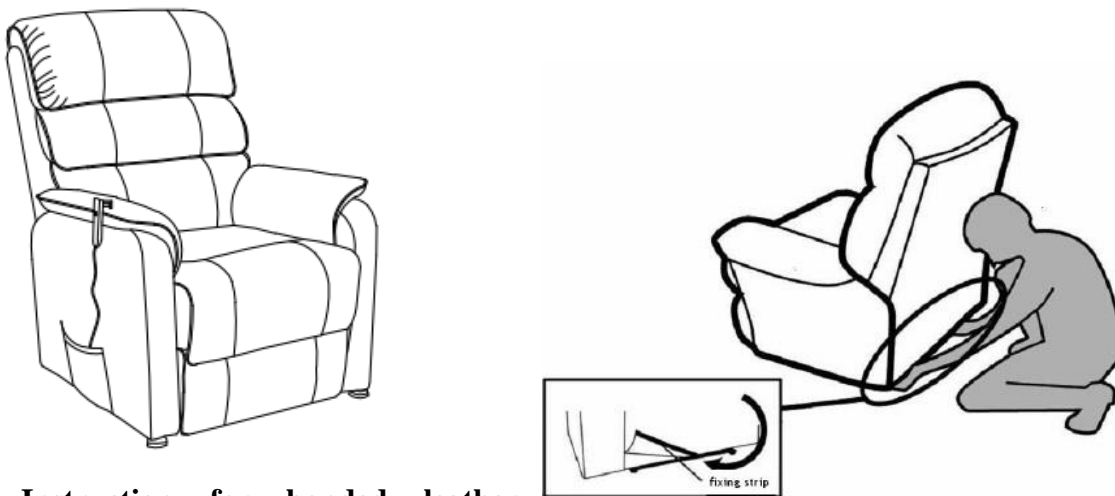

Care Instruction for bonded leather Furniture.

In order to maintain the original appearance of your bonded leather furniture, please follow below simple care procedures:

- Avoid exposing furniture directly to sunlight or heating and air conditioning outlets. Sunlight and/or exposure to heating and air conditioning outlets will fade/damage bonded leather furniture.
- Vacuum or wipe with a soft brush to remove dust.
- Clean bonded leather with professional cleanser.

Warning !

Do not allow electrical cord to be pinched between moving parts or between the frame and the floor. Do not place cord under carpet or in location where it could become worn.