Ear Cleaner with Camera

User Manual
ENDURITIFREES

Steps to use on the phone

Step 1
Download "Souleat" App
For iOS:
Download and install the App "Souleat" from App Store.
For Android:
Download and install the App "Souleat" from Google Play Store.
Optional:
Scan the QR Code below to download the App.


Step 2
Long press the switch key to open the device, after 3 seconds, then step into the next operation.


Step 3
Open the "Souleat" App, click the "CONNECT DEVICE", follow the pop-up prompt and click to connect now to jump to the mobile WiFi setting interface.

Step 4
Find "Souleat-xxxx" on the WiFi setting interface and click to connect.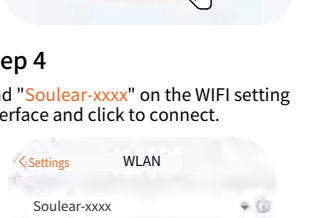

Step 5
After the mobile phone displays "Souleat-xxxx" WiFi is connected, return to the App again and click the "CONNECTED" to enter the device.

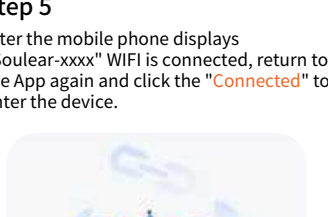After connecting to the WiFi of the product, you will be asked some questions:

IOS:
"Souleat" would like to find and connect to devices on your local network.
Suggestion: Click "OK".

Android:
"Souleat" would like to find and connect to devices on your local network.
This app will be able to discover and connect to devices on the network you use.
Suggestion: Click "Allow".

Allow "Souleat" to use wireless data?
Click: "WLAN/Cellular/WLAN Only".

Allow "Souleat" to use wireless data?
Some functionality may not work when you are not connected to the internet.
Click: "WLAN/Cellular/WLAN Only".

Android:
If your mobile phone displays the following prompts, WiFi has no data connection, whether to continue trying to use the WiFi? Whether to keep the connection? Whether to switch WiFi?
Suggestion: Click "Keep"/"CONNECT"/"Decline" option.

Troubleshooting

Problems	Suggestions
Camera does not photos or videos to your phone	Let Souleat permit the authority of photos, media and camera to your phone.
Camera is no response, camera doesn't light up	Please charge the device.
App crashed	Please uninstall App and scan the QR Code again.
For endoscope's mid light on	Please charge the product.
WiFi connection break	Please restart ear endoscope, open the App, connect to WiFi.
When photos prompt "cannot post this with", the network is temporarily unavailable, please try again later"	When the above situation occurs, please try to turn off the product, turn on the application, and reconnect to WiFi to connect and use.

Product Parameter

Product Name	Working Frequency
Ear Cleaner with Camera	2.4GHz
Network Standard	Resolution
IEEE 802.11b/g/n	1080P
Image Transmission Rate	Real Focal Length
20fps	1.5-2cm
Environment Temperature	Charging Time
10-30°C	1.5h
Gravity Sensor	Working Voltage
6-axis	3.7V-4.2V
Input Parameter	Lens Diameter
5V=0.6A	3.5mm

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