


Quick Reference Manual


HOW TO CHECK THE MEMORY RECORDS

Step 1: year, date, time setting

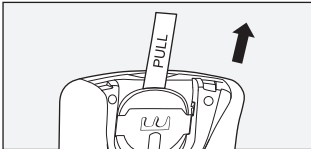


Step 1:

Pull out the isolated strip, the machine will automatically turn on and immediately enter the shutdown state.

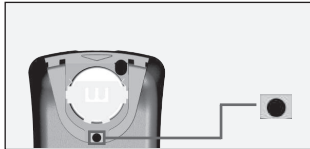

Step 2:

Press **black setting button**  besides the battery with battery back cover for 2 seconds.

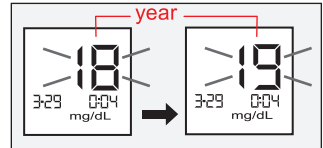


Step 3:

Find the flashing "YEAR" and click query button  until the desired year is displayed.



Step 4:

Repeat STEP 2~3 to successively set month, day, hour and minute.

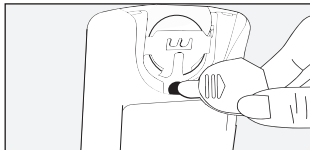

Step 5:


When the desired minute is chosen, press the black setting button  again.



Step 6:


The screen displays off, indicating the setup is complete.

Step 2: check the memory records

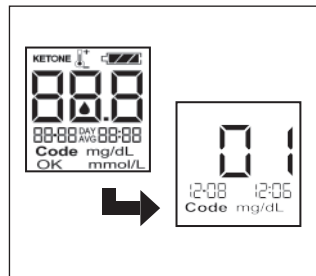


Step 1:

With the meter off, press query button  to turn on the meter.

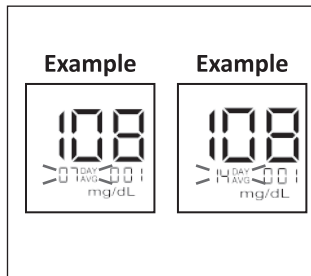

Step 2:

Press query button  to display 7-day average. By repeating this step, you can check the average values of 14-day / 30 days.

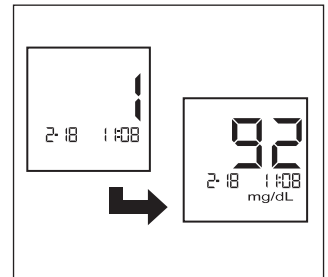

Step 3:


Press query button  to check previous test results starting from most recent test.

Quick Reference Manual

EASY TO USE Start with only 5 steps


①

Make sure the isolated strip is removed and the machine is off and the battery is installed correctly.

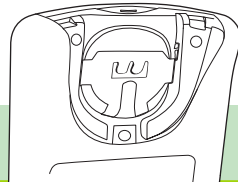


②

Make sure the device is off, then insert the strip with blue surface facing up into the meter.

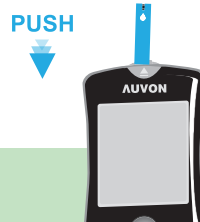


③

See the flashing drop (If you do not find the flashing drop, do make sure the meter is off before inserting the test strip).

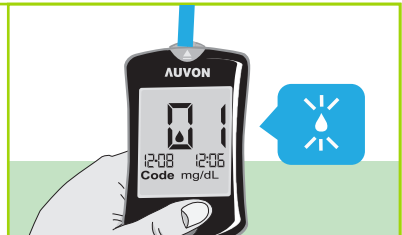

④

Touch the drop to the tip of the application zone of the test strip until you hear a "beep" sound. Do not apply blood on top surface of test strip.

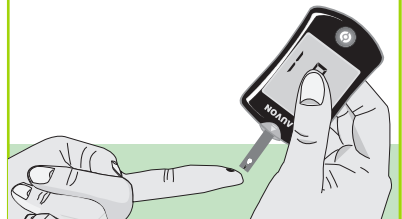

⑤

Get result in 6 seconds.

