BRAGUERO OMBLIGO

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UMBILICAL BELT

ENGLISH WH04 REGULATIONS. INSTRUCTIONS FOR USE, STORAGE AND WARRANTY

Read the instructions carefully. Keep the user manual and the packaging for future reference. If you have any questions, ask your doctor or contact our customer service team.

REGULATIONS

MD This article is a class I medical device. A Risk Analysis (UNE EN ISO 2017/745) has been carried out, minimising the existing risks. Tests have been carried out in accordance with the following European standards: EN ISO 14971: 2019 EN ISO 15223-1: 2021 EN ISO 20417: 2021 EN ISO 10993-1: 2020 EN ISO 10993-5: 2009 EN ISO 10993-10: 2023 EN ISO 10993-23: 2021.

INDICATIONS

Reduction of reducible umbilical hernias.

INSTRUCTIONS FOR USE

Correct sizing is essential to ensure therapeutic efficacy and prolong the life of the umbilical truss. It is advisable to apply compression to a firm but comfortable degree. Excessive pressure may cause intolerance and discomfort during use.

In the event that the product requires fitting, this must be carried out by a qualified health care professional, ensuring that the user understands the operation and use of the umbilical truss.

The following steps should be followed for correct placement:

1. Lying on your back, the hernia should be reduced by bringing it into the cavity of the abdomen (abdominal cavity).
2. Place the containment pad on the area affected by the umbilical hernia. The pad should be fixed with the Velcro in a vertical direction.
3. Place the umbilical band around the umbilical contour, tighten and close.

⚠ PRECAUTIONS

Before each use, check that the product retains all its components. Periodically check the condition of the umbilical cuff rigorously. If you notice any damage, report it to customer service and do not use the product. The material is flammable. Do not expose the product to circumstances that could cause combustion. If this should happen, remove the belt quickly and extinguish it with the means at your disposal.

Excessive perspiration during use may cause irritation. Use a layer of cotton to separate the skin from the fabric. In case of more serious discomfort such as irritation, inflammation or chafing, seek medical advice. The product should be applied to healthy, intact skin. Not recommended for use on open scars that are inflamed or reddened. Extreme precautions should be taken in case of MRI or radiation associated with diagnostic tests.

WARNING

The use of the umbilical truss is conditional on the indications given in this instruction manual or by the medical staff. Although the orthosis is not for single use and can be reused, it is recommended for single use only.

For disposal of the product and packaging, strictly comply with the legislation in force in your community.

COMPOSITION

Consult packaging and/or labelling.

WASHING AND STORAGE RECOMMENDATIONS

It is recommended to keep the product in its original packaging when not in use, in a dry place at room temperature. Wash periodically by hand with lukewarm water (max. 30°) and neutral soap. To dry the product, remove excess moisture with a towel and leave to dry in a well-ventilated place. Do not iron or dry with a dryer. No

Do not expose to direct sunlight or heat sources such as tumble dryers, cookers, etc. Do not use abrasive substances or alcohols during cleaning, as they may irritate the skin and deteriorate the integrity of the product.

WARRANTY

JAZZU EYEWEAR S.L. guarantees all its products, provided that they have not been modified from their initial conformation. The warranty does not cover products which, due to misuse, are deformed or lose their functionality.

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