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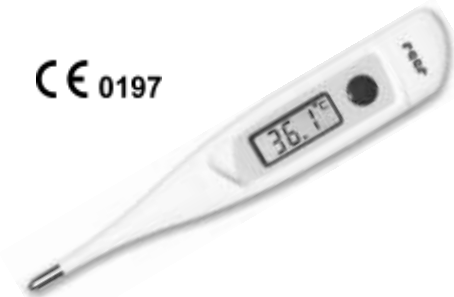
SCALA Electronic GmbH
Ruhlsdorfer Str. 95
14532 Stahnsdorf

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reer GmbH
Postfach 1639
71206 Leonberg

rapid clinical thermometer SC 42 TM (Reer article number 9840) with flexible measuring point

We congratulate you on the purchase of this quality product from REER. Please read the instructions for use before using for the first time.

Special features:

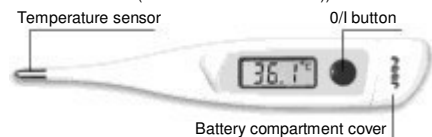
The SC 42 TM is a maximum thermometer, i.e. the highest **temperature** is **displayed after only 30 seconds**. The most recent measured value is always saved.

Safety information:

- The SC 42 TM is suitable only for measuring human body temperatures.
- The thermometer contains small parts (batteries, etc) which could be swallowed by small children. Please keep the thermometer and its batteries out of the reach of small children. If the battery or other parts are swallowed you should seek medical help immediately.
- The enclosed batteries may not be recharged or reactivated by other methods, may not be dismantled, thrown into fires or short-circuited. Batteries must be disposed of properly. Suitable methods for the disposal of batteries include returning them to the place of purchase or disposal in communal collection points.
- Protect the SC 42 TM from direct sunshine and temperatures above 50°C.
- Never drop the SC 42 TM, do not bend or open it (with the exception of the battery compartment for battery replacement).

Operating manual:

Press the I/O button to switch on. A short signal tone shows that the device is in the "switched on" condition. A segment test takes place simultaneously (checks all display segments and carries out an internal function test (see under "Function test")).

After this, if the temperature measured by the temperature sensor is less than 32°C, the display shows an "L" and "°C". After around one second, the "°C" starts to blink. The "L" stands for low (less than 32°C).

The current temperature is continuously displayed during measurement, and the "°C" blinks during the entire measurement time. The measurement is completed after around 30 seconds.

If the measurement is taken orally or rectally, the approximate final temperature is shown after about 30 seconds and changes only marginally after this time. A measurement period of between five and 10 minutes must be observed for the underarm measurement method. The signal tone can be sounded several times during this period. Since the contact between the temperature sensor and the armpit is not as good as those for oral and rectal measurements, the temperature rise is slower which causes this effect.

After the measurement is completed, the temperature value measured remains in the display until the SC 42 TM is switched off. To turn the device off after use, press the I/O button briefly. The signal tone will again be sounded. If the device is not switched off, a signal tone is sounded after around 8 minutes and the device then switches off automatically.

Display of last measured value:

If you keep the I/O button pressed for longer than three seconds, the last measured value will be shown in the display. As soon as you release the I/O button, the device is ready to take measurements.

Function test:

The thermometer is checked for correct functioning automatically every time it is switched on. All segments are displayed, and then L°C. If a failure (measurement inaccuracy) is recognised, "ERR" (Error) is shown in the display.

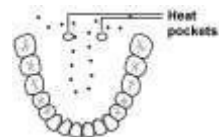
Measured value deviation:

The measured temperatures will always deviate, depending on the measurement location. This is normal, and caused by deviations in temperature distribution on the human body. You could, for example, measure a rectal temperature of 36.6°C, an oral temperature of 36.1°C and an underarm value of 35.6°C.

Measurement locations:

In the mouth (oral)

Small temperature variations in the mouth between measurements are normal. These are caused by varying temperature distribution within the mouth. Try to place the thermometer in one of the two heat pockets under the tongue, to the left and right of the tongue root. The measurement sensor must have good contact with the tissue. The subject should close the mouth and breathe normally through the nose so that the measured result is not affected by breathed air. Measuring period: approx. 30 seconds

In the anus (rectal)

This is the safest method. It is particularly suitable for infants and small children. Guide the thermometer point carefully 2 cm into the anus. Measuring period: approx. 30 seconds.

Under the arm (axillary)

From a medical viewpoint this type of measurement leads to inaccurate values. It is therefore unsuitable for carrying out exact measurements. Measuring period: 5 to 10 minutes.

Measuring the basal body temperature

The device can also be comfortably and practically used for the measurement of fertile or infertile days in the female cycle using the temperature measurement method.

Changing the battery:

As soon as the  symbol appears in the display, the battery has run out and requires replacement. Always replace the discharged battery with one of the same type. To do this, proceed as follows:

Pull the cover off the battery compartment. The battery can now be accessed. Remove the used battery with the help of a pointed object (pincers, small screwdriver etc) and put the new battery (LR 41 or SR 41) in. Please make sure it is the right way round. You can purchase batteries from jewellers or electrical shops.

Measurement checks:

Checking the accuracy of the SC 42 TM is not necessary in most cases. If you require a measurement check (at your expense), please send the SC 42 TM to our customer service address. Please make sure it is well-packaged and that the correct postage stamps are applied.

Cleaning and disinfection

Please use an alcohol pad only. Each pharmacy sells alcohol pads from different suppliers.

We recommend the use of Mucocit®-F2000 or Helipur®Hplus to disinfect the device during use in medical facilities. Both are effective against all hospital-typical pathogens and can be purchased from any dispensing chemist.

Disposal note

The disposal of batteries is regulated. Spent batteries are collected at collection points provided by battery dealers and the communities.

Dispose of the blood pressure monitor as prescribed. i.e., at a collection point for electrical scrap.

Guarantee:

We give a guarantee period of 2 years from the date of purchase. We guarantee perfect design of this article.

We will rectify all material or manufacturing errors during the guarantee period. If, contrary to expectations, defects are found, please send the device back to our customer service department. Please enclose the proof of purchase and a description of the defect so that the problem can be rectified as soon as possible.

The guarantee excludes damages caused by improper handling, related to the battery or to the packaging.

Referenced norms:

The device has been certified in accordance with DIRECTIVE 93/42/EU of 14th June 1993 relating to medicinal products.

The clinical thermometer 9840 complies with the requirements of European norm EN 12470-3:2000.

Specifications:

Type:	maximum thermometer
Measuring range:	32.0°C to 43.9°C Temperature below 32.0°C: display "L" for low. Temperature above 43.9°C: display "H" for high.
Measurement accuracy:	0.1°C between 35.5°C and 42.0°C at a surrounding temperature of 18°C to 28°C.
Self-test:	Automatic internal check of test value 37.0°C. In case of deviation > 0.1°C, "Err" will be displayed.
Display:	3-digit liquid-crystal (LCD display).
Battery type:	1 no.: LR 41 or SR 41.
Battery display:	The symbol  shows when the battery is discharged.
Storage temperature:	0°C to 50°C
Manufactured under U.S. Patent No.:	US 6,379,039

Explanation of symbols

	Manufacturer
	CE-sign of a notified body (TUV Rheinland)
	Product type BF
	Disposal notes
	Read the instruction manual
	Production period (MM/YY)

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