

Instructions for Use of Mouthpieces and compatible Breathalyzers.

The MPP50 is a 50-pack of additional mouthpieces for BACtrack S80, Trace, Scout, Element & S75 Breath Alcohol Testers. Instructions for replacing the mouthpieces are located in the user guides of the aforementioned models. If a user does not have one of previous mentioned products, there is no value in purchasing the BACtrack Professional Mouthpieces, Pack of 50.

Here is a copy of the relevant page from the BACtrack Trace User Manual to provide an example.


PREPARATION

INSTALLING THE BATTERY

Install two AAA alkaline batteries in the battery compartment.

COMPONENTS DIAGRAM

FRONT VIEW

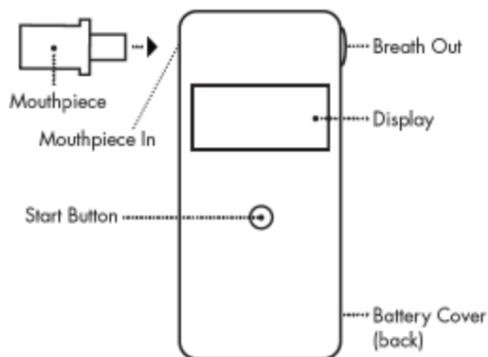

OPERATION

IMPORTANT!



Test subjects should wait 20 minutes after eating, drinking, or smoking before blowing into the BACtrack Breathalyzer. Failure to wait 20 minutes can provide inaccurate test results and damage the sensor.



STEP 1: Insert a mouthpiece into the Mouthpiece In slot. Do not force it—mouthpiece will only insert a few millimeters.



STEP 2: Press  The BACtrack will quickly display the total number of tests performed, and then begin a countdown.



STEP 3: When the countdown approaches 2, begin to inhale a deep breath.