


# Instruction Manual

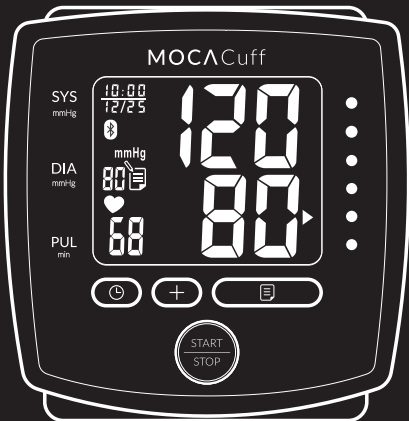

MOCACARE

# Automatic Wrist Blood Pressure Monitor

Model Number: MOCACuff-Black

<b>04</b>	Medical Disclaimer	<b>23</b>	Positioning Guide
<b>04</b>	Intended Use	<b>24</b>	Measurement Procedure
<b>04</b>	About Blood Pressure	<b>28</b>	Memory Function
<b>06</b>	Measurement Method	<b>30</b>	Bluetooth Transmission
<b>07</b>	Accuracy	<b>32</b>	Storage and Maintenance
<b>07</b>	Precautions	<b>34</b>	Troubleshooting
<b>11</b>	Device Overview	<b>36</b>	Warranty & Recalibration
<b>13</b>	Symbol Definitions	<b>37</b>	Specifications
<b>15</b>	Features	<b>39</b>	Note
<b>20</b>	Installing Batteries	<b>42</b>	Appendix
<b>22</b>	Applying the Cuff		

## MEDICAL DISCLAIMER

This manual and product are not meant as a substitute for advice provided by your doctor. You are not to use the information contained herein, or this product for diagnosing or treating a health problem or prescribing any medication. If you have or suspect that you have a medical problem, promptly consult your healthcare provider.

## INTENDED USE

MOCACuff automatically measures human's systolic, diastolic blood pressure and heart rate by using the oscillometric method during deflation. All values can be read out in one LCD panel. Measurement position is at human being's wrist. The intended use of this over-the-counter device is for adults aged 18 years and older with wrist circumference ranging from 5.3 inches to 7.7 inches (approx. 135 mm to 195 mm) and for home use. MOCACuff detects the appearance of irregular heartbeats during measurement; an indicated symbol will appear with measurement readings.

The BP Category Indicator will show the information with the readings on the screen for the user to track their blood pressure level.

## ABOUT BLOOD PRESSURE

### *What is blood pressure?*

Blood pressure is the measurement of the force of blood pushing against the walls of the arteries. Arterial blood pressure is constantly fluctuating during the course of the cardiac cycle. The highest pressure in the cycle is called the systolic blood pressure, and represents the pressure in the artery when the heart is beating. The lowest pressure is the diastolic blood pressure, and represents the pressure in the artery when the heart is at rest. Both the systolic and the diastolic pressure are necessary for a physician to evaluate the status of a patient's blood pressure. Many factors such as physical activity, anxiety or the time of day, can influence your blood pressure. Blood pressure is typically low in the mornings

and increases from the afternoon to the evening. It is on average lower in the summer and higher in the winter.


### *Why is it useful to measure blood pressure at home?*


Having one's blood pressure measured by a doctor in a hospital or a clinic, is often associated with a phenomenon called "White Coat Hypertension" where the patient becomes nervous or anxious, thus raising his blood pressure. There are also numerous other factors that might cause your blood pressure to be raised at a specific time of day. This is why medical practitioners recommend home monitoring as it is important to get readings of blood pressure during different times of the day to really get an idea of your real blood pressure. Medical practitioners generally recommend the "Rule of 3", where you are encouraged to take your blood pressure three times in a row (at 3 ~ 5 minute interval), three times a day for three days. After three

days you can average all the results and this will give you an accurate idea of what your blood pressure really is.

### *WHO blood pressure classifications:*

Standards for assessment of high or low blood pressure without regard to age, have been established by the World Health Organization (WHO), as shown in the chart. However this chart is not exact for classification of blood pressure and it's intended to be used as a guide in understanding non-invasive blood pressure measurements. Please consult with your physician for proper diagnosis.

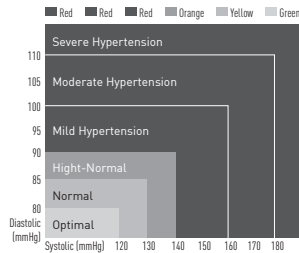

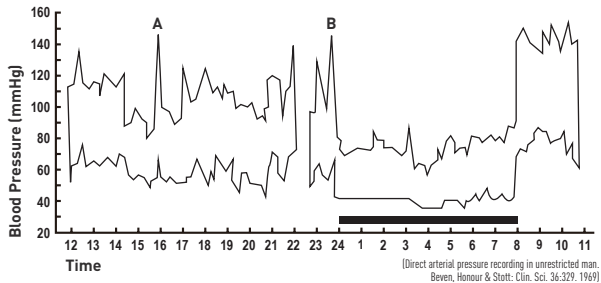

#### ***Variations in blood pressure:***

Individual blood pressures vary greatly both on a daily and a seasonal basis. These variations are even more pronounced in hyper tense patients. Normally the blood pressure rises while at work and is at its lowest during sleeping period. ***(hyper tense: means a person who has high blood pressure symptom.)*** The graph above illustrated the variations in blood pressure over a whole day with measurement taken every five minutes.

The thick line represents sleep. The rise in blood pressure at 16:00 (A in the graph) and 24:00 (B in the graph) correspond to an attack of pain.

### **MEASUREMENT METHOD**

MOCACuff Automatic Wrist Blood Pressure Monitor measures blood pressure and heart rate by oscillometric method, meaning the fluctuations in pressure are measured. Once the cuff is wrapped around your wrist, just turn on the monitor and

inflation automatically starts. The inflation of the cuff creates pressure around the arteries inside the wrist. Within the cuff is a gauge which senses the fluctuations (oscillations) in pressure. The fluctuation measured represents the degree of intensity that your arteries contracting with each heart beat, and also a result of the pressure that the cuff has placed on the wrist. The monitor measures these contractions and converts the information to a digital value. This is the result displayed on the monitor screen. Once the measurement is complete, the cuff will automatically deflate.

#### **\* Note!**

- 1. The patient is an intended operator.**
- 2. The applied part is the cuff.**

### **ACCURACY**

MOCACuff Automatic Wrist Blood Pressure Monitor has been clinically tested against a scientific device called a mercury

sphygmomanometer, considered the gold standard in blood pressure measurement. All MOCACuff Automatic Wrist Blood Pressure Monitors have performed equivalent to measurements taken with this scientific device and are within the accuracy limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers. The SPHYGMOMANOMETER was clinically investigated according to the requirements of ISO 81060-2:2013.

- \* We suggest our users have their blood pressure monitor checked every 2 years. This operation should only be performed by Manufacturer or by authorized representatives.**

### **PRECAUTIONS**

- Do not use this manual and product as a substitute for advice, diagnosing or treating a health problem or prescribing any medication by your doctor. If you have

a medical problem, promptly consult your healthcare provider.

- Read the Instruction Manual thoroughly before measuring and keep it at hand for your reference.
- This device uses the oscillometric method to measure systolic and diastolic blood pressure as well as your heart rate. It's recommended for use by people over the age of 18 and not to be used on infant or children.
- The device is designed for home use and not suitable for clinical use.
- This monitor is not intended for use in the MR environment.
- Do not take a measurement in a low (less than 41 °F/5 °C) and high (more than 104 °F/40 °C) temperature, nor in a place outside humidity ranges (15% ~ 93% R.H.), and atmospheric pressure ranges (700 ~ 1060 hPa), or you may get inaccurate readings.
- Wait 30 ~ 45 minutes before measurement if you've just consumed caffeinated beverages or smoked cigarettes.

- Rest at least 5 ~ 10 minutes before taking a measurement.
- To allow your blood vessels to return to the condition prior to taking the measurement, please wait at least 3 ~ 5 minutes in between measurements. You may need to adjust the wait time according to your physiological situation.
- We recommend you using the same wrist (preferably the left wrist) and measuring around the same time each day.
- Sit down comfortably and place your elbow on the table with your feet flat on the floor. Please do not cross your legs during measurements.
- Keep the device at heart level. Relax your hand with the palm facing up.
- Perform measurements in a quiet and relaxed environment at room temperature.
- Do not move or shake the device during a measurement. Please keep quiet and do not talk during measurements.
- This product is not suitable for:
  - \* Pregnant women

- \* People with arrhythmias
- \* Undergoing intravenous injection on any limb
- \* Currently in a dialysis treatment
- \* In pre-eclampsia condition
- For those who have had mastectomy surgery (especially whose' lymph nodes were removed), it's recommended to take a measurement on the unaffected side.
- When used among medical electronic equipments on the same limb, pressurization of the cuff may cause temporarily interference to other devices.
- Keep in mind that blood pressure naturally varies from time to time through out the day and is affected by lots of different factors such as stress, eating, smoking, alcohol consumption, medication, and physical activity, etc.
- Normally the blood pressure rises while at work and is at its lowest during sleeping period.
- Blood pressure measurements should be interpreted by a physician or a trained health professional who is familiar with

your medical history. By using the unit and recording the results regularly for your physician to interpret, you will keep your physician informed of the continuing changes in your blood pressure.

- If you have one of the circulatory problems as arteriosclerosis, diabetes, liver disease, kidney disease, severe hypertension, peripheral circulation....., please consult your healthcare professional before using the device.
- Blood pressure measurements taken with this device are equivalent to those obtained by a trained observer using the cuff / stethoscope auscultation method and are within the accuracy limits prescribed by the Standard of EN 1060-4 and American National Standard for Manual, electronic, or Automated Sphygmomanometers.
- Results are not intended for direct diagnosis. Please consult with a physician if you have any questions or concerns about your results.


**\* Attention!**

1. Do not use the device on infants, children, or those who cannot express their own intention.
2. The device is equipped with sensitive electronic components. While measuring, avoid strong electrical or electromagnetic fields, e.g. mobile phones, microwave ovens, etc; or it may lead to temporary reading error or inaccuracy.
3. Consider the electromagnetic compatibility of the device (ex. power disturbance, radio frequency interference etc.) Please use it indoor only.
4. Over high frequency measurements may result in blood flow interference, which is likely to cause uncomfortable sensations, such as partial subcutaneous

hemorrhage, or temporary numbness to your wrist. In general, these symptoms should not last long. However, if you do not recover in time, please seek your medical practitioners for help.

## DEVICE OVERVIEW

### Product components

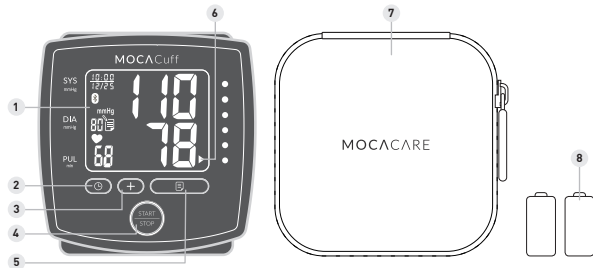


1. LCD Display
2. Mode Button
3. Set Button
4. Start/Stop Button
5. Memory Button
6. BP Category Indicator
7. Storage Case
8. AAA "LR03"(1.5V) Alkaline Battery x 2

**\* Caution!**

**Substitution of a component different from that supplied might result in measurement error.**




## DEVICE OVERVIEW

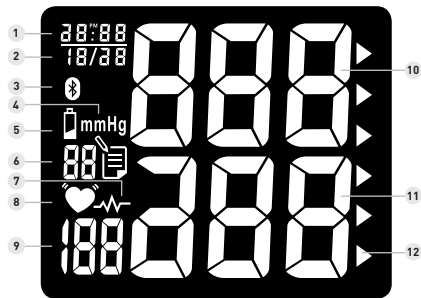


### Units display



1. Hour : Minute  
*(Also represents  
YEAR in Setting Mode)*
2. Month / Date
3. Bluetooth Symbol
4. Blood Pressure Unit
5. Low Battery Symbol
6. Memory Record Number
7. Irregular Heartbeat Detector
8. Pulse Symbol
9. Heart Rate
10. Systolic Rate
11. Diastolic Rate
12. BP Category Indicator

## SYMBOL DEFINITIONS

Symbols	Definitions
 LOW BATTERY SYMBOL	<p><i>This symbol appears when the battery power is excessively low or the polarity reverses. We suggest you replace all batteries with new ones, and make sure the +/- polarities are properly positioned.</i></p>
 PULSE SYMBOL	<p><i>Once pulse is detected, the symbol flashes with each pulse beat. Our suggestion: Please do not talk or move during measurements.</i></p>
 IRREGULAR HEARTBEAT DETECTOR	<p><i>This symbol appears for 1 minute when the user was talking, moving, shaking, or an irregular heart beat was detected during measurements. Our suggestion: Please do not talk or move during measurements. Repeat the measurement after resting for at least 5 minutes, and restart your measurement while sitting down comfortably and quietly.</i></p>

Symbols	Definitions
 <b>BLUETOOTH SYMBOL</b>	LCD displays this symbol when Bluetooth Function turns ON.
 <b>BP Category Indicator BAR</b>	The arrowhead points out the specific BP Category that your measurement reading fits in.
 <b>MEMORY RECORD SYMBOL</b>	This symbol goes along with figures according to the order of reading stored in the memory.


Stages of Blood Pressure Levels	Systolic (mmHg)	Diastolic (mmHg)	Color	Recommendations by SIGN 49: Hypertension in older people
Grade 3 Severe Hypertension	≥ 180	≥ 110	Red	Confirm immediately and repeat BP in one day and again within one week depending on clinical situation.
Grade 2 Moderate Hypertension	160-179	100-109	Red	Serial blood pressures repeated within one month.
Grade 1 Mild Hypertension	140-159	90-99	Red	Provide advice about lifestyle modification and confirm within two months.
High-Normal	130-139	85-89	Orange	Provide advice about lifestyle modification and recheck in one year.
Normal	120-129	80-84	Yellow	Recheck in 2 - 5 years. (patients aged → 75 years offered annual health check)
Optimal	< 120	< 80	Green	

\*Source: WHO, 2003

## FEATURES

### BP Category Indicator

This device is equipped with BP Category Indicator which classifies your blood pressure measurements into six stages (Optimal to Severe hypertension) as shown in below chart: After each measurement is completed, LCD display will show your position automatically on the six segments of

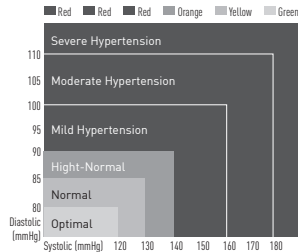



the bar indicator which corresponds to BP Category Indicator.

**\* Note!**

***When a person's systolic and diastolic pressures fall into different categories, the higher category should apply. e.g. systolic rate 181 & diastolic rate 99 → Red category (Severe Hypertension) e.g. systolic rate 110 & diastolic rate 95 → Red category (Mild Hypertension)***


For adults 18 and older who are not on medicine for high blood pressure, are not having a short-term serious illness, and do not have other conditions, such as diabetes and kidney disease. To determine category of risk when systolic and diastolic readings fall into two areas, use the higher of the two numbers for classification. There is an exception to the above definition of high blood

pressure for people with diabetes and chronic kidney disease. A blood pressure of 130/80 mmHg or higher is considered high blood pressure for those individuals.

**\* Note !**

***The above table is not exact for classification of blood pressure and it's intended to be used as a guide in understanding non-invasive blood pressure measurements. Usually this is not a cause for concern; however we recommend you consult with your physician for proper diagnosis or seek medical advice according to our recommendation mentioned above. Please note that the device is not appropriate for diagnosing hypertension, and it is only for user reference on blood pressure monitoring.***

### ***Irregular Heartbeat Detector***

The symbol  will appear on screen indicating a certain heartbeat irregularity was detected during measurement. The heartbeat rhythm that is more than or less than 25% from the average rhythm is usually defined as an irregular heartbeat rhythm. Talking, moving, shaking or an irregular pulse during the measurement can result in the appearance of this symbol. Usually this is not a cause for concern, however if the symbol appears often, we recommend you seek medical advice. And please note that the device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

**\* Note!**

***1. The pulse display is not suitable for checking the frequency of heart pace-makers. If a certain pulse irregularity is detected during measurement often, we recommend you seek***

***medical advice.***

- 2. As a safeguard, we recommend that if you have arrhythmias such as atrial or ventricular premature beats and atrial fibrillation or any other special conditions you should check with your physician before using your device.***
- 3. The IHB function is not designed for use by people with arrhythmias nor for diagnosing or treating an arrhythmic problem. In order to filter the unstable status of user and avoid affecting the detection of heart rate from any movement, shaking or talking in the beginning of measurement, the method of averaging heart beat intervals of subject device is calculated with the three proper heart beat pulses***

*detected in the beginning of measurement and that is different from a strict mathematical averaging of all recorded intervals.*

**4. At least 3 beats with at least 25 % difference from the average heart beat interval will generate the IHB icon on the screen**

#### **Bluetooth Data Transmission**

MOCACuff features a built-in “Bluetooth Data Transmission” function, which enables the device automatically transmit measuring results to paired Bluetooth-enabled device after measurement. When connection established, BPM would transmit memory data such as Measure Date, Systolic, Diastolic and Pulse to the Bluetooth enabled device. If paired Bluetooth-enabled device is not working or is not within RF range of this device, the measuring results will be stored in the blood pressure monitor’s memory. Besides, user can press “ + ” button for

one time to open the Bluetooth function.

#### **Bluetooth compatibility**

With blood pressure monitor for Bluetooth-enabled device is:

- Bluetooth 4.0 for Android 4.3 or above,
- Bluetooth 4.0 for iOS 7.0 or above

#### **\* Note!**

**1. MOCACuff is subject to and complies with electromagnetic compatibility (EMC) standard of IEC 60601-1-2, EN 301 489-1, EN 301 489-17, EN 300 328 and U.S. federal guidelines, Part 15 of the FCC (Federal Communications Commission) rules for devices with RF capability. These guidelines help ensure that your device will not affect the operation of other nearby devices. Additionally, other devices should not affect the use of your device.**

**2. Other wireless devices that**

***are in use nearby, such as a cell or mobile phone, or a wireless network, may prevent or delay the transmission of data from your device to paired Bluetooth-enabled device. Moving away from the source of the interference or turning off these devices to resolve the problem.***

**3. Make sure MOCACuff and paired Bluetooth-enabled device are within acceptable distance (no more than 10 meters) with each other. If not, put them closer.**

**4. Be sure to select the correct User on the monitor before your blood pressure measurement begins.**

**5. Bluetooth data transmission is not available under measurement.**

#### **About Bluetooth Communication Function**

The Bluetooth communication function might not be workable to some Bluetooth devices because of the compatibility of Android system. Some issues that the Bluetooth implementations on these devices have unresolved errors. It is not because of the Bluetooth module in monitor is not supported.

#### **Application Software for Bluetooth**

To fully utilize this feature, users need to ensure Bluetooth support of Wireless (usually under settings menu) on their Android or iOS device for contactless data exchange is turned on. Then, download and install “MOCACARE” app application software from Google Play on the Bluetooth-enabled device which is compatible with Android 4.3 or iOS 7.0 or above. Please follow the following steps for installing:

1. To install MOCACARE FREE APP, go to the Google Play TM APP store, and search for MOCACARE app.
2. Click the INSTALL button. Once installed, click on MOCACARE app icon.

3. Now you can start using your Android version or iOS version of MOCACARE app with Bluetooth feature, it's a simple tool to log, track and trend your test results from your Bluetooth-enable Device.


**\* Note!**

*MOCACARE app Software Manual contains explanations of functions and instructions of how to activate them.*


*Access MOCACARE Software via MOCACARE app Software to completely utilize this feature.*




## INSTALLING BATTERIES

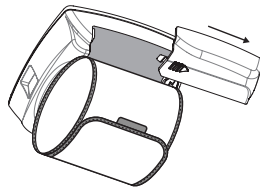

1 Take off the battery cover

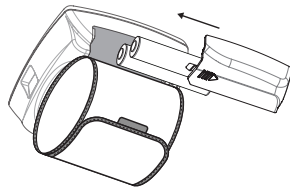

2 Place batteries in the chamber and put the cover back

When LOW BATTERY SYMBOL  appears on the display, or no reaction toward operation, please change batteries. Replace all worn-out batteries with new ones and do not mix new and used batteries. Do not mix alkaline, standard (carbon-zinc) or rechargeable (cadmium) batteries either. Such action may shorten the battery life or cause the device to malfunction.


Slide the battery cover and insert 2 AAA (LR03) alkaline batteries into the battery

compartment as shown on the figure below. Make sure the polarities "+" and "-" ends are coinciding with similar markings engraved on the battery housing.

**\* Attention!**

**1. Batteries are hazardous waste. Do not dispose of them together with the household garbage. Please discard worn-out batteries to the recycling site according**

to local regulations.

2. Keep the battery away from children in case they choke on it.
3. To prolong the battery life and prevent damage caused by leakage, remove the batteries from the device if the device is not to be used for a long period.
4. The device will keep the last measuring results after changing batteries, please reset date and time.
5. Please replace all worn-out batteries with new ones when you are operating the Bluetooth transmission function, and the **LOW BATTERY SYMBOL**  appears on the display.
6. All batteries used must be the same type. Do not mix alkaline, standard (carbon-zinc) or rechargeable

(cadmium) batteries. Do not mix old and new batteries.



## APPLYING THE CUFF


1. Wrap the cuff snugly. Do not make it too tight.
2. Fold the remaining part of the cuff back out of the way.
3. Leave approximately 0.4 inch (10 mm) between the cuff and the bottom of your hand palm.

### \* Note!


1. Do not place the pressure cuff over a jacket or sweater

1. *sleeve. Wrap the pressure cuff around the bare wrist with the monitor facing you*
2. *Do not use this device if your wrist has any wound or injury*
3. *Do not wrap the cuff around any body part other than your wrist.*
4. *In case the cuff kept pumping up no-stop, open the cuff at once.*

Position 1

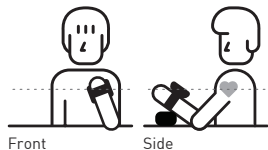

Position 2

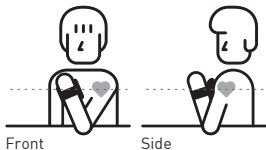


## POSITIONING GUIDE

It is extremely important that the cuff be at the same height as the heart. Having the cuff higher or lower may cause inaccurate results.

1. Sit down comfortably with your feet flat on the floor and back supported.
2. Position the blood pressure monitor on your wrist.
3. Place your elbow on the table and rest the back of your hand on the device storage case or other object.

4. Rest your wrist on the armrest until it's at the same height as your heart.
5. Relax your hand and turn your palm upwards.

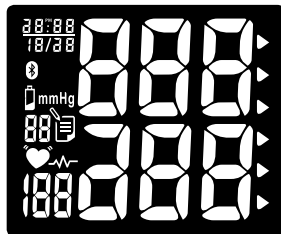

## MEASUREMENT PROCEDURE

### Switch on the Monitor

1. Press **START/STOP** button to switch on the monitor.
2. All segments appear on the screen.

### Setting Year, Date and Time

- Press **⌚** button ("YEAR" flashes). Press **+** button to adjust YEAR value.
- Press **📅** button ("MONTH" flashes). Use **+** button to adjust MONTH (1, 2, 3,....., 12).

- Adjust DATE (1, 2, 3,....., 31), HOUR (1, 2, 3,....., 12 PM, 1 PM,....., 12) and MINUTE (00, 01, 02, 03,....., 59) as described in Step A above.
- When settings are done, press **⏹** button to confirm the above settings. The device is ready to measure.



Bluetooth feature ON





Bluetooth feature OFF

### Turning Bluetooth Feature ON/OFF

User can press and hold **+** button 3 seconds to turn the Bluetooth feature ON/OFF in Standby Mode.

#### \* Note

1. The Bluetooth Feature Switch default setting is ON
2. Once Bluetooth Feature turns ON, the LCD appears Bluetooth symbol in any mode.

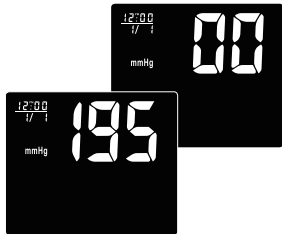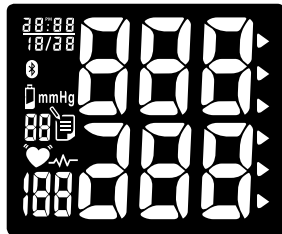

### Get Started

1. With the cuff wrapped around your wrist, press **START/STOP** button to start measurement. All display units appear on the screen.

#### \* Note!





*Do not inflate the cuff until it is wrapped around your wrist.*

2. After all symbols disappear, the display will show "00". The monitor is "Ready

to Measure" and will automatically inflate to the level that is right for you.  
3. After inflation of the cuff, the pressure will slowly decrease. When pulse is detected, PULSE SYMBOL **♥** flashes.

#### \* Note!

1. If the cuff does not stop inflating, remove the cuff at once.
2. To stop measurement, press **START/STOP** button. The

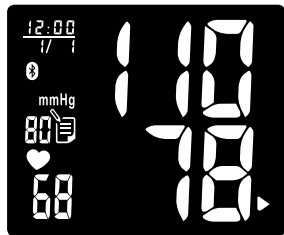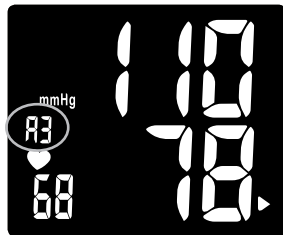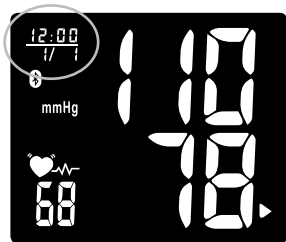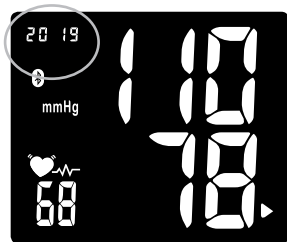

*cuff will deflate immediately  
after the button is pressed.*


- LCD screen displays your systolic rate, diastolic rate, pulse, BP Category Indicator Bar, and Irregular Heartbeat Detector symbol (if any) with date and time for 1 minute. (Year and Date / Time display alternate automatically)
- Without any operation for 1 minute, device automatically shuts off.

## MEMORY FUNCTION




### Storing data

After each measurement, the systolic and diastolic pressure, heart rate with date and time will be automatically stored. The monitor can store up to 99 memory sets for one user, and automatically replace the oldest data with new one.

### Recalling data





- Press  button to enter Memory Mode. LCD displays average of last 3

measuring results first.

- Press  button again, LCD displays the latest measuring result. Use  button to scroll through all stored measuring results. (Year and Date / Time display alternate automatically)
- To stop reading memories, press  button, and switch to Standby Mode.




#### **Erasing data**

1. Press  button to enter Memory Mode.
2. Press and hold  and  buttons at the same time, the data will be erased automatically.
3. To confirm the data in the selected user has been erased, press  button and no data should appear.

#### **\* Note:**

***Once deleted, your data can NOT be restored.***

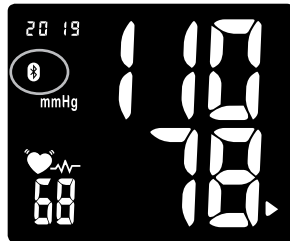

Bluetooth Symbol lit constantly under transmission

## **BLUETOOTH TRANSMISSION**

To activate Bluetooth function, please make sure your Bluetooth-enabled device have downloaded APP, and follow pairing instruction. There are 2 ways to process Bluetooth Transmission if Bluetooth function is ON:


#### **Measurement Completed:**

1. After measurement completed, the device activates Bluetooth function automatically, and the Bluetooth symbol will begin flashing on the screen.

2. While transmitting the reading to your Bluetooth-enabled device, MOCACuff Bluetooth Symbol will remain steady on the screen.
3. MOCACuff can only pair up with one Bluetooth-enabled device at a time. To transmit measuring results to other Bluetooth-enabled device, please retry Steps 1 ~ 2.

#### **Press button for one time:**

Under Standby Mode,

1. Press  button for one time to wake up the device and starting Bluetooth function (Bluetooth symbol flashing).
2. While transmitting the reading to your Bluetooth-enabled device, MOCACuff Bluetooth symbol will remain steady on the screen.
3. MOCACuff can only pair up with one Bluetooth-enabled device at a time. To transmit measuring results to other Bluetooth-enabled device, please retry as mention above.



### **Date/Time Synchronization**

1. The BPM's Date/Time Setting can be synchronized by Bluetooth-enabled device (e.g. smart phone) which has downloaded and installed DailyChek® application software.
2. When Bluetooth connection is established, the Bluetooth-enabled device can send command with the date/time information to BPM and the BPM's date/time will be updated.

#### **\* Note!**

1. ***Without any operation in 1 minute, the device shuts off automatically and Bluetooth Transmission OFF.***
2. ***Standby Mode: Segments appeared but not under BPM measuring or data transmitting.***
3. ***Sleeping Mode: Clear all LCD segments.***

## **STORAGE AND MAINTENANCE**

### **General Use**

- Do not in any way twist the cuff.
- Do not press **START STOP** button if the cuff is not wrapped around the wrist.
- Do not drop the product and avoid any strong impacts.

### **Storage**

- If the device is not to be used for a long time, please remove the batteries from the device (leaking of battery acid can cause the device to malfunction).
- Always store the unit in the storage case after use. It is intended to be transported or stored in a carrying case between uses.
- Do not place the device directly under sunlight, in high temperature, or in humid or dusty places.

Do not store the device in extremely low (less than -13 °F/-25 °C) and high (more than 158 °F/70 °C) temperature, nor in a place where its humidity exceeds 93% R.H.

### **Maintenance**

- Use a piece of cloth with water or mild cleansing agent to wipe the device and dry it immediately with a dry cloth.
- Do not use detergent or any strong chemicals to clean the device.
- Disinfection - Use a piece of cloth with 75% alcohol or a dilution of neutral detergent to wipe the surface of the cuff for 10 seconds.
- Make sure the cuff is completely dry before using.
- Do not attempt to disassemble or change any parts of the monitor, including wrist cuff, due to substitution of a component different from that supplied might result in measurement error.
- If any suggestion or service is requested, please consult your service station.
- Do not implement the maintenance procedures for equipment during measurement.
- Only trained technicians are allowed to repair and disassemble the device,

including software upgrades, patches and maintenance.

#### **\* Note!**

***Water quality required for cleaning and disinfection: Tap water.***

## TROUBLESHOOTING

SYMBOLS / SYMPTOMS	CONDITIONS / CAUSES	INDICATION / CORRECTION
Unit does not turn on when button is pushed. <b>START STOP</b>	Worn-out batteries.	Replace them with 2 new AAA (LR03) alkaline batteries.
	Battery polarities have been positioned incorrectly.	Re-insert the batteries in the correct positions.
Measuring Error Symbol appears when blood pressure value displayed is excessively low or high. <b>EE</b>	Cuff has been placed incorrectly.	Wrap the cuff properly so that it is positioned correctly.
	Did you talk or move during measurement?	Measure again. Keep wrist steady during measurement.
	Shaking of the wrist with the cuff on.	
Measuring Error Symbol <b>E1</b>	Air circuit abnormality. Cuff tube may not be plugged into monitor correctly.	Check cuff connection. Measure again.
Measuring Error Symbol <b>E2</b>	Inflation pressure exceeding 300 mmHg.	Switch the unit off, then measure again.
Measuring Error Symbol <b>E3</b>	Can't determine blood pressure measurement data.	Wrap the cuff properly and keep steady. Measure again.
Note: If "EP" appears on the display, just return the device to your local distributor or importer.		

SYMBOLS / SYMPTOMS	CONDITIONS / CAUSES	INDICATION / CORRECTION	
BPM cannot communicate with Bluetooth device	Paring has not been completed.	Please re-pairing the BPM and Bluetooth device with each other.	
	Bluetooth function is not turned on.	Please refer to Page 24 "Measurement Procedure" and Page 30 "Bluetooth Transmission" to turn on the Bluetooth function.	
	The distance between BPM and Bluetooth device is out of transmitting range.	Please make sure the acceptable distance ( $\leq 10$ meters) with each other.	
	Use an incompatible Bluetooth device.	Please refer to Page 18 "Bluetooth compatibility" & Page 38 "RF Specification".	
	Use non-Bluetooth device.		
	Unexpected loss of electrical/mechanical integrity.		Re-insert the batteries and try again.
			Return the device to your local distributor or importer.
Note: If "EP" appears on the display, just return the device to your local distributor or importer.			

## WARRANTY & RECALIBRATION

### *Warranty For Two Years from the manufacturing date*

Please note that this warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of importers or distributors.

### *Recalibration Notice*

To ensure continued measurement precision, all digital blood pressure monitors

require recalibration regularly. After 2 years from the manufacturing date, we recommend to have your monitor recalibrated at the local distributor or importer. The recalibration service plus the charge of shipping and handling fee shall be charged accordingly.

## SPECIFICATIONS

<b>Model Number</b>	MOCACuff-Black
<b>Measurement Method</b>	Oscillometric
<b>Rated Range of Cuff Pressure</b>	0-300mmHg
<b>Rated Range of Determination</b>	40 - 280 mmHg
<b>Measurement Range of Heart Rate</b>	40 - 199 beats/minute
<b>Accuracy</b>	Pressure: $\pm 3$ mmHg Pulse: $\pm 5$ % Max.
<b>Inflation</b>	Automatic Inflation (Air Pump)
<b>Deflation</b>	Automatic Air Release Control Valve.
<b>Display</b>	Liquid Crystal Display
<b>Memory</b>	99 Memory Total for one user.
<b>Unit Dimensions</b>	74 X 74.5 X 30.5 mm (L X W X H) 2.91 X 2.93 X 1.20 inch (L X W X H)
<b>Unit Weight(Cuff &amp; BatteriesExcluded)</b>	97.5g $\pm$ 10g (3.44oz $\pm$ 0.35oz)
<b>Cuff Size</b>	135 - 195 mm (approx. 5.3 - 7.7 inch)
<b>Storage/</b>	Temperature: -25 °C - 70 °C (-13 °F - 158 °F)
<b>Transportation Environment</b>	Humidity: $\leq 93$ % R.H.
<b>Operation Environment</b>	Temperature: 5 °C - 40 °C (41 °F - 104 °F) Humidity: 15 % - 93 % R.H. Atmospheric pressure: 700 hPa - 1060 hPa
<b>Power Supply</b>	DC 3 V, AAA "LR03" (1.5 V) Alkaline Battery x 2
<b>Battery Life</b>	Approx. 200 Measurements
<b>Product Life</b>	5 Years (4 times per day)
<b>Sleeping Mode</b>	Without any operation for 1 minute, device automatically shuts off.
<b>Accessories</b>	Instruction Manual, 2 AAA (LR03) Alkaline Batteries, Storage Case

The contents of this manual and the specifications of the device covered by this manual are subject to change for improvement without notice.

<b>RF Type</b>	Bluetooth 4.0 BLE
<b>RF Modulation</b>	GFSK
<b>Effective Radiated Power</b>	0 dBm
<b>Data Throughput</b>	0.2 Mbps
<b>Expected Delay (Latency Range) in Wireless (RF) Communication Integrity</b>	The latency time is less than 0.3 ms second from sender to receiver.
<b>Security</b>	Channel Quality-Driven Data Rate (CQDDR) technology increases the effective data rate and integrity in noisy environments.
<b>Wireless Operation Distance</b>	AES-128 and application layer user defined Class 2 [Maximum: 10 meter]
<b>RF Frequency /Need for Spectrum Management</b>	2402 – 2480 MHz (allowing for guard bands)
<b>Maximum Limitation</b>	Unlimited
<b>Maximum Permitted Power</b>	5 mW
<b>Proximity of Other In-band Transmitters Used in Vicinity</b>	Up to 40 bands [2 MHz spacing; centered from 2402 to 2480 MHz]
<b>Wireless Communication Profile</b>	GATT – Client and Server
<b>Wireless Coexistence</b>	Support for 802.11 Coexistence
<b>System requirement of the Bluetooth device</b>	Bluetooth 4.0 for Android 4.3 or above Bluetooth 4.0 for iOS 7.0 or above

The contents of this manual and the specifications of the device covered by this manual are subject to change for improvement without notice.

## NOTE

This blood pressure monitor complies with the EC Directive [93/42/EEC] and bears the CE mark. This blood pressure monitor also complies with mainly following standards (included but not limited): Safety standard: EN 60601-1 Medical electrical equipment part 1: General requirements for safety. EMC standard: EN 60601-1-2 Medical electrical equipment part 1-2: General requirements for safety- Collateral standard: Electromagnetic compatibility-Requirements and tests. Performance standards: EN 1060-1 Non-invasive sphygmomanometers - General requirements EN 1060-3 Non-invasive sphygmomanometers - Supplementary requirements for electromechanical blood pressure measuring systems EN 1060-4 Non-invasive sphygmomanometers - Test procedures to determine the overall system accuracy of automated non-invasive sphygmomanometers

EN ISO 81060-1 Non-invasive sphygmomanometers–Part 1: Requirements and test methods for non-automated measurement type (partially applied) IEC 80601-2-30 Medical electrical equipment—Part 2-30: Particular requirements for basic safety and essential performance of automated non-invasive sphygmomanometers



Follow instructions for use.



BF Classification:

- Internally powered equipment
- BF type applied part
- IP22-Degrees of protection provided by enclosures
- Not suitable for use in presence of flammable anesthetic mixture with air or with Oxygen or nitrous oxide
- Continuous operation with short-time loading



To avoid inaccurate results caused by electromagnetic interference between electrical and electronic

equipments, do not use the device near a mobile phone or microwave oven. At least keep a maximum output power of 2 W yields and a distance 3.3m away from this equipment.



Discard the used product to the recycling collection point according to local regulations. Do not dispose the product into household wastes.



EMERGO EUROPE

Prinsessegracht 20, 2514 AP The Hague, The Netherlands



Serial Number



**Manufacturer for:**

HEALTH & LIFE CO., LTD.

9F, No. 186, Jian Yi Road, Zhonghe District, New Taipei City, Taiwan.

www.healthandlife.com.tw

**Distributed by:**

MOCACARE Corporation

887 Federation Way,

Palo Alto, CA 94303 USA

mocacare.com

**\* NOTE**

*This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.*

*This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment*

*does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:*

*The user is encouraged to try to correct the interference by one or more of the following measures:*

- 1. Reorient or relocate the receiving antenna.*
- 2. Increase the separation between the equipment and the receiver.*
- 3. Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.*
- 4. Consult the dealer or an experienced radio/TV technician for help.*

**• CAUTION:**

*To assure continued FCC compliance:*

- 1. Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the equipment.*
- 2. This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance 20cm between the radiator & your body.*

**FCC Label Compliance Statement:**

*This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:*

- 1. this device may not cause harmful interference, and*
- 2. this device must accept any interference received,*

*including interference that may cause undesired operation.*

**\* NOTE**

*“Changes or modifications not expressly approved by the manufacturer could void the user’s authority to operate the equipment”.*

MOCACuff essential performance per IEC 80601-2-30 additional essential performance requirements:

- 201.12.1.102 Limits of the error of the manometer from environmental conditions Over the temperature range of 5 °C to 40 °C (41 °F ~ 104 °F) and the relative humidity range of 15 % to 93 % (non-condensing), the maximum error for the measurement of the CUFF pressure at any point of the NOMINAL measurement range shall be less than or equal to ± 3 mmHg (± 0.4 kPa) or 2 % of the reading, whichever is greater.


- 201.12.1.107 Limits of the change in error of the blood pressure determination. The laboratory limits of the change in error of the BLOOD PRESSURE DETERMINATION of the AUTOMATED SPHYGMOMANOMETER shall be less than 3 mmHg (0.4 kPa).

## APPENDIX

Guidance and manufacturer’s declaration – electromagnetic emissions		
The device is intended for use in the electromagnetic environments listed below, and should only be used in such environments:		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	RF energy is used only to maintain device’s operation. Therefore, its RF emissions are so low that it’s not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments, and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not Applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not Applicable	

Guidance and manufacturer's declaration – electromagnetic immunity			
The device is intended for use in the electromagnetic environments listed below, and should only be used in such environments:			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact discharge ±15 kV air discharge	±8 kV contact ±15 kV air	The relative humidity should be at least 5%.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m 50 or 60 Hz	30 A/m 50 or 60 Hz	<p>Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. Recommended separation distance</p> $r = \frac{I}{188} \text{ (m)}$ <p>where <math>I</math> is the current in amperes in a power bus or an appliance wire and <math>r</math> is the recommended separation distance between your device and the power bus or appliance wire, in meters (m).</p>

Recommended separation distances between portable and mobile RF communication equipment and the device.			
The device is intended for use in an electromagnetic environment where radiated RF disturbances are under control. User can help prevent electromagnetic interference by keeping the device at a minimum distance from portable and mobile RF communications equipment (transmitters). Below table details the maximum output power of transmitter:			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter(m)		
	150 kHz to 80 MHz Not applicable	80 MHz to 800 MHz d = 1.2 √p	800 MHz to 2.5 GHz d = 2.3 √p
0.01	NA	0.12	0.23
0.1	NA	0.38	0.73
1	NA	1.2	2.3
10	NA	3.8	7.3
100	NA	12	23
<p>For transmitters rated at a maximum output power not listed above, the recommended separation distance <math>d</math> in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.</p> <p>NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			

Guidance and manufacturer's declaration – electromagnetic immunity			
The device is intended for use in the electromagnetic environments listed below, and should only be used in such environments:			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms At 0.15-80 MHz 6 Vrms At ISM & Radio Amateur Freq.	Not Applicable	Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  Recommended separation distance  $d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts [W] according to the transmitter manufacturer and d is the recommended separation distance in metres [m]. Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.
Radiated RF IEC 61000-4-3	3 V/m at 80-2700 MHz [10V/m Home Healthcare] AM Modulation And 9-28V/m at 385 -6000 MHz, Pulse Mode and other Modulation.	3 V/m at 80-2700 MHz [10V/m Home Healthcare] AM Modulation And 9-28V/m at 385 -6000 MHz, Pulse Mode and other Modulation.	Interference may occur in the vicinity of equipment marked with the following symbol: 

<ul style="list-style-type: none"> <li>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</li> <li>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</li> </ul>
<p>Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.</p>





Customer service [support@mocacare.com](mailto:support@mocacare.com)

More information please visit [mocacare.com](http://mocacare.com)

P/N: 22123 Rev.D (ATOM)

P/N: 323103159 VER:001 (HL)



MOCACARE