



KneeRover[®] Evolution Scooter

User Manual

CE

Congratulations on your purchase of the KneeRover[®] Evolution Scooter!

The versatile KneeRover[®] Evolution seated scooter provides a new level of freedom to users recovering from knee or leg injuries. The large gel padded seat and convenient front leg stand allows the patient to rest in comfort while elevating the non-weight bearing leg. Increase your mobility without the use of painful crutches.

Operating Instructions:

This KneeRover[®] scooter is designed to maximize mobility during your recovery from foot/ leg surgery or injury.

It is important to set the seat and handlebar at a comfortable and safe operating position. The best position for the handlebar is approximately at hip level, with the seat at a comfortable height for propelling the KneeRover[®] scooter and resting your injured leg.

Ensure that the hand brake is fully engaged and locked before mounting the KneeRover[®] scooter. Place your hands on the handgrips and while seated, place the heel of the injured leg on the leg rest. Now release the parking brake and propel yourself using your good leg. Allow the KneeRover[®] scooter to glide, and as it slows down, push again.

Safety First:

The KneeRover[®] Evolution scooter is designed for easy steering and stopping and should be used to carefully move about at slow, controlled speeds. When turning the scooter, you should have the brake applied and be moving at a slow, safe, controlled speed. It is easy to tip the scooter if you are making a turn with too much speed. Use caution and balance and operate at your own risk.

With routine use, nuts and bolts on the steering column and brake mechanism can and likely will become loose. Periodically check nuts and bolts after use and re-tighten as appropriate.

Evolution Set Up Instructions:

Please follow the step by step instructions on the following pages to properly assemble your KneeRover® Evolution scooter. The diagram below identifies the key components of the scooter to aid with assembly. [Visit kneerover.com for assembly videos.](http://www.kneerover.com)

Weight Capacity = 300 pounds



Setup Instructions:

Step 1: Install Handlebars & Steering Column

Please **carefully complete all steps** before operating your new KneeRover® scooter. The steering column and handlebars will be separate and in a folded position when unpacking. Insert the handlebars into the steering column, making sure not to twist the brake cable. Next, raise the steering column and push the clamp lever to the left to allow security pin to lock into place (as shown below). Raise the clamp lever upward and tighten by turning it clockwise. Finally, push clamp lever down firmly to secure the steering column (as shown below). Reverse these steps when needing to lower steering column for transport.




Push Clamp Lever to Left

Step 2: Rotate Front Wheels & Set Guide Pin

Pull down the guide pin ring as shown below, and turn the handlebars so that the front wheels and brake handles are facing forward. **The BACK sticker MUST be facing back toward the rear wheels with the guide pin between the guide stops.** (see photo on top of next page). Be sure that the brake cables are not twisted. The front of the scooter should look similar to the picture on the previous page.




Guide Pin Ring

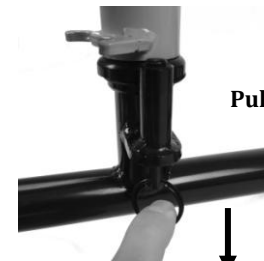


Pull Ring Down

Step 2: Rotate Front Wheels & Set Guide Pin (cont.)

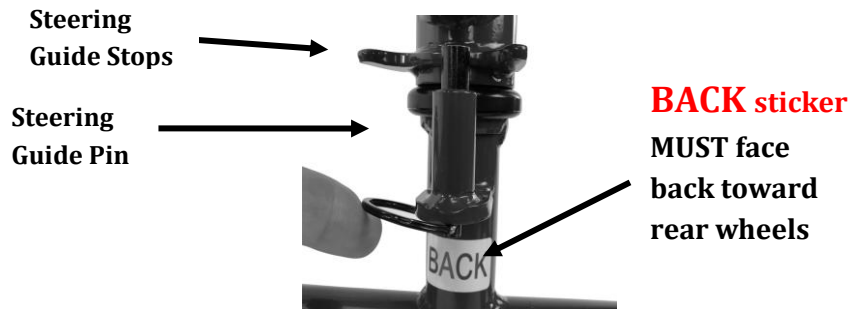

Step 3: Install Seat & Attach Basket

The KneeRover® Evolution scooter is designed to be used for right or left leg injuries. To setup the seat and/or adjust the height, remove the locking pin and place the seat at the desired height then reinsert the pin. Pull the clamp lever downward so the end of the lever is pointing towards the ground to lock the seat in the desired position (rotate the lever counter clockwise to loosen and clockwise to tighten the pressure on the lever). Removable basket attaches to the steering column.

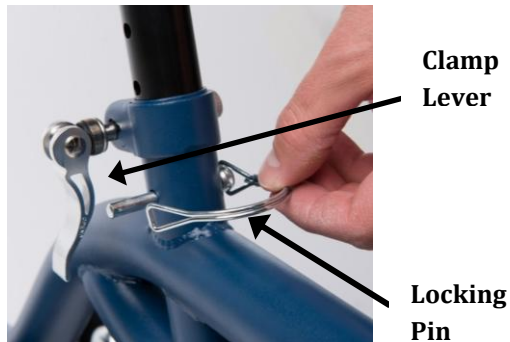

Step 4: Brake Operation

To engage the hand brake, squeeze the brake lever on handle bar.

The KneeRover® Evolution scooter comes equipped with a brake lock / parking brake. Simply squeeze brake handle firmly and press the button to engage the lock. To release the lock, squeeze the brake handle again.



Ongoing Maintenance

With routine use, nuts and bolts on the steering column and brake assembly can become loose. To ensure safe operation of this KneeRover® scooter it is important to periodically inspect the knee walker for loose nuts and bolts and re-tighten as appropriate.

Please visit [kneerover.com/pages/kneerover-maintenance-videos](https://www.kneerover.com/pages/kneerover-maintenance-videos) for additional support in maintaining your KneeRover.

Your Satisfaction is Our Top Priority:

Thank you for your business – we appreciate having you as a customer! We are committed to providing innovative and affordable mobility solutions to get you moving as soon as possible after injury or surgery and for the duration of your recovery – free to go about your busy, active life – and have a little fun doing it.

We hope that you are completely satisfied with your purchase. If you have any issues, please contact us and we will do everything possible to help you. Reviews have a significant impact on our small business. If you purchased this product on Amazon or eBay, we would greatly appreciate a review once you've had a chance to use your new KneeRover® scooter. We hope we've earned a 4 or 5 star review – if not, please give us the opportunity to help you before leaving a review. Thank you for your support. We appreciate you!

Need Help – Contact Us:

Phone: 877-737-9877

Email: help@kneerover.com

Web: www.kneerover.com

Assembly Support: kneerover.com/pages/assembly-video

EU AUTHORIZED REPRESENTATIVE

Murray's Medical Equipment Ltd
Airton Park, Airton Road, Tallaght,
Dublin 24 (Eircode D24 DY09), Ireland
T: 00 353 1 855 5696
www.murrays.ie



KneeRover® is a registered trademark of Rover Mobility, LLC.

Limited Lifetime Warranty

This warranty applies only to the original purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. Your KneeRover® brand product is warranted to be free of defects in materials and workmanship for a period of one year from the date of purchase. The side frames and cross braces are warranted for the lifetime of the KneeRover® scooter.

This warranty does not cover device failure due to owner misuse, negligence, or normal wear and tear. The warranty does not extend to non-durable components, which are subject to normal wear and need periodic replacement. These components include, but are not limited to tires, grips, brake pads and upholstery.

The warranty shall not apply for damages or defects caused by unauthorized parts, attachments or repairs, negligence, accident, or any circumstances beyond the control of KneeRover®, such as improper operation, maintenance or storage.

If you have any questions regarding your KneeRover® product or this warranty, please contact our Customer Care Team. We are available, for your convenience, toll free at 1-877-737-9877, Monday – Friday from 9 am to 5 pm Eastern.

Need Parts or Accessories?

Add an accessory like a knee pad cover, cup holder or headlight to enhance your experience on your new KneeRover® scooter. Keep your scooter operating in top shape with original manufacturer parts. Visit kneerover.com, to explore our full range of parts and accessories.