

MODVEL ANKLE BRACE

Safety Information

The Modvel Ankle Brace is specifically designed to support and enhance recovery from various foot and ankle conditions including plantar fasciitis, Achilles tendinitis, arthritis, swelling, ankle sprains, and post-surgery recovery.

Its sleek, comfortable design provides reliable compression to stabilize joints, support mobility, and alleviate discomfort.

FEATURES AND BENEFITS:

- Targeted Compression:** Effectively reduces swelling, eases pain, and promotes better blood circulation for faster recovery.
- Breathable Material:** Made from sweat-wicking, breathable fabric to keep your feet comfortable, dry, and odor-free, even during extended wear.
- Innovative Knit Technology:** Uses advanced weaving technology to ensure durability, comfort, and consistent compression, maintaining shape and support even with prolonged use.
- Lightweight & Comfortable:** Easy to wear throughout the day, seamlessly fitting into most footwear without bulk.

HOW TO USE PROPERLY:

- Slide your foot gently into the ankle sleeve, ensuring your heel aligns naturally with the brace's heel area.
- Adjust the brace to evenly cover your ankle, ensuring a smooth fit without wrinkles.
- Confirm the brace provides a secure yet comfortable fit, supporting your ankle without causing discomfort or restricting circulation.

Proper use ensures optimal support and effectiveness.

CARE INSTRUCTIONS:

Maintain your Modvel Ankle Brace with care:

- Machine Wash: Warm water (up to 40°C)
- Do Not Bleach
- Do Not Iron
- Do Not Tumble Dry (Air drying is recommended to preserve elasticity and durability.)

PRODUCT SPECIFICATIONS:

- Materials:** Premium blend of Polyester and Rubber to ensure optimal durability, flexibility, and comfort.
- Size:** Adjustable sizing to accommodate various foot & ankle sizes (refer to included measuring guide to find your perfect fit).
- Design:** Sleek, slip-on design (no straps) optimized for daily use and effortless mobility.

SAFETY PRECAUTIONS:

- Consult a Physician:** Before prolonged or continuous use, consult your physician, especially if you have existing medical conditions or concerns.
- Proper Fit:** Avoid overly tight application to prevent restricted blood flow. Always ensure the brace fits snugly yet comfortably.
- Skin Safety:** Only use on clean, intact skin. Do not use if you have allergies or sensitivities to Polyester or Rubber.
- Irritation or Discomfort:** Discontinue use immediately if you experience pain, numbness, irritation, or discomfort, and consult a healthcare professional.
- Not for Children:** Keep the brace out of reach of children. This product is designed specifically for adult therapeutic use only.

CERTIFICATIONS & DISCLAIMER:

The Modvel Ankle Brace is carefully crafted from high-quality materials, providing reliable and safe compression. The manufacturer is not responsible for improper use or incorrect application of the product. Always follow these guidelines and consult a healthcare professional for specific, personalized advice.

70%
NYLON

20%
LATEX FIBER

10%
SPANDEX

MADE IN
CHINA

DESIGNED IN
THE USA