2 WAYS

1) Using a Lansinoh® Pump:

Pump directly into bags following instructions provided below.

2) For use with Other Pumps: Use with Breastmilk Storage Adapters for pumping directly in Lansinoh® Breastmilk Storage Bags (compatible with most Medela®, Ameda®, Evenflo® or other standard neck pumps)

Visit Lansinoh.com for information on where to buy

IMPORTANT! Ensure that your Lansinoh® Breastmilk Storage Bag has white and purple flowers and a white line above the logo as shown in the image. If your bag does not have this design, it is not compatible with pumping directly into the bag.

NOTE: Pumping directly into bags requires two hands to safely remove milk storage bags. For best results use with a hands free pumping bra such as the Lansinoh Simple Wishes bra.

Directions for use with Lansinoh Pumps

Use Lansinoh® Pump to pump directly into Lansinoh® Breastmilk Storage Bags (compatible with Signature Pro®, Smartpump™ or Lansinoh Manual Breast Pump)

1) Open Breastmilk Storage Bag by tearing away tamper-evident safety seal and pulling zipper seal apart.

2) To insert expression set:
a) Tilt expression set as shown.

b) Insert one side of the expression set to align with the printed white line and flower.

c) Gently pull the other side of bag up over expression set so the clear zipper is above the expression set on all sides.

3) Gently pull the bag down until the expression set is just below the clear bottom zipper all the way around the bag. Tug gently on bottom of bag to ensure bag is secure.

4) If working with a double pump, repeat steps 1-3.

5) Pump as usual.

6) Securely hold the bag and gently push the zipper seal past expression set until the bag releases.

Maintaining a firm grip on bag, tilt and remove expression set. Carefully remove as much air as possible and seal bag.

NOTE: Do not fill past the 6oz. mark.

For use with Other Pumps

Use with Breastmilk Storage Adapters for pumping directly in Lansinoh® Breastmilk Storage Bags (compatible with most Medela®, Ameda®, Evenflo® or other standard neck pumps)

Visit Lansinoh.com for information on where to buy

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