



Help Manual


Hip Replacement Recovery Kits

Our Hip Kits are most often recommended by therapists for post hip, knee, back surgery or recovering from leg injury. We are your destination for Hip Kit, An Everyday Essential for Independent Living.

Reacher Grabber


1. This grabber reacher is not convenient for installing the bulb. Please don't use it to install bulbs.
2. The grabber reacher can pick up items within 5 pounds, but It is recommended to use the grabber reacher to pick up about 2 pounds of items.

Fold & Open

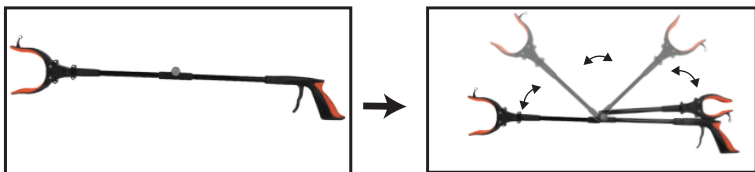

Attention: Using proper force to open and fold the extractor will not cause damage to the product.

Tied up

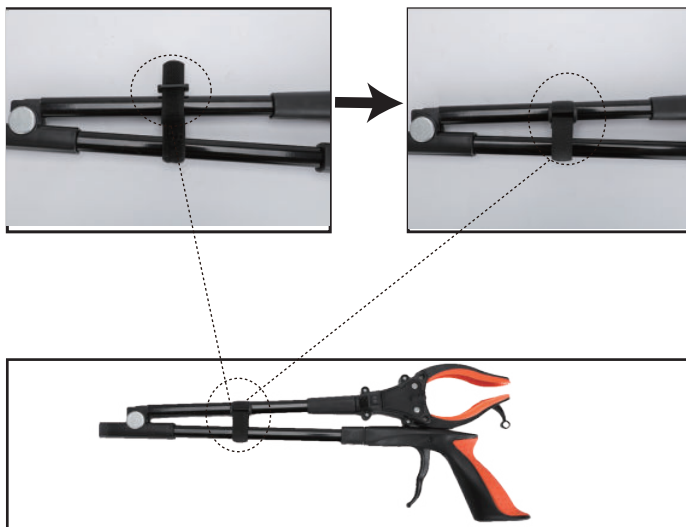

33" Long Dressing Stick



Make Getting Dressed Easier



Sock Removal Tool



Long Shoe Horn

Sock Aid



Easy Steps to Put On Your Socks:



STEP 1

Step 1: Pull sock or stocking over the end of the RMS Sock Aid.



STEP 2

Step 2: Place Sock Aid in front of your foot on the floor.



Step 3: Insert toes into sock or stocking opening.



Step 4: Pull straight back until toes are well into sock or stocking and the Sock Aid is out of sock or stocking.

EC	REP
----	-----

E2EURO CONSULTING HUNGARY (for Prods
authorities only)

Hoferstasse 9B, 71636 Ludwigsburg, Germany

UK	REP
----	-----

Business name: GSG CONSULTING GROUP LIMITED

Business address: Unit 3 Matthew street Gorton Manchester



Shenzhen Six Yi Technology Co.,Ltd



support@kekoylife.com



No.219, Erqi Building, Shouzaowenhua Road, Longgang
District, Shenzhen, Guangdong, China 518129

Made in China



RoHS

