DOMICILE - HOMECARE

CLIPPER

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EN - Instructions for installation and use

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HERDEGEN SAS,
6 rue Marc Seguin
77500 CHELLES – France

Copyright 2024 par HERDEGEN SAS– ALL RIGHTS RESERVED

In the interests of continuous improvement, HERDEGEN SAS reserves the right to change any detail or specification in this document without prior notice.

The information contained in this document is confidential. It may only be shared with a third party with the prior written consent of HERDEGEN SAS. This document may not be reproduced or communicated, in full or in part, in any way or by any means, whether technical or electronic, by photocopying, recording, or using any information or data extraction systems.

You can obtain a new copy of this document from HERDEGEN SAS by contacting your regional sales representative or directly contacting HERDEGEN SAS.

First edition printed in: 2024
Printed in France on certified paper

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FOREWORD

You have just purchased a HACARE product and we are pleased to have you as one of our clients. We also thank you for placing your trust in our company and our products.

Your trust assures us that the HACARE company continues to meet your specific requirements and needs. That's why we always strive to innovate, providing you with quality solutions and products that improve the comfort and quality of patient care while making the caregiver's daily tasks easier.

HACARE products have gained the confidence of many professionals and institutions due to their functional and aesthetic qualities, and are now considered to be a solid benchmark. Simplicity, reliability and safety are the hallmarks of our electrical healthcare cradles.

You've chosen HACARE, and we're committed to satisfying your needs.

However, you are responsible for the proper use of the cradles. That's why we invite you to familiarize yourself with the technical specifications, and the instructions for use and maintenance of the equipment by reading through this manual.

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EN - Please read these safety instructions carefully before use and keep them for future question.

1. SAFETY INDICATION, ABNORMAL USES, DANGERS

- Always consult with your physician or supplier to determine proper adjustment and use of the device.
- Do not lean out of the raised seat to avoid falling out of the device.
- Do not seat or step on the lid.
- Don't store outside and don't let under the sun (beware: don't let

the product in a car in case of high outside temperature).

- Don't use outside but only inside and on standard toilet bowls.
- Check carefully all parts of this item (screws, tubes, clips..) And if all screws are tightened.
- Use only as a raised toilet seat and in proper way : don't knee or step on the product, don't seat on the lid.
- Always observe the weight limit on the labelling of your device.
- No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- After assembling the product, make sure that it is securely locked and fixed before using.
- In case you didn't use your raised seat for a long time, let it checked by the technical staff, before using.
- Always check stability before using.

- Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the member state in which the user and/or the patient is established.

- The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

2. INTENDED USE, INDICATIONS AND CONTRAINDICATIONS

Intended use:

A raised toilet seat is an attachment to an existing toilet which adds extra height for the user. It is designed for those with limited mobility who have difficulty sitting down or standing up from the toilet.

Indication

- Decreased joints (knee or hip) and spinal mobility
- Difficulty in standing up and seating making impossible the use of a standard toilet.(Clipper I/II/III-500400-500410-500411).
- Maintaining a correct seating position and lifting aid thanks to the armrests (Clipper IV/V-500420-500421).
- When an height adjustment is necessary (Clipper VI/VII-500430-500431).

Contraindication

To be determined by the prescribing physician based on the patient's condition (such as severe cognitive disorder, severe loss of balance, sitting impairment). Patient weight is greater than the product's supported weight.

3. INSTALLATION

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually by the technical staff to suit your needs.

Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

Those items (except ref 500400) are equipped with clips enabling a tight fixation on the toilet bowl.

The references 500420 and 500421, Clipper 4 and 5, are equipped with armrests which offer extra support to the user. However, if you suffer from a severe disability and need a strong support, you should use the ref 500430 and 500431. Do not use the product if you see any risk.

Ref. 500400

1-Lift the toilet seat and lid.

2-Position the raised seat correctly on rim of toilet.

3-This product isn't fixed on the toilet, so it can be instable. In case of doubt ask the technical staff.

Ref. 500410 et 500411 see [A]

1-Lift the toilet seat and lid.

2-Adjust the clips at the required position (1), place the seat onto the toilet bowl (2). Position raised toilet seat correctly on rim of the toilet and push the blue brackets (first the ones located at the front) tight to the bowl (3).

3-Check the stability before using.

4-Simply remove the raised seat by lifting up. Restart from point 1 to install again.

Ref. 500420 et 500421 see [B]

1-Proceed as for ref 500410/411.

2-The arms can be rotated backwards : first press the pushbutton located under the armrest (1), then push the armrest on side (2) and fold up the armrest (3).

Ref. 500430 et 500431 see [B]+[C]

1-Proceed as for ref 500410/411.

2-Adjust the legs with the push buttons. All legs must be leveled to the ground.

Rental:

This product can be rented to different patients. A thorough maintenance, cleaning and disinfection must be performed before the product is given to a new patient in accordance with the following steps:

1- Visual inspection: ensure that the product has not been damaged and check for any defect.

2- Functional inspection: ensure that the product functions properly and repair it if needed.

3- Cleaning and disinfection: this product can be cleaned and disinfected with mild, chlorine-free detergents such as Survanios (Anios laboratories).

Do not use any abrasive cleaners or high-pressure cleaners that could damage the product. Use only appropriate tools to perform the maintenance, cleaning and disinfection. The user's manual is an integral part of the product and must be handed over to every new user.

4. CLEANING

The raised seat has to be cleaned regularly with a mild cleaning agent and a soft cloth. It may be washed with soap and water or disinfected with non-chlorinated products. Do not use any abrasive cleaners or high-pressure cleaners that could damage the product.

5. DISPOSAL

Dispose of this product in accordance with your regional waste disposal regulations.

6. TECHNICAL SPECIFICATIONS

See table page 6. This product is biocompatible. Duration of use : 2 years.

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