




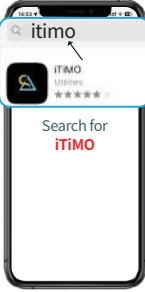
# EARWAX REMOVAL TOOL PRODUCT MANUAL

## Download for Apple users


1 Step 1: Find the **App Store**




2 Step 2: Search for **iTIMO**  
Click to download the app



3 Step 3: Return to the desktop and open the APP




4 Step 4: APP display interface




## Android users download


1 Step 1: Find the **Google Play**

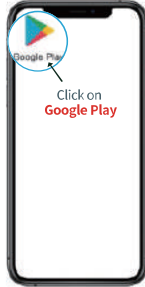


2 Step 2: Search for **iTIMO**  
Click to download the app



3 Step 3: Click to download the app



4 Step 4: After the download is complete. Click to open the APP.

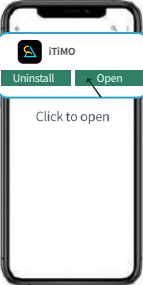

## Connection Flow


1 First of all, when you get the product, ensure sufficient power, press switch key to open




2 Open phone settings WiFi Function, Find iTIMO-xxxx




3 Connect iTIMO-xxxx

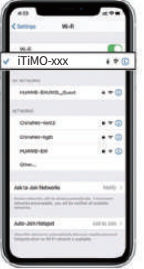


4 Open the APP click the start




5 If you encounter this prompt, please select "Allow", otherwise you will not be able to continue using it.



6 If you encounter this prompt, please select "Ok", otherwise you will not be able to continue using it.



7 Introduction to the functions of icons



- Zoom in and out
- Photograph
- Photo album
- Video recording
- Open front camera
- Horizontal flip

8

If you encounter this prompt, please select "Keep Trying WLAN", otherwise you will not be able to continue using it.



### Ear Pick Installation

Insert the ear scoop cover vertically into the ear scoop head, and use it when it is equipped in place



Note: It must be clamped. Avoid falling for use.



Note: Only the camera part waterproof other parts are not waterproof

### Product Accessories

1

Replaceable silicone ear scoop cover



2

Type-c charging cable\*1charge the product extremely



### Product Parameter

Product Name	Earwax Removal Tool
Weight	18g
Network Standard	IEEE802.11b/g/n
Working Frequency	2.4GHz
Image Transmission Rate	30Fps
Image Sensor	CMOS
Operating Temperature	-10-40°C
Battery	250mAh
Input Current	DC 5V 0.4A
Best Focal Length	1.5-2CM
Gravity Sensor	Ultra HD Camera
Connect Hotspot	iTIMO-xxxx
LED	6"LED
Waterproof lens	IP67

### Precautions

- When cleaning the lens, please wipe it carefully with a professional alcohol swab.
- When the image is not clear, wipe the camera with a cotton swab.
- When you are moving, please do not use it and protect it from being hit by others.
- This product is not suitable for children under 3 years old.
- Avoid direct exposure of the product to the sun, especially the ear scoop cover, to avoid softening.
- Do not use the machine body in liquid to avoid damage caused by immersion.
- Avoid direct exposure of the product to the sun, especially the accessories, to avoid softening.
- During use, the temperature of the product will rise slightly (up to 35°C), please use safely.
- In order to ensure the best user experience, please ask your child to protect the sitting position.
- To keep the ear canal clean and safe, replace the ear scoop when the head of the ear scoop is damaged for more than 30 times.

- Long press for 3 seconds to shut down.
  - Light description: red light is on when charging, and light is off when it is full. Slow flashing means no connection, fast flashing means no power, long blue light means it is Normal link.
- PS: Slow flashing and fast flashing refer to the normal opening (not charging) Case.
- Wipe the lens with alcohol or water, and do not let the body get water.
  - Do not disassemble, modify or repair this product by yourself.

### Hazardous Material

Part name	(PB)	(Hg)(Ca)	(Cr)(V)	(PBB)	(PBDE)
Printed circuit components	○	○	○	○	○
Plastic shell	○	○	○	○	○
Plastic rubber accessories	○	○	○	○	○

This form is compiled in accordance with SJ/T11364  
 ○ indicates the content of harmful substances in this book  
 Partly below the requirements specified in GB/T26572-2011

### Troubleshooting

Malfunction	Solution
Light does not turn on after opening no response	1. Press the power button to turn on 2. Please use the charging cable to charge to ensure sufficient power

Malfunction	Solution
APP quickly exit	Delete the existing APP and install it
The ear pick accessory tool is not installed smoothly	First, align the ear muff with the lens spoon. Cover the outer ring of the camera again to ensure that it is installed to the end
Cannot see image, the image connection is stuck	Click the return key in the App, then click start, if it is invalid, press the power button again to reconnect
The phone cannot connect to the product	If the device WiFi cannot be connected, but the button and camera indicator light are always on, it means the device is connected to other phones. Please disconnect the WiFi connection of the other mobile phone at this time, the blue light flashes. Retry the connection after flashing

Q: Can't use the product after WIFI connection is completed? (FIGURE 1)




- If you are using VPN, please turn off the VPN service and try again.
- If you can't solve the problem after turning off the VPN, please follow the instructions below for setting.



Step 1: Turn on the WIFI and find the wireless link



Step 2: Click the circled icon

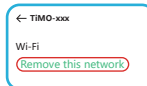

Step 3: Choose to Remove this network and reconnect

- If it still can't be solved, please uninstall the software and install it again.