

## Kniebandage / Knee support

<b>DE</b> Gebrauchsanweisung .....	4
<b>EN</b> Instructions for use .....	8
<b>FR</b> Instructions d'utilisation .....	13
<b>IT</b> Istruzioni per l'uso .....	18
<b>ES</b> Instrucciones de uso .....	22
<b>PT</b> Manual de utilização .....	27
<b>NL</b> Gebruiksaanwijzing .....	32
<b>SV</b> Bruksanvisning .....	37
<b>DA</b> Brugsanvisning .....	41
<b>NO</b> Bruksanvisning .....	46
<b>FI</b> Käyttöohje .....	50
<b>PL</b> Instrukcja użytkowania .....	54
<b>HU</b> Használati utasítás .....	59
<b>CS</b> Návod k použití .....	63
<b>HR</b> Upute za uporabu .....	68
<b>SK</b> Návod na používanie .....	72
<b>TR</b> Kullanma talimatı .....	76
<b>EL</b> Οδηγίες χρήσης .....	81
<b>RU</b> Руководство по применению .....	86
<b>JA</b> 取扱説明書 .....	91
<b>ZH</b> 使用说明书 .....	95

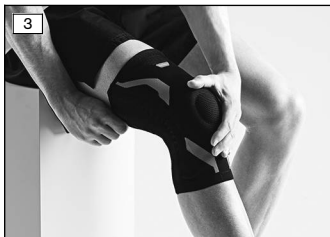	Größe Size	Umfang Circumference			
		A		B	
		cm	inch	cm	inch
	XXS	25-28	9.8-11.0	35-38	13.8-15.0
	XS	28-31	11.0-12.2	38-41	15.0-16.1
	S	31-34	12.2-13.4	41-44	16.1-17.3
	M	34-37	13.4-14.6	44-47	17.3-18.5
	L	37-40	14.6-15.8	47-50	18.5-19.7
	XL	40-43	15.8-16.9	50-53	19.7-20.9
	XXL	43-46	16.9-18.1	53-56	20.9-22.0

<b>Material</b>	Gestrick/Knit fabric: PA, EL
	Frotteebezüge/Terry cloth covers: Baumwolle/Cotton, PES
	Spiralfedern/Spiral springs: verzinkter Federstahl/zinc-coated spring steel, PE
	Pelotte/Pad: viskoelastisches Technogel®/viscoelastic Technogel®

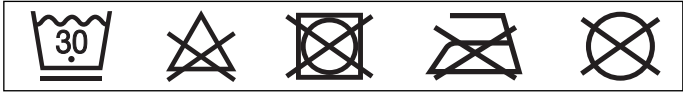# 1 Vorwort

Deutsch

## INFORMATION

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### INFORMATION

Date of last update: 2020-11-25

- Please read this document carefully before using the product and observe the safety notices.
- Please contact the distributor if you have questions about the product or in case of problems.

- |   |
|---|
| <ul style="list-style-type: none"><li>▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.</li><li>▶ Please keep this document for your records.</li></ul> |
|---|

The instructions for use provide you with important information regarding the use of the VoltActive knee support.

## 2 Intended use

### 2.1 Indications for use

The support is intended **exclusively** for orthotic fittings of the knee joint and **exclusively** for contact with intact skin.

The product is intended **exclusively** for use on **one** user. Use of the product by another person is not approved by the manufacturer.

The support must be used according to the indication.

### 2.2 Indications

- Irritation in the knee joint (posttraumatic, postoperative and chronic)
- Swelling in the knee joint (posttraumatic, postoperative and chronic)
- Slight feeling of instability
- Minor ligamentous instability
- Minor knee pain
- Fibromyalgia
- Insertion tendinitis

Indications must be determined by the physician.

### 2.3 Contraindications

#### 2.3.1 Absolute Contraindications

Not known.

#### 2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb; pronounced varicose veins, especially with return flow impairment, lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the support will be applied; sensory and circulatory disorders in the area of the lower leg and the foot, e.g. in case of diabetic neuropathy.

## 2.4 Effects

The form-knitted support promotes sensorimotor functions and thereby activates muscular stabilisation of the knee joint. The support with integrated shape-retaining pad offers relief for the kneecap.

## 2.5 Conditions of use

The product is suitable for everyday and moderate sports activities, such as running, hiking, football, tennis, fitness training and gardening.

## 3 Safety

### 3.1 Explanation of warning symbols

**⚠ CAUTION** Warning regarding possible risks of accident or injury.

**ℹ NOTICE** Warning regarding possible technical damage.

### 3.2 General safety instructions

**⚠ CAUTION**

#### Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

**⚠ CAUTION**

#### Continuous use of the product

Changes in the state of health, worsening of the complaints

- ▶ Consult a physician promptly if you note any unusual changes on yourself.
- ▶ Do not continue using the product in case of doubt.

**⚠ CAUTION**

#### Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

**NOTICE****Contact with oils, salves, lotions or other products that contain oils or acids**

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

**NOTICE****Improper use and changes**

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

## 4 Handling

**INFORMATION**

- ▶ The daily duration of use and period of application are generally determined by the physician.

### 4.1 Size Selection

The support is unisex and suitable for either side.

- 1) Measure the circumference of the thigh 14 cm above the centre of the patella.
- 2) Measure the circumference of the lower leg 12 cm below the centre of the patella.
- 3) Determine the size of the support (see size chart).

### 4.2 Application

**⚠ CAUTION****Incorrect or excessively tight application**

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

#### NOTICE

#### Use of a worn or damaged product

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

- 1) Grasp the support with both hands on the sides and upper end of the coil springs.
- 2) Pull the support over the knee (see fig. 1).
- 3) Position the knee pad so the patella is encompassed without pressure (see fig. 2).
- 4) You can correct the position of the knee pad as an option (see fig. 3).
- 5) Check the fit and correct positioning of the support.

#### 4.3 Removal

- 1) Grasp the support with both hands on the side and on the lower end of the spiral springs.
- 2) Pull the support down over the foot.

### 5 Cleaning

#### NOTICE

#### Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

Clean the support regularly:

- 1) **Recommendation:** Use a laundry bag or net.
- 2) Wash the support in warm water at 30 °C with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 3) Allow to air dry. Do not expose to direct heat (e.g. sunshine, stove or radiator).

### 6 Disposal

Dispose of the product in accordance with national regulations.

### 7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

## 7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

## 7.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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GlaxoSmithKline Consumer Healthcare GmbH & Co. KG