DE

EN

NL

FR

IT

ES

PL

RU

JP

NO

SWE

FI

CN

AR

DK

TR

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230V ≈ AC : 50 Hz

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*Start to Smile!*

## ONLYSMILE TEETH WHITENING SYSTEM FOR HOME

- **Visible result after the first application**
- **Up to 9 shades lighter**
- **Multiple use**
- **Can be repeated at any time**

### **Application**

Charge the OnlySmile Lamp using the included charger.

Attach the silicone mouth tray to the OnlySmile Lamp.

Spread 4 ml of the OnlySmile Teeth Whitening Gel on the bottom and top side of the silicone mouth tray. Insert the mouth tray attached to the OnlySmile Lamp. Press the button to activate the OnlySmile Lamp. After 30 minutes the OnlySmile Lamp will turn off automatically.

Detach the silicone mouth tray from the OnlySmile Lamp and wash the silicone mouth tray under warm water. You can repeat the treatment to reach your best possible result.

**Tip: Use twice a day for 5 days (2 x 30 min) to achieve optimal results.**

### **Amount of applications**

The 40 ml tube OnlySmile Teeth Whitening Gel lasts for 8–10 treatments.

### **Ingredients of OnlySmile Teeth Whitening Gel**

Glycerin, Aqua, Propylene Glycol, Carbomer, Urea Peroxide, Sodium Saccharin, Aroma, Limonene, Linalool, Triethanolamine, Mica, Ethylhexylglycerin, Phenoxyethanol, CI 77891

The OnlySmile Teeth Whitening Gel is developed and produced in Germany and contains less than 0.1 % hydrogen-peroxide according to the European Directive 2011/84/EU.

### **Warning OnlySmile Teeth Whitening Gel**

If you experience any unpleasant irritations, stop the treatment and ask your dentist. Avoid eye contact. Not suitable for children under the age of 6. Not suitable for pregnant or breast-feeding women.

### **Warning OnlySmile Lamp**

Keep away from water. Do not look directly into the lamp. Keep out of reach of children.

### **Technical specifications:**

LED-Lamp Model: OnlySmile Lamp

Wavelength: 460 nm – 465nm

### **NiMH-Battery**

Charging Time: 1h

AC Adapter: 5V, 1A

230V ≈ AC : 50 Hz

Warranty: 2 years from date of purchase.

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Only Kosmetik GmbH