SOVA[®]
night guard

Quick-fit Guide

Watch our fitting video online:
sovanightguard.com/fitting

Braces:

If you have braces, **DO NOT** attempt to fit your SOVA night guard at home. Instead, contact your orthodontist (see right for more info).

Fitting Checklist:

- Bowl, ceramic or glass
- Hot, but not boiling, water (140° F)
- Fork or similar utensil
- Mirror

✓ Do This:

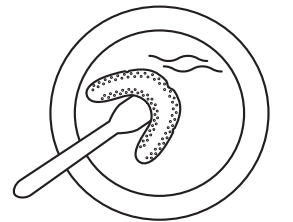
- Use water that is hot to the touch to activate your night guard.
- Double check that the bottom of your front teeth line up with the bite pad.
- Suck in air gently, creating a snug but still comfortable fit.

❌ Don't Do This:

- Use boiling water to mold your night guard.
- Overly stretch the material, creating too large perforations.
- Suck in air too vigorously, creating a fit that's too tight.

Step 1

- Wash your hands and the night guard with soap and water.
- Heat water in a small bowl until hot to the touch (140°F / 60°C).
- Place your SOVA into the hot water until soft and pliable (use utensil to determine softness).


Step 2

- Once soft, use the utensil to carefully lift the night guard out of the water. **DO NOT let the softened night guard fold on itself – the material will glue together, making it difficult to reshape.**

Step 3

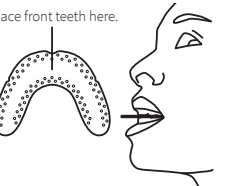- Balancing the pliable guard with your thumbs, lean in toward the mirror and open your mouth wide.
- Insert the SOVA and bite down gently on the bite pad in the middle of the guard.

Step 4

- Hold the night guard in place using your teeth and tongue.
- With your fingers, fold the guard straight up. Be sure to mold the guard on the front and sides of your teeth to create a solid shield.

Step 5

- Close your lips over the guard and gently create suction until the material has hardened to a snug, perfect fit.
- Allow the guard to remain in the mouth for approximately 3 minutes.

Congrats!

You've successfully fitted the world's most advanced night guard.

Fitting Test

1. Does your guard cover the outside of your teeth?
2. Is your night guard snug enough that you need your hands to remove it?
3. Can you clearly say "SOVA"? If "No" to any of the above, start back at Step 1 and remold. SOVA guards are remoldable up to 20 times!

Got Braces?

If undergoing orthodontic treatment, have your dentist or orthodontist fit your night guard for you! He/she will use the appropriate barriers between the guard and braces so that the material, while pliable, doesn't bond to your brackets.

Visit sovanightguard.com/fitting for more information.

Cleaning & Storage

Your SOVA guard can be brushed with a regular toothbrush and toothpaste. Carefully rinse and dry before storage. Store at room temperature under dry conditions.

Protect your SOVA night guard from warm environments, such as hot dashboards in cars. When hot environments are unavoidable, we suggest keeping your night guard in a container with water.

Diffusix™ Technology

SOVA night guard is made with a non-compressible, perforated material that diffuses and absorbs grinding forces.

For more information on the science behind your night guard, visit sovanightguard.com.

120 days durability warranty.

WARNINGS:

- If you are wearing braces, you must have the SOVA night guard fitted by your orthodontist or dentist. DO NOT TRY TO FIT IT ON YOUR OWN.
- Do not use with artificial dentures. Improper use may cause health risks or decrease the effectiveness of the protective properties of the night guard.
- Do not use an improperly fitted mouth guard.
- Do not overstretch the material when it is soft. This may thin out the material and decrease the strength of the night guard.
- Do not expose the night guard to temperature exceeding 160 F (70 C) or to open flames.
- Keep protected from warm environments, such as hot dashboards in cars.
- Be careful if you use nail polish or artificial nails. The material can stick to these surfaces.
- Avoid situations where inhalation or swallowing of the night guard could occur.
- If you experience pain while using the SOVA night guard, please see your dentist before continued use.
- Do not use if you have loose or wiggly teeth, loose fillings, loose caps or cavities with no fillings.
- Do not use if you have sores in your mouth or bleeding gums.
- Do not use the SOVA night guard as an athletic mouthguard. Instead please go to www.sisuguard.com to explore SISU® Mouthguards, our athletic mouthguard line.

DISCLAIMER:

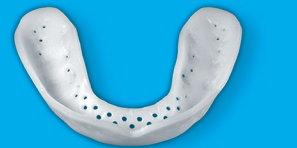
This warranty is made in lieu of all other warranties, express or implied, including but not limited to any implied warranty or merchantability or fitness for a particular purpose and any implied warranty arising out of a course of a dealing, custom, usage or trade. Except as is otherwise expressly provided herein, ATI disclaims all liability for loss, injury or damage based upon negligent design or defects in materials and workmanship for the within mouth guard. Keep this dental warranty, disclaimer and store receipt in your records. In case of injury, mail mouth guard, store receipt, the notice of injury and accompanying statements to:

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