OMRON®

GETTING STARTED

5 Series Arm Blood Pressure Monitor

BP742

The American Heart Association's guideline for upper limit of normal home blood pressure is:

135 mmHg systolic **85** mmHg diastolic

Please read the instruction manual for complete product information.


Have questions?

DO NOT RETURN THIS PRODUCT TO THE STORE.

Call our Toll-Free Consumer Help Line at **800-634-4350** for assistance with your Omron product.

www.omronhealthcare.com

STEP 1 Insert four AA batteries into bottom of the unit.
Plug arm cuff into the main unit.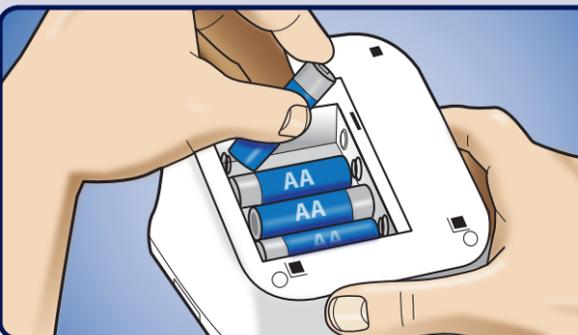

STEP 2 Date and Time Setting Option: (A or B)

A. To Set Date and Time:

While the unit is off, press the **⏻** button twice to show the year.* The year will flash. Use the **◀** and **▶** buttons to advance to the current year, then press the **⏻** button to confirm. The screen will then show the month and date. Change month, date and time the same way. Press **START/STOP** when complete to shut off unit.

*Pressing the **⏻** once gives you the option to turn on and off sound alerts on your monitor.


B. To Bypass Setting Date and Time:

Simply proceed to **STEP 3** to continue. Your readings will be recorded in the memory of the unit, but will not have the correct Date and Time connected to them for reference.

For more information on date and time setting, see instruction manual.

STEP 3 Place cuff directly on bare skin of upper left arm, with cuff edge 1/2 inch above elbow. Align cuff so the arrow **▶** is centered on the inside of your arm with the tubing pointing down toward your hand. Close cuff snugly.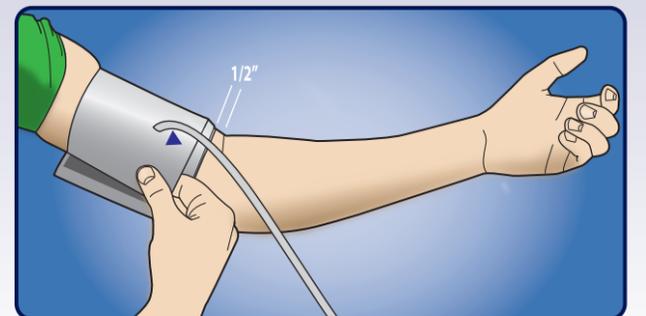

To use on right arm, see instruction manual.

STEP 4 To turn unit on and take a measurement, press **START/STOP** button once. The cuff will inflate automatically. Hold still until cuff deflates and results are displayed.

FREQUENTLY ASKED QUESTIONS

- Q.** Why are my blood pressure readings sometimes different?
- A.** Your blood pressure changes constantly based on your daily activities and time of day. Because of this, some fluctuations can be expected. That's why it is important to use averages over time to compare readings. Because each person is different, it's important to consult with your doctor about what your numbers mean to you.
- Q.** Why could my doctor's readings be different than my readings taken at home?
- A.** Having your blood pressure measured by a healthcare professional in a doctor's office can cause nervousness and may result in an elevated readings. Because these readings can vary considerably, you should take your blood pressure on a regular basis at home and share your results with your healthcare provider.

OTHER IMPORTANT TIPS:

- For accurate readings, sit quietly with feet flat on the floor and rest for five minutes before taking measurements. Do not measure within 30 minutes after eating, exercising, bathing, smoking or drinking alcohol.
- This monitor allows two users to take and track their readings separately in memory. To review readings for multiple users, Press USER A/B button to switch between users. Press the MEMORY buttons to scroll through readings.
- To use the Advanced Averaging function and display averages, see the instruction manual.

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