02274 English RAPID SARS-COV-2 ANTIGEN TEST CARD INSTRUCTION GUIDE FOR ANTERIOR NASAL SWAB SPECIMENS For self-testing

REF	1N40C5-2	For 1 Test/Box
REF	1N40C5-4	For 5 Tests/Box
REF	1N40C5-6	For 20 Tests/Box

Please follow the instruction leaflet carefully. INTENDED USE: Rapid SARS-CoV-2 Antigen Test Card is an immunochromatography based step in vitro test. It is designed for the rapid qualitative determination of SARS-CoV-2 viral antigen in anterior nasal swabs from individuals with or without symptoms or other epidemiological reasons to suspect COVID-19. Rapid SARS-CoV-2 Antigen Test Card detects the SARS-CoV-2 nucleocapsid protein (N protein). Rapid SARS-CoV-2 Antigen Test Card shall not be used as sole basis to diagnose or exclude SARS-CoV-2 infection. Children under 14 years of age should be assisted by an adult.

Components	For 1 Test/Box	For 5 Tests/Box	For 20 Tests/Box
Rapid SARS-CoV-2 Antigen Test Card (sealed foil pouch)	1	5	20
Stirrer swab	1	5	20
Extraction tube	1	5	20
Sample extraction buffer	1	5	20
Instructions for use (this leaflet)	1	1	1
Tube stand	1 (packaging)	1	1

PERFORMANCE (SENSITIVITY AND SPECIFICITY) For Clinical Study 1: Rapid SARS-CoV-2 Antigen Test Card was compared to the confirmed clinical diagnosis. The study involved 230 nasal samples. The test results are summarized below:

Evaluated Reagent Results	RT-PCR Results		Total
	Positive (+)	Negative (-)	
Positive (+)	101	1	102
Negative (-)	4	124	128
Total	105	125	230

Sensitivity (PPA) = 101/105\*100% = 96.19% (95% CI: 92.53% - 99.85%) Specificity (NPA) = 124/125\*100% = 99.20% (95% CI: 97.64% - 99.99%) Accuracy (OPA) = 225/230\*100% = 97.83% (95% CI: 95.94% - 99.71%)

Evaluated Reagent Results	RT-PCR Results		Total
	Positive (+)	Negative (-)	
Positive (+)	324	0	324
Negative (-)	6	503	509
Total	330	503	833

Sensitivity (PPA) = 324/330\*100% = 98.18% (95% CI: 96.74% - 99.62%) Specificity (NPA) = 503/509\*100% = 98.82% (95% CI: 98.00% - 100.00%) Accuracy (OPA) = 827/833\*100% = 99.28% (95% CI: 98.71% - 99.85%)

None of the following substances at the tested concentration showed any interference with the test.

- Whole Blood: 1%
- Phenylephrine: 15%
- Menthol: 0.15%
- Fluclisone Propionate: 5%
- Oxsalimur Phosphate: 0.5%
- Biotin: 1200 ng/ml
- Alkaloid: 10%
- Tobramycin: 0.0004%
- Cromolyn: 0.15%
- Mupirocin: 0.25%
- Sodium Chloride: 5%
- Mucin: 2%
- Oxymetazoline: 15%
- Benzocaine: 0.15%
- Zincam Nasal Spray: 5%
- Human Anti-mouse Antibody (HAMA): 60 ng/ml

CROSS REACTIVITY: Cross-reactivity of the Test Device was evaluated by testing viruses and other microorganisms. The final test concentrations of viruses and other microorganisms are documented in the Cross-Reactivity Study. The following viruses and other microorganisms exceed the Human SARS-coronavirus HCoV-NL63, MERS coronavirus, Parainfluenza Virus 1-4, Enterovirus EV71, Respiratory syncytial virus, Rhinovirus, Influenza A Virus (H1N1 and H3N2), Influenza B Virus (Yamagata and Victoria), Adeno virus 71, Human Metapneumovirus (hMPV), Staphylococcus epidermidis, Chlamydia pneumoniae, Streptococcus pneumoniae, Bordetella pertussis, Mycobacterium tuberculosis, Legionella pneumophila, Mycoplasma pneumoniae, Haemophilus influenzae, Candida albicans, Staphylococcus aureus, Pseudomonas aeruginosa, Escherichia coli, Pneumocystis jirovecii (PJP) and pooled human nasal wash.

IMPORTANT INFORMATION BEFORE THE EXECUTION: 1. Read this instruction guide carefully. 2. Do not use the product beyond the expiration date. 3. Do not use the product if the pouch is damaged or the seal is broken. 4. Store the test device at 4 to 30 °C in the original sealed pouch. Do Not Freeze. 5. The product should be used at room temperature (15°C to 30°C). If the product has been stored in a cool area (less than 15°C), leave it at normal room temperature for 30 minutes before using. 6. Handle all specimens as potentially infectious. 7. Inadequate or inappropriate specimen collection, storage, and transport may yield inaccurate test results. 8. Use the swabs included in the test kit to ensure optimal performance of the test. 9. Correct specimen collection is the most important step in the procedure. Make sure to collect enough specimen material (nasal secretion) with the swab, especially for anterior nasal sampling. 10. Blow the nose several times before collecting specimen. 11. The specimens should be tested as soon as possible after collection. 12. Apply the drops of test specimen only to the specimen well (S).

LIMITATIONS: 1. The test kit is to be used exclusively for the qualitative detection of SARS-CoV-2 viral antigen in anterior nasal swab specimens. The exact concentration of SARS-CoV-2 viral antigen cannot be determined as part of this test. 2. Proper specimen collection is critical. Failure to follow the procedure may result in inaccurate test results. Improper collection, storage, or even freezing and thawing of the specimen can lead to inaccurate test results. 3. If the viral load of the specimen is below the detection limit of the test, the test may produce a negative result. 4. As with all diagnostic tests, a definitive clinical diagnosis should not be based on the result of a single test, but should be made by the physician after evaluation of all clinical and laboratory results. 5. A negative result does not exclude viral infection except for SARS-CoV-2 and should be confirmed by molecular diagnostic methods if COVID-19 is suspected. 6. A positive result does not exclude coinfection with other pathogens. 7. The SARS-CoV-2 rapid antigen test can detect both viable and non-viable SARS-CoV-2 material. The performance of the SARS-CoV-2 rapid test is dependent on viral load and may not correlate with other diagnostic methods. 8. Users should test specimens as soon as possible after specimen collection and within two hours of specimen collection. 9. Sensitivity for nasal or oropharyngeal swabs may be lower than nasopharyngeal swabs. It is recommended to use the nasopharyngeal swab specimens by healthcare professionals. 10. Monoclonal antibodies may fail to detect, or detect with less sensitivity, SARS-CoV-2 viruses that have undergone minor amino acid changes in the target epitope region. 11. The amount of antigen in a sample may decrease as the duration of illness increases. Specimens collected after day 5-7 of illness are more likely to be tested negative compared to a RT-PCR assay. 12. The kit was validated with the assorted swabs. Use of alternative swabs may result in false negative results. 13. The validity of Rapid SARS-CoV-2 Antigen Test Card has not been proven for identification/confirmation of tissue culture isolates and should not be used in this capacity. 14. Positive test results do not rule out co-infections with other pathogens. Positive results may occur in cases of infection with SARS-CoV.

PREPARATION: 1. Clean, clear and dry a flat surface. 2. Check the test kit contents. Make sure that nothing is damaged or broken. 3. Timer at hand. 4. Blow your nose several times before collecting specimen. 5. Wash hands.

DISPOSAL: The test kit may be disposed of with normal household waste in accordance with the applicable local regulations.

PROCEDURE: This test is applicable for people of all ages. The recommended operators are aging from 14 to 90. Children under 14 years of age should be tested by an adult. Do not continue the use of the kit if they feel any pain.

1. Rotate the lid of sample extraction buffer bottle. Caution: Open it away from your face and be careful not to spill any of the liquid.

2. Squeeze all extraction buffer out of the bottle into the extraction tube. Caution: Avoid touching the bottle against the tube.

3. Find the swab in the sealed wrapper in front of you. Identify the soft, fabric tip of the swab.

4. Peel open the swab packaging and gently take out the swab. Caution: Never touch the soft, fabric tip of the swab with your hands.

5. Carefully insert swab into one nostril. The swab tip should be inserted no less than 2.5 cm (1 inch) from the edge of the nostril. Roll swab 3-4 times along the mucosa inside the nostril. Leave swab in the nostril for several seconds. Using the same swab, repeat this process for the other nostril. Caution: This may feel uncomfortable. Do not insert the swab any deeper if you feel strong resistance or pain.

6. Place swab into extraction tube. Roll swab three to five (3-5) times. Leave swab in extraction buffer for 1 minute.

7. Pinch extraction tube with fingers and remove the solution from swab as much as possible.

8. Install the nozzle cap onto the sample extraction tube tightly.

9. Bring the kit components to room temperature before testing. Open the pouch and remove the card. Place the card on a flat and level surface. Caution: Once opened, the test card must be used immediately.

10. Invert the extraction tube and add 3 drops (about 75 µL) of test specimen into the specimen well (S) by gently squeezing the extraction tube. Caution: The formation of air bubbles in the specimen well (S) must be avoided.


11. Read the results after 15-20 minutes. Caution: Results after 20 minutes may not be accurate. The used device may be disposed of with normal household waste in accordance with the applicable local regulations.

INTERPRETATION OF RESULTS: Positive: If two colored bands appear with one colored band in the Control Zone (C) and another in the Test Zone (T) within 15-20 minutes, the test result is positive. Negative: If one colored band appears in the Control Zone (C) and no colored band appears in the Test Zone (T) within 15-20 minutes, the test result is negative. Invalid: If no colored line appears in the control area (C) within 15-20 minutes, the test is invalid. Repeat the test with a new test card.

QUALITY CONTROL: This is an integrated reagent and is used to control the procedure. The control line appears when the test has been performed correctly and the reagents are reactive.

FREQUENTLY ASKED QUESTIONS (FAQ): 1. How does the detection work? The N protein of the SARS-CoV-2 virus reacts with the stripe-like coating of the test line and, if present, results in a color change, i.e. a red line appears. Therefore, if the sample does not contain any viral proteins or antigens, there will be no red test line (T).

2. What can affect my test result? 3. How can I ensure that my test result is accurate? 4. How can I ensure that my test result is accurate? 5. How can I ensure that my test result is accurate? 6. How can I ensure that my test result is accurate? 7. How can I ensure that my test result is accurate? 8. How can I ensure that my test result is accurate? 9. How can I ensure that my test result is accurate? 10. How can I ensure that my test result is accurate? 11. How can I ensure that my test result is accurate? 12. How can I ensure that my test result is accurate? 13. How can I ensure that my test result is accurate? 14. How can I ensure that my test result is accurate? 15. How can I ensure that my test result is accurate? 16. How can I ensure that my test result is accurate? 17. How can I ensure that my test result is accurate? 18. How can I ensure that my test result is accurate? 19. How can I ensure that my test result is accurate? 20. How can I ensure that my test result is accurate? 21. How can I ensure that my test result is accurate? 22. How can I ensure that my test result is accurate? 23. How can I ensure that my test result is accurate? 24. How can I ensure that my test result is accurate? 25. How can I ensure that my test result is accurate? 26. How can I ensure that my test result is accurate? 27. How can I ensure that my test result is accurate? 28. How can I ensure that my test result is accurate? 29. How can I ensure that my test result is accurate? 30. How can I ensure that my test result is accurate? 31. How can I ensure that my test result is accurate? 32. How can I ensure that my test result is accurate? 33. How can I ensure that my test result is accurate? 34. How can I ensure that my test result is accurate? 35. How can I ensure that my test result is accurate? 36. How can I ensure that my test result is accurate? 37. How can I ensure that my test result is accurate? 38. How can I ensure that my test result is accurate? 39. How can I ensure that my test result is accurate? 40. How can I ensure that my test result is accurate? 41. How can I ensure that my test result is accurate? 42. How can I ensure that my test result is accurate? 43. How can I ensure that my test result is accurate? 44. How can I ensure that my test result is accurate? 45. How can I ensure that my test result is accurate? 46. How can I ensure that my test result is accurate? 47. How can I ensure that my test result is accurate? 48. How can I ensure that my test result is accurate? 49. How can I ensure that my test result is accurate? 50. How can I ensure that my test result is accurate? 51. How can I ensure that my test result is accurate? 52. How can I ensure that my test result is accurate? 53. How can I ensure that my test result is accurate? 54. How can I ensure that my test result is accurate? 55. How can I ensure that my test result is accurate? 56. How can I ensure that my test result is accurate? 57. How can I ensure that my test result is accurate? 58. How can I ensure that my test result is accurate? 59. How can I ensure that my test result is accurate? 60. How can I ensure that my test result is accurate? 61. How can I ensure that my test result is accurate? 62. How can I ensure that my test result is accurate? 63. How can I ensure that my test result is accurate? 64. How can I ensure that my test result is accurate? 65. How can I ensure that my test result is accurate? 66. How can I ensure that my test result is accurate? 67. How can I ensure that my test result is accurate? 68. How can I ensure that my test result is accurate? 69. How can I ensure that my test result is accurate? 70. How can I ensure that my test result is accurate? 71. How can I ensure that my test result is accurate? 72. How can I ensure that my test result is accurate? 73. How can I ensure that my test result is accurate? 74. How can I ensure that my test result is accurate? 75. How can I ensure that my test result is accurate? 76. How can I ensure that my test result is accurate? 77. How can I ensure that my test result is accurate? 78. How can I ensure that my test result is accurate? 79. How can I ensure that my test result is accurate? 80. How can I ensure that my test result is accurate? 81. How can I ensure that my test result is accurate? 82. How can I ensure that my test result is accurate? 83. How can I ensure that my test result is accurate? 84. How can I ensure that my test result is accurate? 85. How can I ensure that my test result is accurate? 86. How can I ensure that my test result is accurate? 87. How can I ensure that my test result is accurate? 88. How can I ensure that my test result is accurate? 89. How can I ensure that my test result is accurate? 90. How can I ensure that my test result is accurate? 91. How can I ensure that my test result is accurate? 92. How can I ensure that my test result is accurate? 93. How can I ensure that my test result is accurate? 94. How can I ensure that my test result is accurate? 95. How can I ensure that my test result is accurate? 96. How can I ensure that my test result is accurate? 97. How can I ensure that my test result is accurate? 98. How can I ensure that my test result is accurate? 99. How can I ensure that my test result is accurate? 100. How can I ensure that my test result is accurate? 101. How can I ensure that my test result is accurate? 102. How can I ensure that my test result is accurate? 103. How can I ensure that my test result is accurate? 104. How can I ensure that my test result is accurate? 105. How can I ensure that my test result is accurate? 106. How can I ensure that my test result is accurate? 107. How can I ensure that my test result is accurate? 108. How can I ensure that my test result is accurate? 109. How can I ensure that my test result is accurate? 110. How can I ensure that my test result is accurate? 111. How can I ensure that my test result is accurate? 112. How can I ensure that my test result is accurate? 113. How can I ensure that my test result is accurate? 114. How can I ensure that my test result is accurate? 115. How can I ensure that my test result is accurate? 116. How can I ensure that my test result is accurate? 117. How can I ensure that my test result is accurate? 118. How can I ensure that my test result is accurate? 119. How can I ensure that my test result is accurate? 120. How can I ensure that my test result is accurate? 121. How can I ensure that my test result is accurate? 122. How can I ensure that my test result is accurate? 123. How can I ensure that my test result is accurate? 124. How can I ensure that my test result is accurate? 125. How can I ensure that my test result is accurate? 126. How can I ensure that my test result is accurate? 127. How can I ensure that my test result is accurate? 128. How can I ensure that my test result is accurate? 129. How can I ensure that my test result is accurate? 130. How can I ensure that my test result is accurate? 131. How can I ensure that my test result is accurate? 132. How can I ensure that my test result is accurate? 133. How can I ensure that my test result is accurate? 134. How can I ensure that my test result is accurate? 135. How can I ensure that my test result is accurate? 136. How can I ensure that my test result is accurate? 137. How can I ensure that my test result is accurate? 138. How can I ensure that my test result is accurate? 139. How can I ensure that my test result is accurate? 140. How can I ensure that my test result is accurate? 141. How can I ensure that my test result is accurate? 142. How can I ensure that my test result is accurate? 143. How can I ensure that my test result is accurate? 144. How can I ensure that my test result is accurate? 145. How can I ensure that my test result is accurate? 146. How can I ensure that my test result is accurate? 147. How can I ensure that my test result is accurate? 148. How can I ensure that my test result is accurate? 149. How can I ensure that my test result is accurate? 150. How can I ensure that my test result is accurate? 151. How can I ensure that my test result is accurate? 152. How can I ensure that my test result is accurate? 153. How can I ensure that my test result is accurate? 154. How can I ensure that my test result is accurate? 155. How can I ensure that my test result is accurate? 156. How can I ensure that my test result is accurate? 157. How can I ensure that my test result is accurate? 158. How can I ensure that my test result is accurate? 159. How can I ensure that my test result is accurate? 160. How can I ensure that my test result is accurate? 161. How can I ensure that my test result is accurate? 162. How can I ensure that my test result is accurate? 163. How can I ensure that my test result is accurate? 164. How can I ensure that my test result is accurate? 165. How can I ensure that my test result is accurate? 166. How can I ensure that my test result is accurate? 167. How can I ensure that my test result is accurate? 168. How can I ensure that my test result is accurate? 169. How can I ensure that my test result is accurate? 170. How can I ensure that my test result is accurate? 171. How can I ensure that my test result is accurate? 172. How can I ensure that my test result is accurate? 173. How can I ensure that my test result is accurate? 174. How can I ensure that my test result is accurate? 175. How can I ensure that my test result is accurate? 176. How can I ensure that my test result is accurate? 177. How can I ensure that my test result is accurate? 178. How can I ensure that my test result is accurate? 179. How can I ensure that my test result is accurate? 180. How can I ensure that my test result is accurate? 181. How can I ensure that my test result is accurate? 182. How can I ensure that my test result is accurate? 183. How can I ensure that my test result is accurate? 184. How can I ensure that my test result is accurate? 185. How can I ensure that my test result is accurate? 186. How can I ensure that my test result is accurate? 187. How can I ensure that my test result is accurate? 188. How can I ensure that my test result is accurate? 189. How can I ensure that my test result is accurate? 190. How can I ensure that my test result is accurate? 191. How can I ensure that my test result is accurate? 192. How can I ensure that my test result is accurate? 193. How can I ensure that my test result is accurate? 194. How can I ensure that my test result is accurate? 195. How can I ensure that my test result is accurate? 196. How can I ensure that my test result is accurate? 197. How can I ensure that my test result is accurate? 198. How can I ensure that my test result is accurate? 199. How can I ensure that my test result is accurate? 200. How can I ensure that my test result is accurate? 201. How can I ensure that my test result is accurate? 202. How can I ensure that my test result is accurate? 203. How can I ensure that my test result is accurate? 204. How can I ensure that my test result is accurate? 205. How can I ensure that my test result is accurate? 206. How can I ensure that my test result is accurate? 207. How can I ensure that my test result is accurate? 208. How can I ensure that my test result is accurate? 209. How can I ensure that my test result is accurate? 210. How can I ensure that my test result is accurate? 211. How can I ensure that my test result is accurate? 212. How can I ensure that my test result is accurate? 213. How can I ensure that my test result is accurate? 214. How can I ensure that my test result is accurate? 215. How can I ensure that my test result is accurate? 216. How can I ensure that my test result is accurate? 217. How can I ensure that my test result is accurate? 218. How can I ensure that my test result is accurate? 219. How can I ensure that my test result is accurate? 220. How can I ensure that my test result is accurate? 221. How can I ensure that my test result is accurate? 222. How can I ensure that my test result is accurate? 223. How can I ensure that my test result is accurate? 224. How can I ensure that my test result is accurate? 225. How can I ensure that my test result is accurate? 226. How can I ensure that my test result is accurate? 227. How can I ensure that my test result is accurate? 228. How can I ensure that my test result is accurate? 229. How can I ensure that my test result is accurate? 230. How can I ensure that my test result is accurate? 231. How can I ensure that my test result is accurate? 232. How can I ensure that my test result is accurate? 233. How can I ensure that my test result is accurate? 234. How can I ensure that my test result is accurate? 235. How can I ensure that my test result is accurate? 236. How can I ensure that my test result is accurate? 237. How can I ensure that my test result is accurate? 238. How can I ensure that my test result is accurate? 239. How can I ensure that my test result is accurate? 240. How can I ensure that my test result is accurate? 241. How can I ensure that my test result is accurate? 242. How can I ensure that my test result is accurate? 243. How can I ensure that my test result is accurate? 244. How can I ensure that my test result is accurate? 245. How can I ensure that my test result is accurate? 246. How can I ensure that my test result is accurate? 247. How can I ensure that my test result is accurate? 248. How can I ensure that my test result is accurate? 249. How can I ensure that my test result is accurate? 250. How can I ensure that my test result is accurate? 251. How can I ensure that my test result is accurate? 252. How can I ensure that my test result is accurate? 253. How can I ensure that my test result is accurate? 254. How can I ensure that my test result is accurate? 255. How can I ensure that my test result is accurate? 256. How can I ensure that my test result is accurate? 257. How can I ensure that my test result is accurate? 258. How can I ensure that my test result is accurate? 259. How can I ensure that my test result is accurate? 260. How can I ensure that my test result is accurate? 261. How can I ensure that my test result is accurate? 262. How can I ensure that my test result is accurate? 263. How can I ensure that my test result is accurate? 264. How can I ensure that my test result is accurate? 265. How can I ensure that my test result is accurate? 266. How can I ensure that my test result is accurate? 267. How can I ensure that my test result is accurate? 268. How can I ensure that my test result is accurate? 269. How can I ensure that my test result is accurate? 270. How can I ensure that my test result is accurate? 271. How can I ensure that my test result is accurate? 272. How can I ensure that my test result is accurate? 273. How can I ensure that my test result is accurate? 274. How can I ensure that my test result is accurate? 275. How can I ensure that my test result is accurate? 276. How can I ensure that my test result is accurate? 277. How can I ensure that my test result is accurate? 278. How can I ensure that my test result is accurate? 279. How can I ensure that my test result is accurate? 280. How can I ensure that my test result is accurate? 281. How can I ensure that my test result is accurate? 282. How can I ensure that my test result is accurate? 283. How can I ensure that my test result is accurate? 284. How can I ensure that my test result is accurate? 285. How can I ensure that my test result is accurate? 286. How can I ensure that my test result is accurate? 287. How can I ensure that my test result is accurate? 288. How can I ensure that my test result is accurate? 289. How can I ensure that my test result is accurate? 290. How can I ensure that my test result is accurate? 291. How can I ensure that my test result is accurate? 292. How can I ensure that my test result is accurate? 293. How can I ensure that my test result is accurate? 294. How can I ensure that my test result is accurate? 295. How can I ensure that my test result is accurate? 296. How can I ensure that my test result is accurate? 297. How can I ensure that my test result is accurate? 298. How can I ensure that my test result is accurate? 299. How can I ensure that my test result is accurate? 300. How can I ensure that my test result is accurate? 301. How can I ensure that my test result is accurate? 302. How can I ensure that my test result is accurate? 303. How can I ensure that my test result is accurate? 304. How can I ensure that my test result is accurate? 305. How can I ensure that my test result is accurate? 306. How can I ensure that my test result is accurate? 307. How can I ensure that my test result is accurate? 308. How can I ensure that my test result is accurate? 309. How can I ensure that my test result is accurate? 310. How can I ensure that my test result is accurate? 311. How can I ensure that my test result is accurate? 312. How can I ensure that my test result is accurate? 313. How can I ensure that my test result is accurate? 314. How can I ensure that my test result is accurate? 315. How can I ensure that my test result is accurate? 316. How can I ensure that my test result is accurate? 317. How can I ensure that my test result is accurate? 318. How can I ensure that my test result is accurate? 319. How can I ensure that my test result is accurate? 320. How can I ensure that my test result is accurate? 321. How can I ensure that my test result is accurate? 322. How can I ensure that my test result is accurate? 323. How can I ensure that my test result is accurate? 324. How can I ensure that my test result is accurate? 325. How can I ensure that my test result is accurate? 326. How can I ensure that my test result is accurate? 327. How can I ensure that my test result is accurate? 328. How can I ensure that my test result is accurate? 329. How can I ensure that my test result is accurate? 330. How can I ensure that my test result is accurate? 331. How can I ensure that my test result is accurate? 332. How can I ensure that my test result is accurate? 333. How can I ensure that my test result is accurate? 334. How can I ensure that my test result is accurate? 335. How can I ensure that my test result is accurate? 336. How can I ensure that my test result is accurate? 337. How can I ensure that my test result is accurate? 338. How can I ensure that my test result is accurate? 339. How can I ensure that my test result is accurate? 340. How can I ensure that my test result is accurate? 341. How can I ensure that my test result is accurate? 342. How can I ensure that my test result is accurate? 343. How can I ensure that my test result is accurate? 344. How can I ensure that my test result is accurate? 345. How can I ensure that my test result is accurate? 346. How can I ensure that my test result is accurate? 347. How can I ensure that my test result is accurate? 348. How can I ensure that my test result is accurate? 349. How can I ensure that my test result is accurate? 350. How can I ensure that my test result is accurate? 351. How can I ensure that my test result is accurate? 352. How can I ensure that my test result is accurate? 353. How can I ensure that my test result is accurate? 354. How can I ensure that my test result is accurate? 355. How can I ensure that my test result is accurate? 356. How can I ensure that my test result is accurate? 357. How can I ensure that my test result is accurate? 358. How can I ensure that my test result is accurate? 359. How can I ensure that my test result is accurate? 360. How can I ensure that my test result is accurate? 361. How can I ensure that my test result is accurate? 362. How can I ensure that my test result is accurate? 363. How can I ensure that my test result is accurate? 364. How can I ensure that my test result is accurate? 365. How can I ensure that my test result is accurate? 366. How can I ensure that my test result is accurate? 367. How can I ensure that my test result is accurate? 368. How can I ensure that my test result is accurate? 369. How can I ensure that my test result is accurate? 370. How can I ensure that my test result is accurate? 371. How can I ensure that my test result is accurate? 372. How can I ensure that my test result is accurate? 373. How can I ensure that my test result is accurate? 374. How can I ensure that my test result is accurate? 375. How can I ensure that my test result is accurate? 376. How can I ensure that my test result is accurate? 377. How can I ensure that my test result is accurate? 378. How can I ensure that my test result is accurate? 379. How can I ensure that my test result is accurate? 380. How can I ensure that my test result is accurate? 381. How can I ensure that my test result is accurate? 382. How can I ensure that my test result is accurate? 383. How can I ensure that my test result is accurate? 384. How can I ensure that my test result is accurate? 385. How can I ensure that my test result is accurate? 386. How can I ensure that my test result is accurate? 387. How can I ensure that my test result is accurate? 388. How can I ensure that my test result is accurate? 389. How can I ensure that my test result is accurate? 390. How can I ensure that my test result is accurate? 391. How can I ensure that my test result is accurate? 392. How can I ensure that my test result is accurate? 393. How can I ensure that my test result is accurate? 394. How can I ensure that my test result is accurate? 395. How can I ensure that my test result is accurate? 396. How can I ensure that my test result is accurate? 397. How can I ensure that my test result is accurate? 398. How can I ensure that my test result is accurate? 399. How can I ensure that my test result is accurate? 400. How can I ensure that my test result is accurate? 401. How can I ensure that my test result is accurate? 402. How can I ensure that my test result is accurate? 403. How can I ensure that my test result is accurate? 404. How can I ensure that my test result is accurate? 405. How can I ensure that my test result is accurate? 406. How can I ensure that my test result is accurate? 407. How can I ensure that my test result is accurate? 408. How can I ensure that my test result is accurate? 409. How can I ensure that my test result is accurate? 410. How can I ensure that my test result is accurate? 411. How can I ensure that my test result is accurate? 412. How can I ensure that my test result is accurate? 413. How can I ensure that my test result is accurate? 414. How can I ensure that my test result is accurate? 415. How can I ensure that my test result is accurate? 416. How can I ensure that my test result is accurate? 417. How can I ensure that my test result is accurate? 418. How can I ensure that my test result is accurate? 419. How can I ensure that my test result is accurate? 420. How can I ensure that my test result is accurate? 421. How can I ensure that my test result is accurate? 422. How can I ensure that my test result is accurate? 423. How can I ensure that my test result is accurate? 424. How can I ensure that my test result is accurate? 425. How can I ensure that my test result is accurate? 426. How can I ensure that my test result is accurate? 427. How can I ensure that my test result is accurate? 428. How can I ensure that my test result is accurate? 429. How can I ensure that my test result is accurate? 430. How can I ensure that my test result is accurate? 431. How can I ensure that my test result is accurate? 432. How can I ensure that my test result is accurate? 433. How can I ensure that my test result is accurate? 434. How can I ensure that my test result is accurate? 435. How can I ensure that my test result is accurate? 436. How can I ensure that my test result is accurate? 437. How can I ensure that my test result is accurate? 438. How can I ensure that my test result is accurate? 439. How can I ensure that my test result is accurate? 440. How can I ensure that my test result is accurate? 441. How can I ensure that my test result is accurate? 442. How can I ensure that my test result is accurate? 443. How can I ensure that my test result is accurate? 444. How can I ensure that my test result is accurate? 445. How can I ensure that my test result is accurate? 446. How can I ensure that my test result is accurate? 447. How can I ensure that my test result is accurate? 448. How can I ensure that my test result is accurate? 449. How can I ensure that my test result is accurate? 450. How can I ensure that my test result is accurate? 451. How can I ensure that my test result is accurate? 452. How can I ensure that my test result is accurate? 453. How can I ensure that my test result is accurate? 454. How can I ensure that my test result is accurate? 455. How can I ensure that my test result is accurate? 456. How can I ensure that my test result is accurate? 457. How can I ensure that my test result is accurate? 458. How can I ensure that my test result is accurate? 459. How can I ensure that my test result is accurate? 460. How can I ensure that my test result is accurate? 461. How can I ensure that my test result is accurate? 462. How can I ensure that my test result is accurate? 463. How can I ensure that my test result is accurate? 464. How can I ensure that my test result is accurate? 465. How can I ensure that my test result is accurate? 466. How can I ensure that my test result is accurate? 467. How can I ensure that my test result is accurate? 468. How can I ensure that my test result is accurate? 469. How can I ensure that my test result is accurate? 470. How can I ensure that my test result is accurate? 471. How can I ensure that my test result is accurate? 472. How can I ensure that my test result is accurate? 473. How can I ensure that my test result is accurate? 474. How can I ensure that my test result is accurate? 475. How can I ensure that my test result is accurate? 476. How can I ensure that my test result is accurate? 477. How can I ensure that my test result is accurate? 478. How can I ensure that my test result is accurate? 479. How can I ensure that my test result is accurate? 480. How can I ensure that my test result is accurate? 481. How can I ensure that my test result is accurate? 482. How can I ensure that my test result is accurate? 483. How can I ensure that my test result is accurate? 484. How can I ensure that my test result is accurate? 485. How can I ensure that my test result is accurate? 486. How can I ensure that my test result is accurate? 487. How can I ensure that my test result is accurate? 488. How can I ensure that my test result is accurate? 489. How can I ensure that my test result is accurate? 490. How can I ensure that my test result is accurate? 491. How can I ensure that my test result is accurate? 492. How can I ensure that my test result is accurate? 493. How can I ensure that my test result is accurate? 494. How can I ensure that my test result is accurate? 495. How can I ensure that my test result is accurate? 496. How can I ensure that my test result is accurate? 497. How can I ensure that my test result is accurate? 498. How can I ensure that my test result is accurate? 499. How can I ensure that my test result is accurate? 500. How can I ensure that my test result is accurate? 501. How can I ensure that my test result is accurate? 502. How can I ensure that my test result is accurate? 503. How can I ensure that my test result is accurate? 504. How can I ensure that my test result is accurate? 505. How can I ensure that my test result is accurate? 506. How can I ensure that my test result is accurate? 507. How can I ensure that my test result is accurate? 508. How can I ensure that my test result is accurate? 509. How can I ensure that my test result is accurate? 510. How can I ensure that my test result is accurate? 511. How can I ensure that my test result is accurate? 512. How can I ensure that my test result is accurate? 513. How can I ensure that my test result is accurate? 514. How can I ensure that my test result is accurate? 515. How can I ensure that my test result is accurate?

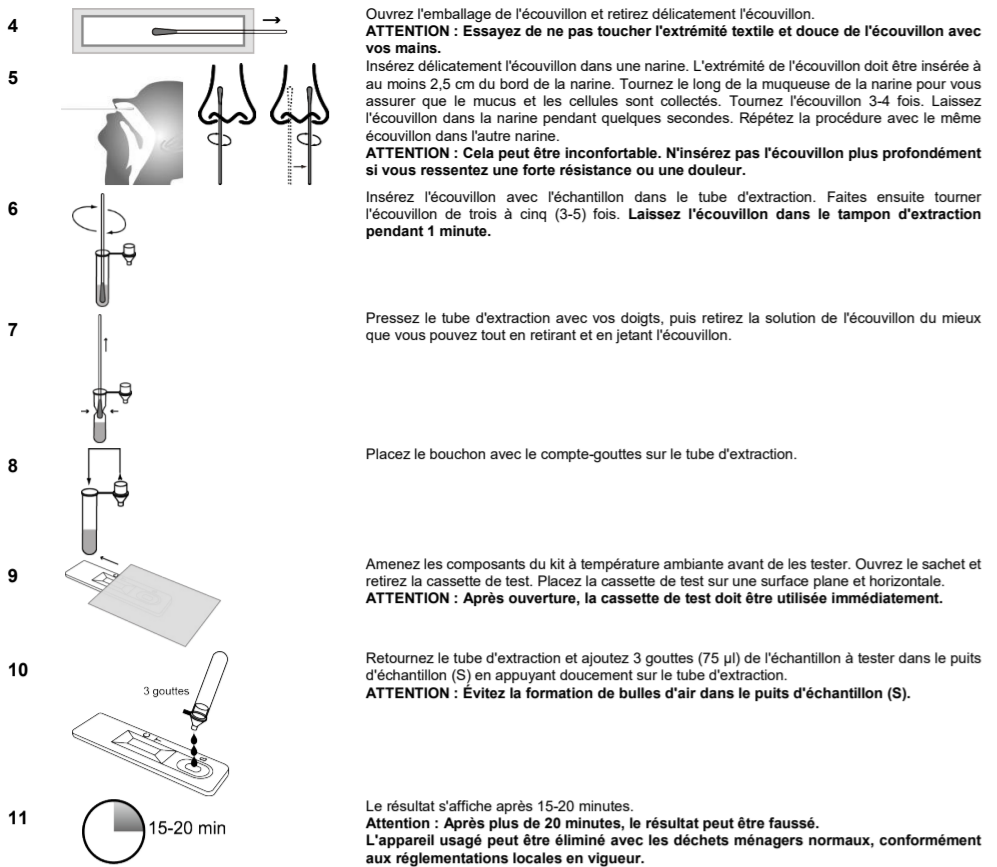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ACCESSORI Accessori Prodotto Rappresentante UE Marchio CE Tamponcino A Jiansu Changfeng Medical Industry Co., Ltd. Toubiao Town, Guangning District Yangzhou 225109 Jiangsu P.R. China Lins Service & Consulting GmbH Obere Seegasse 34/2,69124 Heidelberg Germany secondo la 93/42/CEE Tamponcino B Goodwood Medical Care Ltd. 1-2 Floor, 3-919 Yongzheng Street Jinzhou District Dalian 116100 Liaoning China CMC Medical Devices & Drugs S.L. C/ Horacio Lengua No18, CP 29006, Málaga, Spain secondo la 93/42/CEE Tamponcino C Zhejiang Gongdong Medical Technology Co., Ltd. No. 10 Beiyuan Ave., Huangyan 318020 Taizhou, Zhejiang, P.R.China Shanghai International Holding Corp. GmbH (Europe) Eiffelstrasse 80, 20537 Hamburg, Germany secondo la 93/42/CEE Tamponcino D Jiansu Hanheng Medical Technology Co.Ltd. 16-B4-81 North Qingyang Road, Tianning District, 213017, Changzhou, Jiangsu, China Luxus Lebenswelt GmbH Kochstr. 1, 47677, Willich, Germany secondo la 93/42/CEE

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