



Get started!



CareLift™ SUPPORT HARNESS

Life can be heavy, but love is strong.
Let's get your best friend back on his paws.

A dark blue geometric shape, resembling a wide 'V' or a mountain peak, is positioned at the bottom of the page, pointing upwards towards the center.

Before we start


You want to help your pet get up and around. We want to help you do this as safely and comfortably as possible.

It's important to check your dog's support harness often to be sure there is no chafing, pinching or rubbing in sensitive areas like your dog's belly or hips.

Keep the support harness clean and dry. The leg loop padding and the belly strap are machine washable.


With the right fit, your dog can wear the full support harness comfortably for several hours at a time. If you're using the rear support harness alone, it's best to take it off when your dog isn't using it.

1

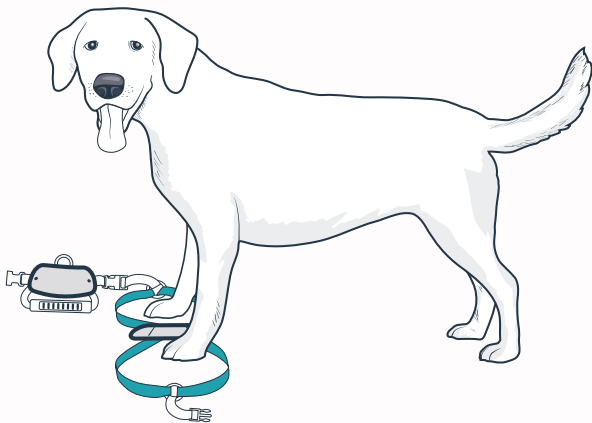

Unfasten one of the buckles on the front support harness and lay the open support harness on the ground. Gently help your dog's legs into the front leg openings, one leg at a time.


2

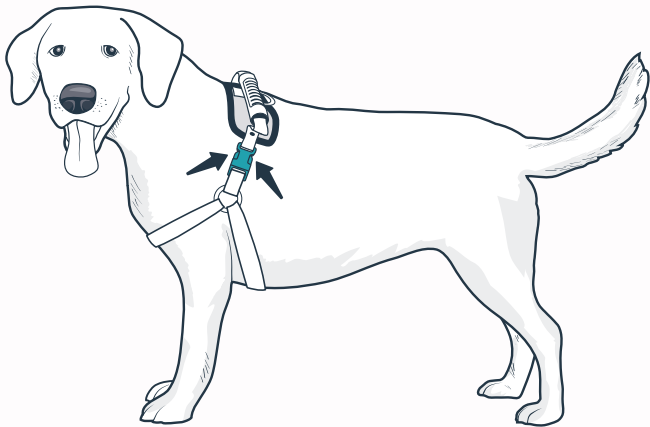

Lift the support harness up to your dog's chest and place the back panel (with the handle) over your dog's shoulders. Refasten the buckle.


Adjust the straps for a comfortable fit. You should be able to fit two fingers between your dog's fur and the straps. When the straps move, the fur should move with them.

3

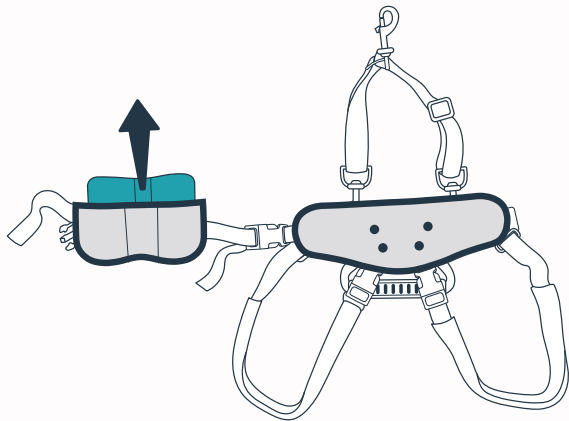


On the rear support harness, remove the male protection shield from the belly strap if your dog is a female. With a snug fit, this optional plastic insert helps relieve pressure on a male dog's sensitive areas. You can buckle the belly strap up to the side when your male dog is ready for a potty break.

4

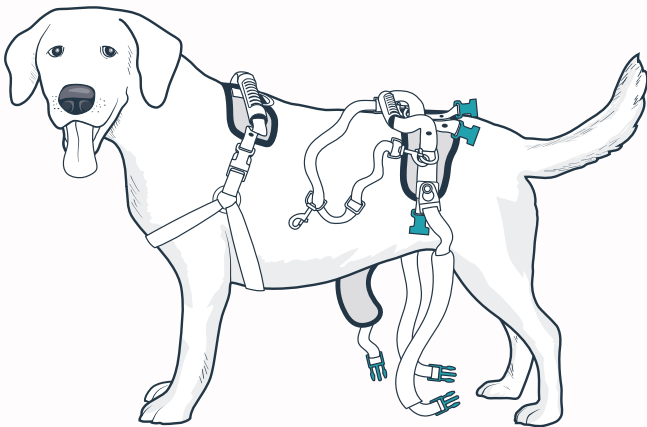

Unfasten both leg loop buckles and one of the side buckles on the rear support harness. Drape the open support harness over your dog's back with the lifting leash facing the front.


5

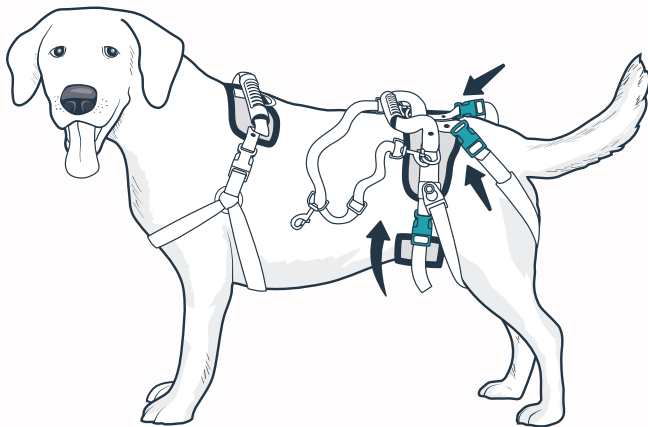

Fasten the belly strap around your dog's belly. Loop each rear leg strap under the legs and fasten the buckles over the hips.


Adjust the straps for a comfortable fit. You should be able to fit two fingers between your dog's fur and the straps. When the straps move, the fur should move with them.

6

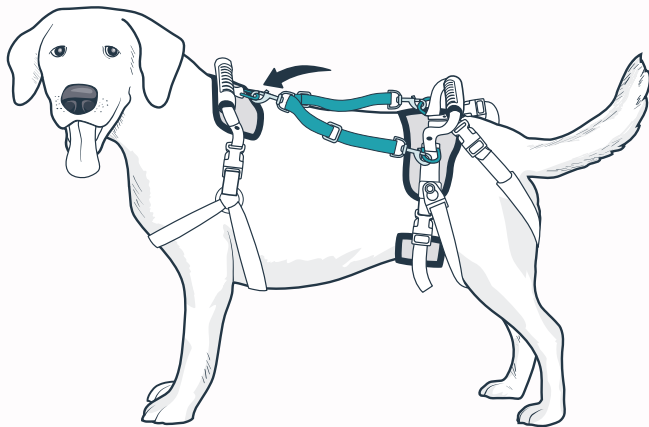

Connect the front and rear support harnesses by attaching the free clip of the lifting leash to the D-ring on the front support harness.

Adjust the lifting leash to prevent the rear support harness from slipping back off your dog's hips.

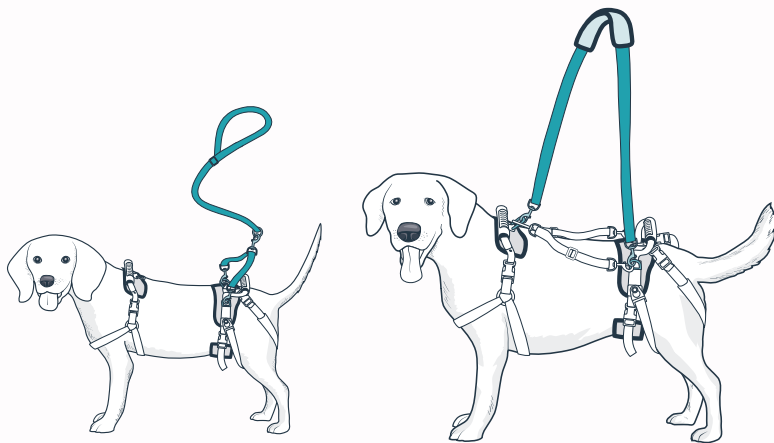

Use the lifting leash or the front and rear handles to gently help your dog get up or move around. For rear support, you can use the lifting leash extender provided with the small support harness. For heavier dogs, you can use the shoulder strap provided with the medium and large support harnesses.

We're with you every step of the way, so your dog
can be too. Let us help you find the perfect fit.

Let us help!

For questions or additional tips:

+1 (800) 732-2677

petsafe.com

1-Year Warranty

©2019 Radio Systems Corporation

Models: 62366, 62367, 62428 | JE400-2319

For important safety information, please see the customer care guide.

live *happy* together™