

Nitro rollator Nitro rollator ambulateur Nitro

item #s:

RTL10266

RTL10266BK

RTL10266WT

RTL10266-T

RTL10266BK-T

RTL10266WT-T

RTL10266-H

RTL10266BK-H

RTL10266WT-H

www.drivemedical.com

TM
drive

parts list



WEIGHT CAPACITY: 300 LBS (136 KG)

- | | |
|-----------------------------------|------------------------|
| 1. Handgrip | 7. Seat |
| 2. Adjustable tubing handgrip | 8. Bag |
| 3. Rear reflector on handgrip | 9. Front wheel |
| 4. Brake lever | 10. Brake adjust screw |
| 5. Interlock button for handgrips | 11. Rear wheel |
| 6. Back Rest | |

introduction

The Walking aid is suitable for a single user with limited mobility. The Nitro Rollator is ideal for indoor and outdoor use.

Nitro Rollator includes:

1. Bag
2. Back Rest
3. Operating Instructions


unfolding the Nitro Rollator

Position the Nitro Rollator on its wheels. Hold each hand grip and pull outward until the Nitro unlocks and opens fully (figures 1 & 2).



height adjustment

Press the red release buttons for handgrips below, as shown in figure 3, and pull the handgrip upwards at the same time. As the handle moves, release the button. This automatically locks in the next possible height. Repeat this process until the desired height is reached.

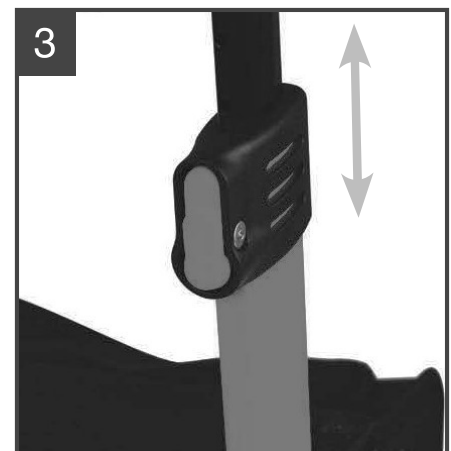

removing / attaching the carry bag

To remove bag, twist knobs located on the front end of the seat frame outward (Figure 4 & 5). Pull out knobs and slide bag out of receptacle.

To replace bag, fold the Nitro Rollator slightly and reverse above steps.



braking / locking

Pull both brake levers upwards to apply the brakes while the Nitro Rollator is moving (see figure 6). Always brake slowly and steady.

Press both brake levers down to apply the locks while the Nitro Rollator is stationary (see figure 7). Pull up the brake levers and release to release the locks.

The effect of the brakes can be affected by abrasion of the wheels.

Regularly test the brakes.



adjustment of brakes

To adjust the braking strength, adjust the brake by pressing the screw housing downwards (figure 8) and then turn left or right (figure 9).



sitting


Position the Nitro Rollator on an even and solid surface. Lock the brakes (parking position). The Nitro Rollator must stand and must not slip. Sit straight up and do not lean backwards. (See figure 10).

DO NOT MOVE ROLLATOR WHILE SEATED.



folding / transport / storage

To fold, pull seat upwards by the handle on the seat place as showing in figure 11. The Nitro Rollator can now be folded completely and secured by pushing the hand grips together until an audible 'click' is heard. This is when the underseat locking mechanism is engaged (figures 12 & 13).

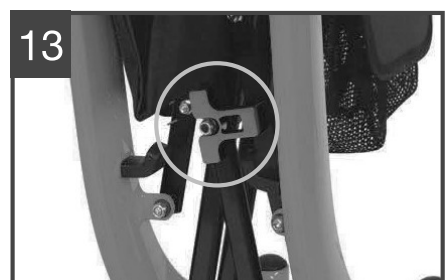

safety instructions

- Use only as a walking aid!
- Use only on even and solid surfaces.
- Maximum user weight: 300 lbs.
- Regularly check the screws and fasteners on the Nitro Rollator and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 11 lbs.
- Do not expose the product to temperatures above 104°F for an extended period of time, as this can cause the Nitro Rollator's parts to overheat and burn the user.
- Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace tires when significantly worn or damaged.
- **DO NOT MOVE ROLLATOR WHILE SEATED.** This is a walking aid only and is not to be used as a transportation device.

cleaning / maintenance

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.