

# IMPORTANT SAFEGUARDS

Read all instructions before using.

## ATTENTION:

1. When cleaning the lens, please wipe it carefully with professional alcohol cotton swabs.

3. **Don't move** around when using the product. Protect yourself from accidental knocks or bumps.

5. Children are not allowed to use this product alone and must be used in the **company of adults**.

7. Rechargeable lithium battery included, if not used frequently, please charge once a month.

9. Do not apply excessive force when using, **gently cleaning** to avoid ear scratching.

2. **Make sure the head of the ear tip is tightened and the silicone head is not loose.**

4. This product is suitable for people aged **3-60**, under **3** years old is **prohibited** to use.

6. The head of the **ear tip** is recommended to be **replaced** at least once **in 2 months**.

8. The **temperature** may rise slightly (up to **95°F**) when using.

## STORAGE:

1. Don't store in humid environments or soak in liquids to avoid damage.

3. Avoid the product from being pressed, especially the head of the ear tip, to prevent the lens or the head of the ear tip from being crushed.

2. Do not directly expose the product to the sun, especially ear tip head fittings to avoid product softening.

4. To prevent the product from falling from a high place and breaking the head of the ear tip.



Loose ear tip



Silicone head off



Don't replace tip in time



Pressed to cause breakage



Use while moving



Use by children alone



With too much strength



Expose to the sun