

KLD-9218 FLYINGJOY ROLLATOR WALKER

Instruction Manual

NOTICE: When you unpacking it please check that if any accessories were damaged during shipping. if the accessories was damaged do not use it, please consult with the retailer.

Safety Inform:

Do not use this rollator walker without reading or fully understanding this instruction manual, please save the instruction manual for reference. If you do not understand the warnings, precautions, etc. in the manual, please consult the seller or technician to install the rollator walker before use, avoid injuries caused by unqualified instruction.


Instructions for Safe Use:

1. When installing the front and rear tubes, it must be noted that all button maintain a uniform height. Make sure that the adjustment holes are consistent. If the above warnings are not followed, it will cause injury.
2. Please ensure the balance of the rollator walker during use. Do not lean your body in front of the rollator to prevent tipping over when the center of gravity changes.
3. Do not pull away the rollator walker violently, so as not to damage the accessories.
4. If the user's weight exceeds the maximum weight of this product, please do not use this product. (The maximum weight capacity of this product is 300LBS.)
5. Please regularly check the connection accessories and wheels for damage to ensure safe use.
6. Do not push while sitting on the rollator walker. This product is not a wheelchair and is only used to assist walking.

Features:

1. This rollator walker is comfortable, durable and stable in performance;
2. It's easy to disassemble and assemble, easy to carry and store;
3. The height of the handlebar can be adjusted according to personal height;
4. The height of the seat is adjustable;
5. The product has a bag for storage;

Accessories List

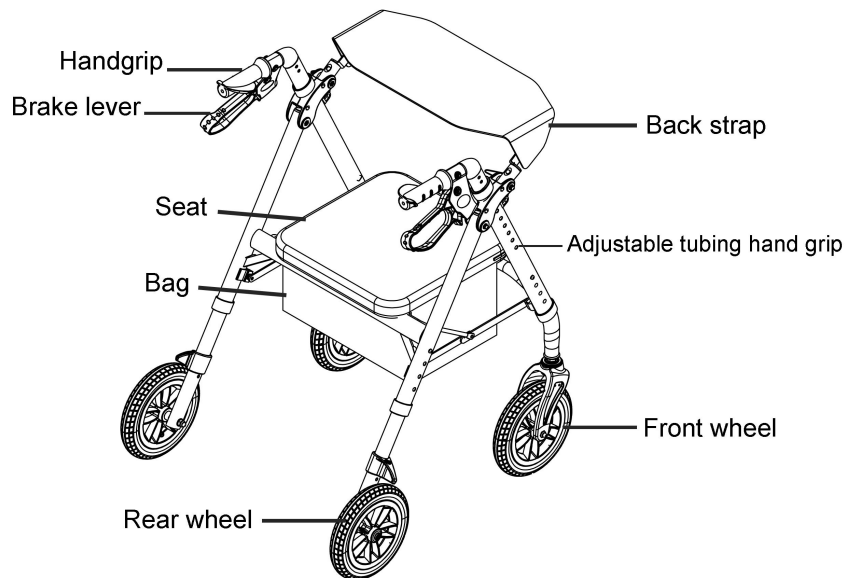

Product Usage:

Provide assistance for short-distance walking for the disabled or the elderly.

Product Specifications:

Armrest handle height adjustment range: 31.22-37.8 inch(+1CM)/793-960mm

Seat height: 18.5-21.46 inch/470-545mm

Total length: 25.6 inch/650mm

Total width: 24.61 inch/625mm

Total height: 37.8 inch /960mm


GW : 15.2LBS/6.9KG

User weight: 300LBS

Installation Instruction:

[open]


1. Open the carton, take the product out carefully, as shown in (P1)

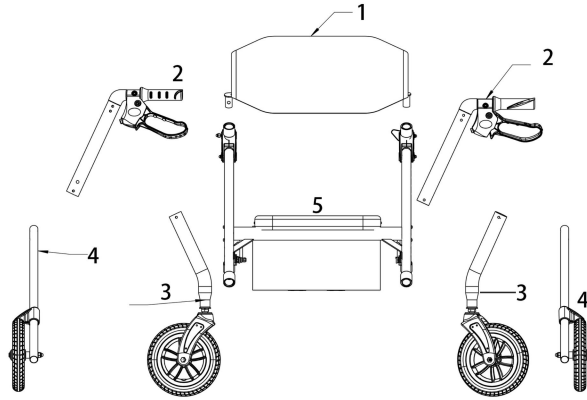

(P1)

[Install]

2.Take out the frame, grab the handle and press down until the frame is unfolded (P2)

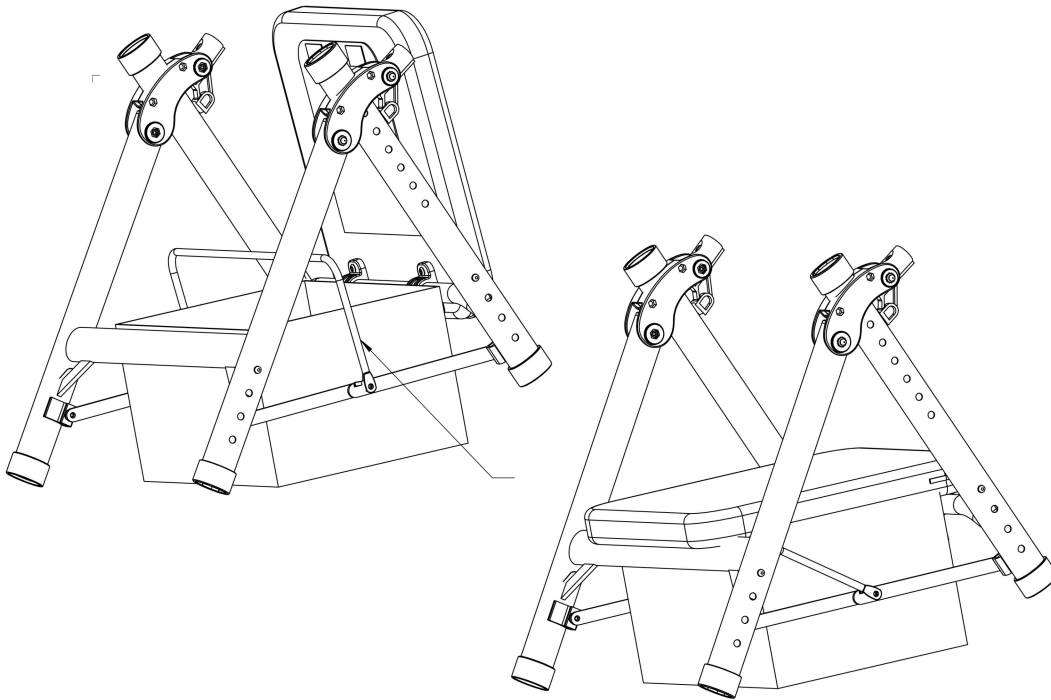


(P2)


Frame folded state

Frame unfolded state


3. When installing product, insert the handrail tube into the front frame tube of the frame, and the rear wheel into the rear frame tube. Adjust the height through the button on the tube as shown in the picture (P3)

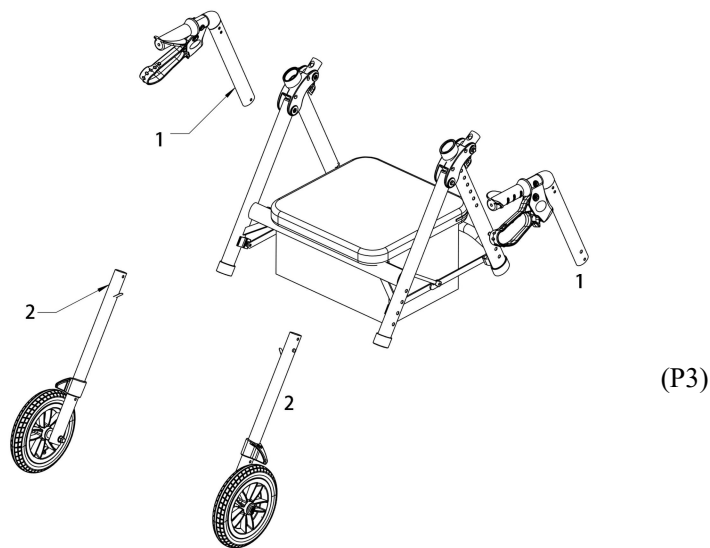

4. Installation complete diagram

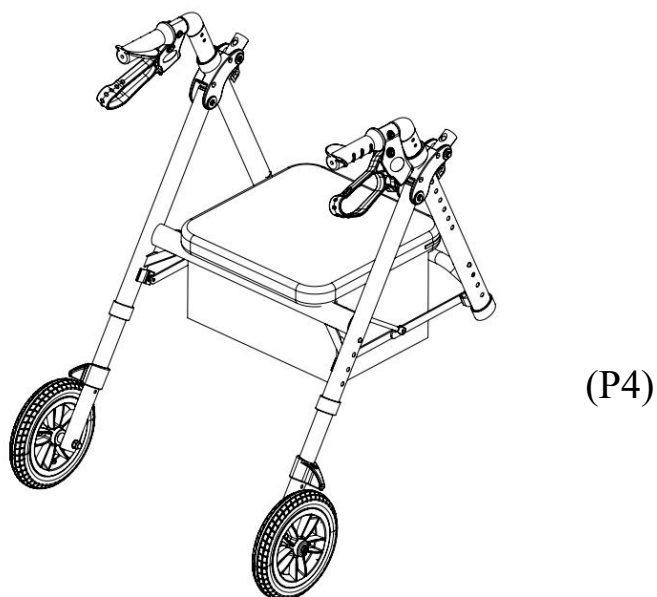


5. After completing step 1 of installation, Take out the front wheel set

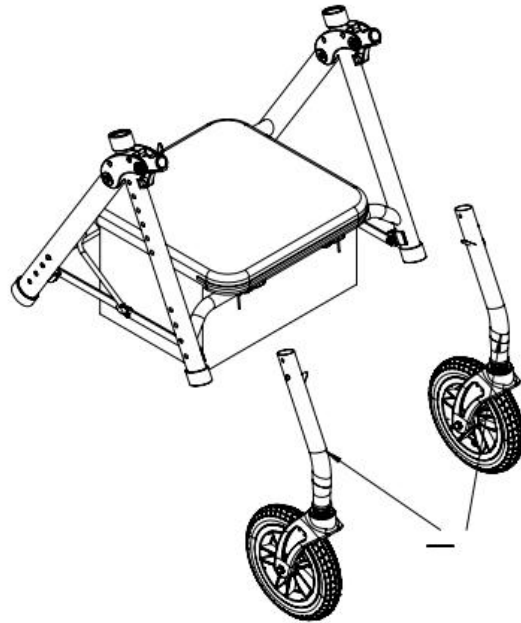


(P5)

6.Insert the front wheel into the frame nozzle,use marbles to adjust the position.Notice:When the front wheel is installed,pay attention to the direction of the marbles on the front wheel tube and the bending direction of the front wheel tube.

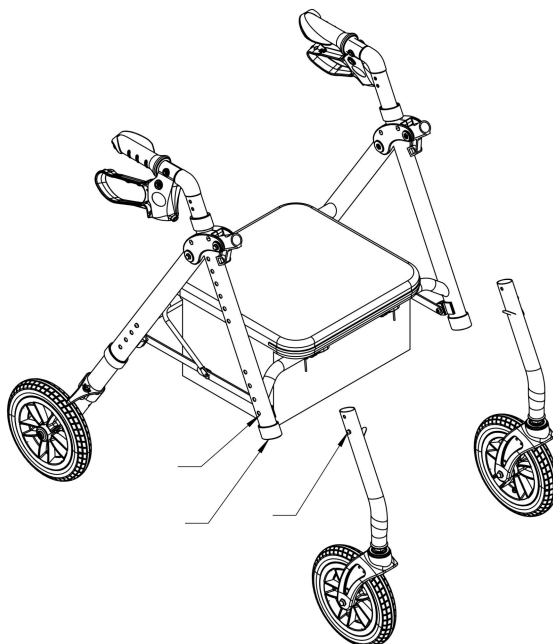


(P6)

7.Installation complete diagram

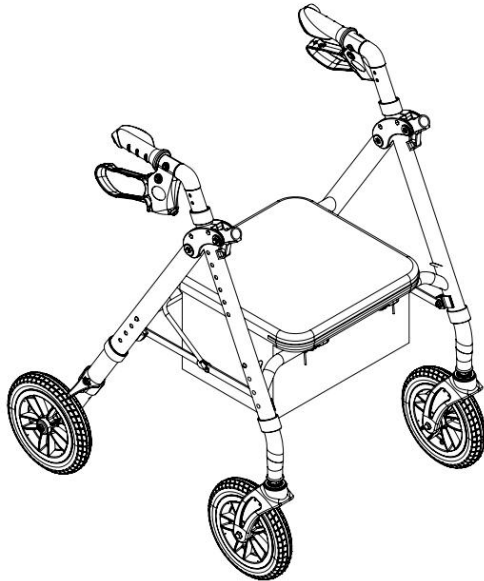


(P7)

8.After installation is complete, take out the backrest

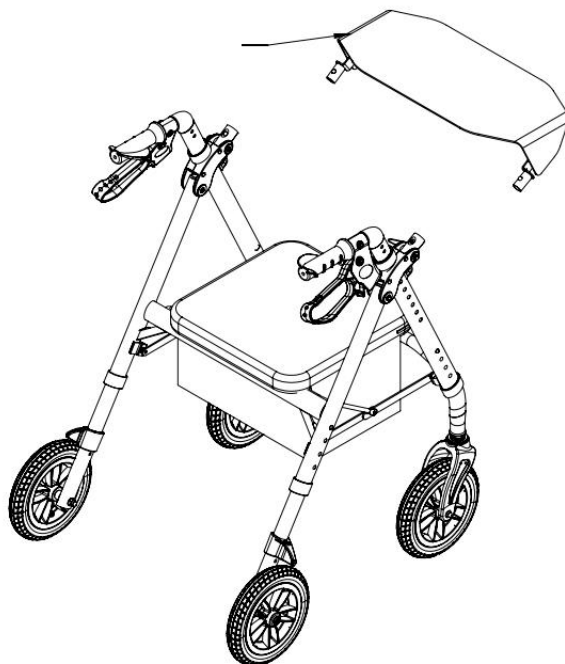


(P8)

9.Insert the backrest tube into the backrest swivel sleeve and secure it with marbles.

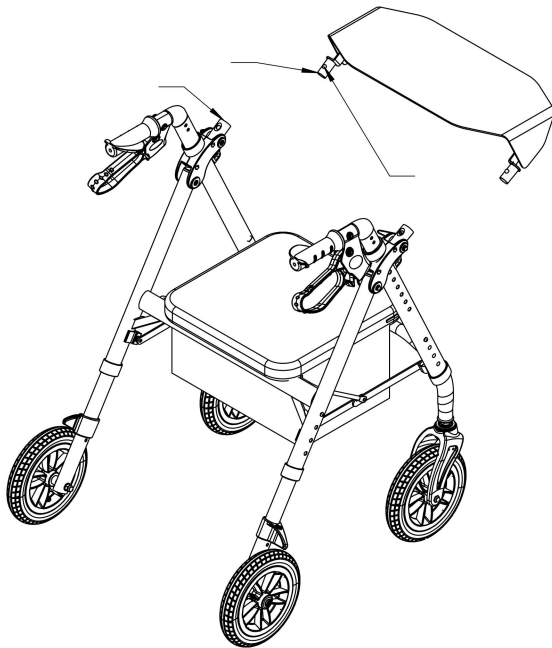

(P9)

10. Installation complete diagram

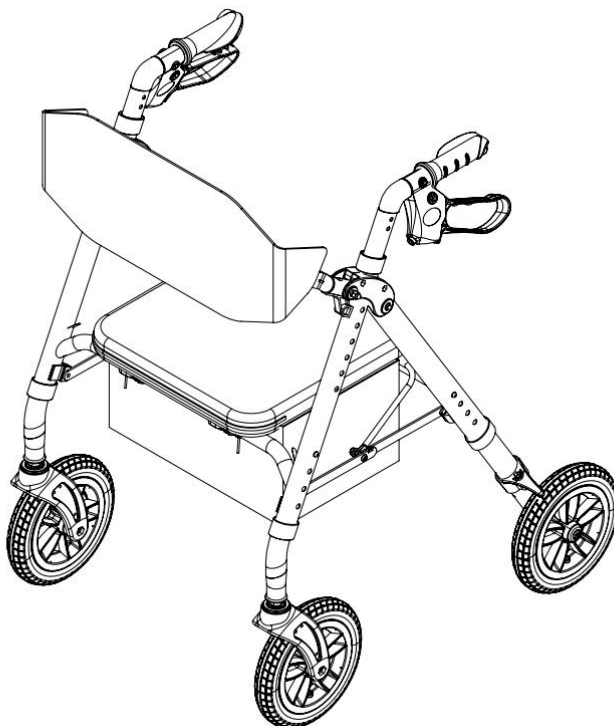

(P10)



FOSHAN CRADLE MEDICAL EQUIPMENT TECHNOLOGY CO.,LTD

Repair and Maintenance:

1. If the product fails problems, please do not disassemble it if you do not understand the situation, avoid compromising other functions. Please contact the retailer or manufacturer;
2. Keep product surfaces clean with normal neutral detergents;
3. Store product in a ventilated and dry place to avoid oleic acid substances.

Maintenance Period and Scope:

1. The rollator walker has one year warranty.
2. During the warranty period, users can repair or replace accessories free of charge due to faults caused by product workmanship or material defects;
3. Rollator walker parts (It is normal for the wheels and brake pads to wear when using this product) the product in natural wear, intentional man-made damage or improper operation, does not belong to the scope of free warranty.