FILTER TECHNOLOGY

ENGLISH

INSTRUCTIONS FOR USE

GVS ELIPSE PARTICULATE SAFETY RESPIRATOR

» The GVS Eclipse Safety Respirator is approved as a halfmask in accordance with NIOSH Standards 42 CFR Part 84, the GVS Eclipse filter is part of the approved assembly and only when the above items are used together is the device approved for occupational applications.

APPLICATIONS

» The GVS Eclipse half-mask together with HESPA particulate air filters give protection against both solid and liquid hazardous and toxic dust particles, metal fumes, mist and micro-organisms (e.g. bacteria and viruses).

» The P100 classification provides minimum efficiency of 99.97% and is symbolised by magenta colour code. The protection provided by classification P100 also covers the protection provided by lower class P99 and P95 filters.

» The GVS Eclipse half-mask is available as a valved or non-valved version. The non-valved version offers P100 filtration efficiency on both inhalation and exhalation, protecting both the wearer and other people.

SELECTION OF RESPIRATORY PROTECTION

» Respiratory protection must be selected by a competent person familiar with workplace hazards and their concentration.

» Guidelines for the selection and use may vary between countries. Always check prior to use to ensure that your selection for the intended application meets national requirements.

FITTING INSTRUCTIONS

A The respirator must be inspected by the wearer before and after each use to ensure that it is in good working condition and valve diaphragms (if included) and filters are seated correctly.

B Remove eyewear (if worn). If facial hair goes beyond the border of the mask an effective seal will not be achieved. Alternative RPE may be required.

① Grasp the respirator with one hand and the lower headband with the other hand. Put the headband over your head and allow to rest around the neck.

② Place the respirator over mouth and nose, then lift the upper headband and position over the crown of the head to support the respirator.

③ Grip the upper headband ends at the respirator buckle, and pull gently towards back of head to tighten.

④ Grip the lower headband ends at the respirator buckles and gently pull towards back of head to tighten.

» Check respirator is seated correctly when tight enough to prevent air leakage between respirator and face.

» **NOTE:** OSHA regulations require that all users be fit tested prior to use either qualitatively or quantitatively.

» **FACE FIT CHECK** every time the mask is worn to ensure the respirator is seated correctly and tight enough to prevent air leakage between respirator and face. Place the palm of your hand over the exhalation valve, or over the filters for the non-valved version and exhale moderately. If the face piece bulges slightly and no air leaks between the face and the face piece are detected, a proper fit has been achieved. If air leakage is detected, reposition the respirator on the face and/or readjust the elastic strap to eliminate the leakage.

CLEANING & MAINTENANCE

» **DO NOT CLEAN OR DISINFECT FILTERS.**

⑤ Clean and disinfect the face-piece assembly periodically with a soft cloth dampened with mild soapy water. This will form part of the facemask maintenance and should be logged.

» **FACE-PIECE DEEP CLEANING: DISASSEMBLE** the respirator by removing the filters, head straps and valve diaphragms (if included). Immerse the facepiece in warm water and scrub with a soft brush. A neutral detergent can be used if necessary. The face-piece can also be soaked in a disinfectant for a short period of time. It should then be rinsed in clean, warm water and allowed to air dry in a non-contaminated atmosphere.

CHANGING FILTERS

» Filters should be changed once a month or more frequently if you find a resistance to breathing. A maintenance record should be kept of these filter changes. Filter fitting instructions are supplied with replacement filters (please refer for full details).

STORAGE

» The respirator should be stored and transported out of direct sunlight, away from sources of high temperature, in a cool dry atmosphere at temperature between -10°C to 40°C, in an uncontaminated environment and in the original packaging.

» The nearest item of packaging is individually marked with the month and year of shelf life expiry.

» Shelf life is 5 years for P100 / P100 Nuisance OV filters and mask.

» Products must be discarded when they reach their shelf life date.

» Storage or transportation under conditions other than those specified may affect shelf life.

CAUTIONS & LIMITATIONS

A - Not for use in atmospheres containing less than 19.5% oxygen.

B - Not for use in atmospheres immediately dangerous to life or health.

C - Do not exceed maximum use concentrations established by regulatory standards.

J - Failure to properly use and maintain this product could result in injury or death.

L - Follow the manufacturers Users Instructions for changing cartridges, canisters and/or filters.

M - All approved respirators shall be selected, fitted, used and maintained in accordance with MHSA, OSHA and other applicable regulations.

N - Never substitute, modify, add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.

O - Refer to users instructions and/or maintenance manuals for information on use and maintenance of these respirators.

P - NIOSH does not evaluate respirators for use as surgical masks.

Δ **ALWAYS** conduct a pressure face fit check prior to each use. If the check fails the mask may be faulty or require replacement depending on age, frequency of use and replacement of filters.

Δ The use of this respirator presupposes exact knowledge of the instructions for use, applicable health and safety standards, guidelines for selection and use, and appropriate training in matters of personal respiratory protection.


Δ Leave the contaminated area immediately if:

- » Dizziness or distress is experienced
- » You taste or smell contaminants or irritation occurs
- » Breathing becomes difficult

» A defect is discovered in the respirator

Δ Store respirator as per instructions given above.

Δ Before occupational use of this respirator a written respiratory protection program must be implemented meeting all the local government requirements. In the United States employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training, and fit testing.

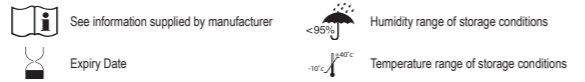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####

ENGLISH

GVS Filter Technology UK
Vickers Industrial Estate
Morecambe, Lancashire, UK
Tel: +44 (0) 1524 847600

GVS Filter Technology UK
Vickers Industrial Estate
Morecambe, Lancashire, UK
Tel: +44 (0) 1524 847600

GVS Filter Technology UK
Vickers Industrial Estate
Morecambe, Lancashire, UK
Tel: +44 (0) 1524 847600

GVS Filter Technology UK
Vickers Industrial Estate
Morecambe, Lancashire, UK
Tel: +44 (0) 1524 847600

THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:

TC-	PROTEC- TION ¹	ALTERNATE FILTERS		ALTERNATE FACEPIECE				CAUTIONS & LIMITATIONS ²
		40.069.0006	40.069.0076	51.039.0003	40.039.0005	51.039.0045	51.039.0046	
84A-6949	P100	X		X	X			ABCJLMNOP
84A-6950	P100		X	X	X			ABCJLMNOPS
84A-9320	P100	X				X	X	ABCJLMNOP

1. PROTECTION

P100 – Particulate Filter (99.97% filter efficiency level). Effective against all particulate aerosols.

2. CAUTIONS AND LIMITATIONS

- A - Not for use in atmospheres containing less than 19.5% oxygen.
- B - Not for use in atmospheres immediately dangerous to life or health.
- C - Do not exceed maximum use concentrations established by regulatory standards.
- J - Failure to properly use and maintain this product could result in injury or death.
- L - Follow the manufacturers User Instructions for changing cartridges, canisters and/or filters.
- M - All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations.
- N - Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O - Refer to users instructions and/or maintenance manuals for information on use and maintenance of these respirators.
- P - NIOSH does not evaluate respirators for use as surgical masks.
- S - Special or critical User Instructions or specific use limitations apply. Refer to User Instructions before donning.

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