

Quick Guide


With a minimal appearance and ergonomic design, our digital hearing amplifier is portable, easy to operate, and comfortable to wear all day.



8-Channel Digital Hearing Amplifier

Instruction

- The amplifier comes with 4 pairs of silicone caps. Choose the one that fits best. Wrong cap size may cause feedback whistling.
- Each time the device is used for more than 15 minutes, the system will automatically save the mode and volume in use and set it as default setting for future convenience.
- In Tinnitus Concealment mode, the device will produce white noise only to cover the tinnitus and help relief the discomfort.

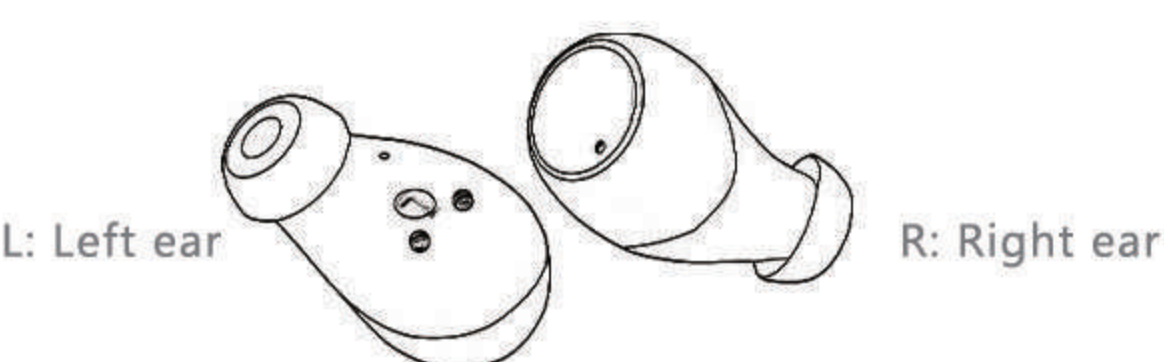


How to put on

- Clean the ear canal. Push the hearing amplifier into the canal.
- Gently turn the hearing amplifier left and right until it is stable and comfortable.

2

Operation Instruction

- 1. On and Off**
1. Press and hold the key for 5s to turn the device on.
2. Press and hold the key for 5s again to turn it off.
*When take out from the charging case, hearing aid will automatically turn on;
When put back into the charging case, hearing aid will automatically turn off.
- 2. Switch modes**
Press and hold the key for 2s to switch among 3 different modes:
-Normal listening mode
-Noise environment mode
-Tinnitus masking mode
- 3. Adjust Volume**
A quick press on the key will adjust the volume from minimum ("Du.. Du" signals) to maximum ("Di... Di... Di" signals) and then back to minimum again if continuing pressing the key.

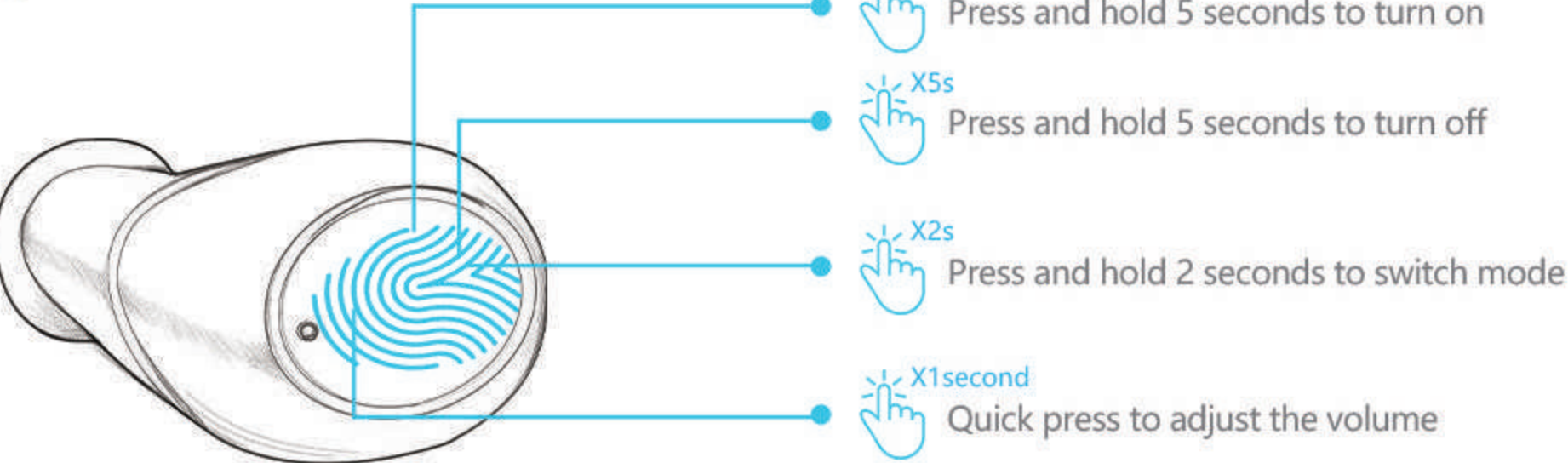


3

Battery

Capacity: 45mAh
Usage time: 45 hours (Continuous use)
Capacity of charging case: 400mAh
Size of charging case: 65*38*26.5mm

Charging

The indicator light is on when battery is in charging and turns off automatically when charging is done. The average charging time is around 2 hours.

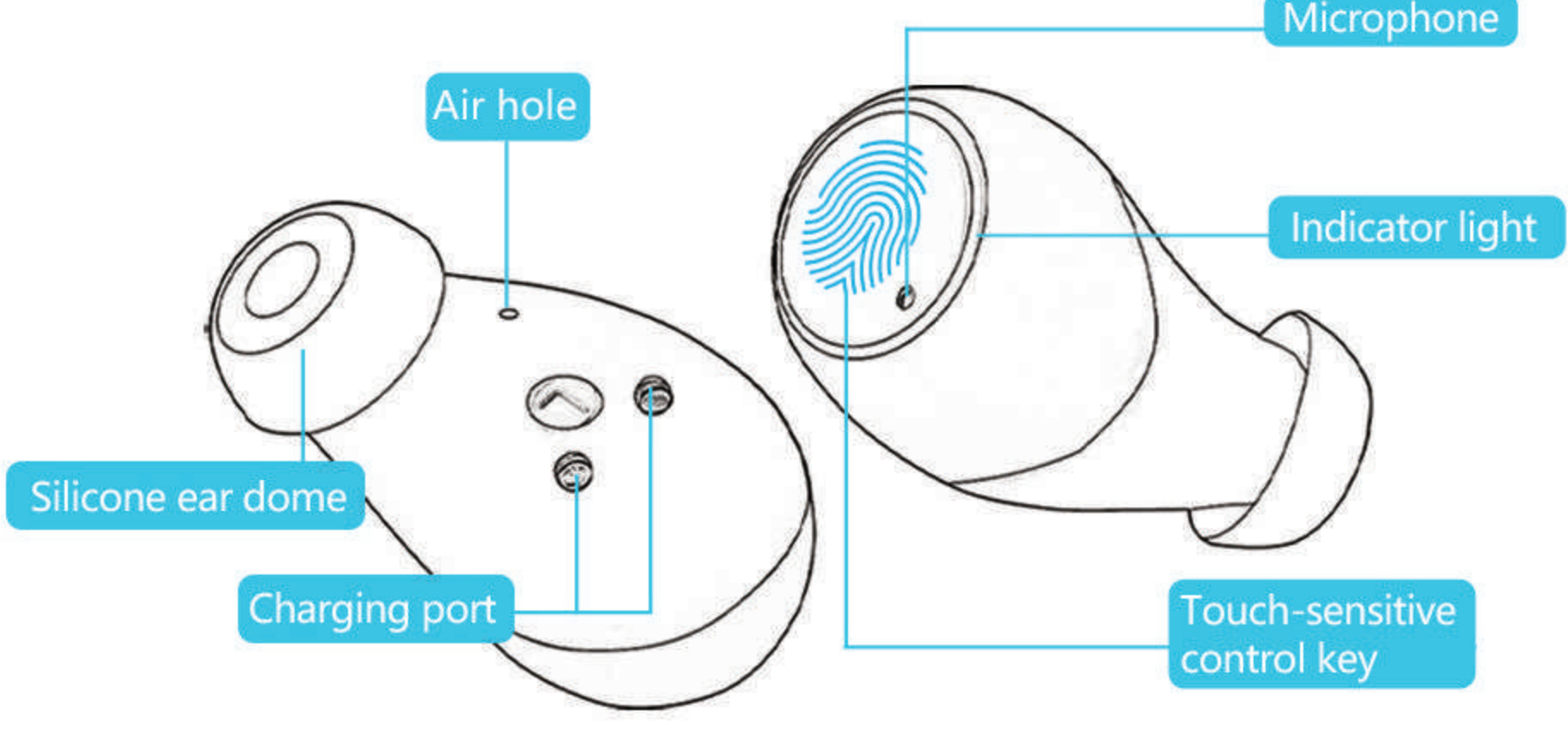


4

Specifications

Maximum Output	≤115dB SPL	Total Harmonic Distortion (THD)	500Hz: 0.9%
Average Output	106±4dB SPL		800Hz: 0.6%
Peak Gain	38±5dB		1600Hz: 0.4%
Average Gain	33±5dB	Equivalent Input Noise	≤12dB
Frequency Range	350Hz~5200Hz	Operating Power Supply	≤1.1mA@3.7V

Accessories

1. Charging case	*1	5. Silicone ear dome	4 pairs
2. Hearing amplifier	1 pair	6. User's Manual	*1
3. Charging line	*1	7. Quick Guide	*1
4. Cleaning brush	*1		

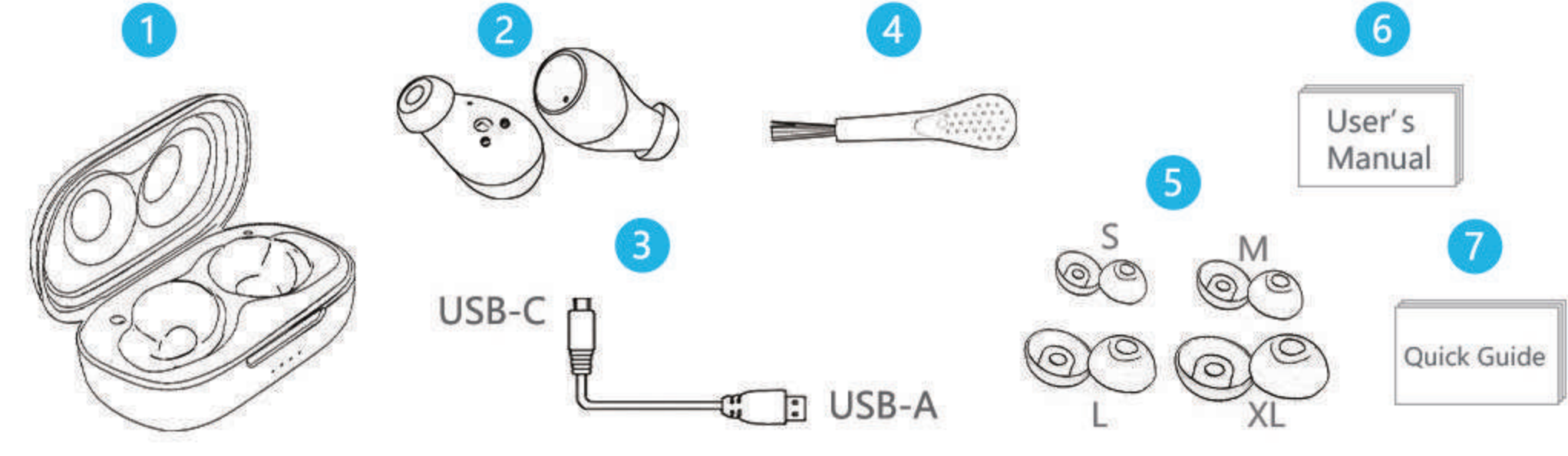

5

Cleaning

- Clean the silicone ear dome with the sound hole facing down.
- Clean the dust screen with the cleaning brush. Remove the silicone ear dome if needed.
- Clean the exterior with disinfecting wipes.
- Store the hearing amplifiers in the charging case.

6