One Step HCG Test Strip Instruction Insert

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INTENDED USE

MomMed One Step HCG Test Strip is an in vitro diagnostic home use test device intended for the rapid determination of human chorionic gonadotropin (hCG) in urine to aid in the early detection of pregnancy. You can easily carry out yourself.

It is intended for self-testing use.
For in vitro diagnostic use only.

HOW DOES IT WORK

Human chorionic gonadotropin (hCG) is a hormone, produced by the developing placenta and secreted into the urine. The pregnancy test contains antibodies which specifically react with this hormone.

When the sample end of the test is dipped in the urine specimen, the urine specimen is applied to the strip and migrates along the strip. When hCG in the sample reaches the Test Zone region of the membrane, it will form a colored line. Absence of this colored line suggests a negative result.

A colored line will appear at the Control Zone region if the test has been performed properly.

CONTENT OF THE TEST KIT

1. One pouch containing a reaction strip.
2. Leaflet with instructions for use.
3. Urine Cups.

WHAT ELSE DO YOU NEED?

Timer or clock.

PRECAUTIONS

1. This kit is for external use only. Do not swallow.
2. Discard after first use. The test strip cannot be used more than once.
3. Do not use the test past the expiration date printed on the foil pouch.
4. Do not use the kit if the pouch is punctured or not well sealed.
5. Keep out of the reach of children.

STORAGE AND STABILITY

1. Store at 39°F to 86°F (4°C to 30°C) in the sealed pouch up to the expiration date.
2. Keep away from direct sunlight, moisture and heat.
3. DO NOT FREEZE.
4. You should perform the test soon after opening the pouch. You should keep the test in a dry environment before use.

WHEN TO COLLECT URINE FOR THE TEST?

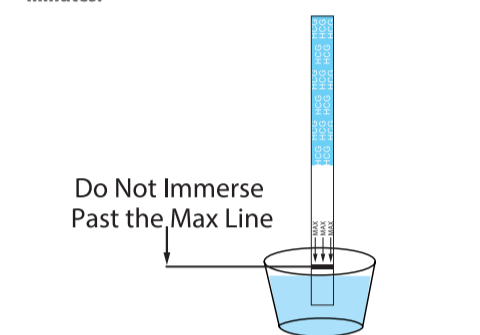
For many women, HCG may be detectable with this test by the day after the missed period. Any urine specimen is appropriate for Pregnancy Testing but the first morning urine specimen is optimal because of its highest concentration of HCG.

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HOW TO DO THE TEST?

1. The test strip and urine have to be at room temperature 59-86°F (15-30°C) for testing.
2. Remove the test strip from the sealed pouch.
3. Immerse the strip into the urine with the arrow pointing towards the urine. Take the strip out after 5 seconds and lay the strip flat on a clean, dry, non-asorbent surface (such as the mouth of the urine container).
4. Read results in five minutes. Positive results may sometimes appear as early as one minute but you should wait the full 5 minutes before determining a negative result. **Do not read results after more than 5 minutes.**

HOW TO READ THE RESULTS?

Negative (not pregnant):

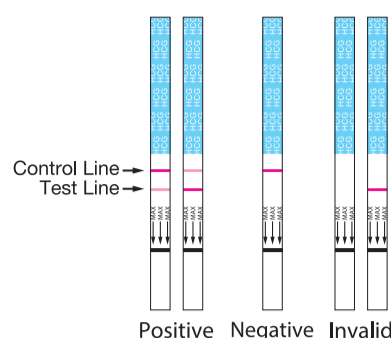
Only one color band appears, in the Control Zone. No apparent band on the Test Zone. This indicates that no pregnancy has been detected.

Positive (pregnant):

Distinct color bands appear in the Control and Test Zones. It indicates that you are pregnant.

Invalid:

No visible band at all, or there is a visible band only in the test region and not in the control region. Repeat with a new test kit.

NOTE: If the color in the Test Zone is weak, it is recommended that the test be repeated in 48 hours, or if the test shows negative but you suspect you may still be pregnant, the test may be repeated after a few days.

WHAT TO DO WHEN YOU KNOW THE RESULT

IF THE RESULT IS POSITIVE (TWO LINES)

Consult your physician. He or she should confirm your pregnancy after evaluating all clinical and laboratory findings. In some rare situations the test may give a false positive result. He or she will also give you further information and advice on your pregnancy.

IF THE RESULT IS NEGATIVE (ONE LINE)

If you have reasons to believe that you are pregnant (e.g. no menstrual bleeding), you can repeat the test a few days later. If the test is still negative, consult your physician. See also "Understanding test results".

UNDERSTANDING TEST RESULTS

1. If a urine sample is too dilute (ie, low specific gravity) it may not contain a representative level of hCG. If pregnancy is still suspected, another urine specimen should be collected 48 hours later and tested.
2. Low concentration of hCG in a very early pregnancy can give a negative result. In this case, another specimen should be obtained at least 48 hours later and tested.
3. Any treatment with hCG may cause false results.

EXPECTED RESULTS

Although the hCG levels in normal early pregnant women are variable, One Step HCG Test is generally capable of detecting pregnancy as early as 1 day after the first missed menses.

QUESTIONS & ANSWERS

1. There is much difference between the control lines of two tests. Is this a concern?
No. Variations in the color of the control band will not affect the test result.
2. Can test results be interpreted after more than five minutes?
No. Test results must be read in 5 minutes. A negative result may change to a false positive within minutes after the end of the testing period, which would not be an accurate reading. It is always best to read the results at the 5 minute testing period and then discard the test to avoid confusion.

PERFORMANCE CHARACTERISTICS

Sensitivity:

One Step HCG Test has a sensitivity of 25 mIU/ml.

Interfering Substances:

The following potentially interfering substances were added to hCG negative and positive specimens, no interferences were observed. Acetaminophen, aspirin, ascorbic acid, atropine, caffeine, glucose, hemoglobin, tetracycline, ampicillin, albumin, bilirubin.

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INDEX OF SYMBOLS

CE 0123	CE Mark	IVD	In vitro diagnostic medical device
Read instructions carefully before use			Contains sufficient for < n > tests
Temperature limitation 4-30 °C			Do not reuse
Manufacturer	EC REP		European Representative
keep away from sunlight			keep dry
LOT	Batch code		Expiry date

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MomMed

One Step LH Test Strip Instruction Insert

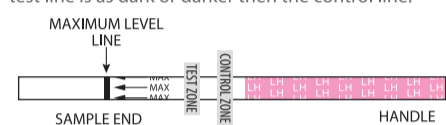
Please read all the information in this insert before performing the test.

INTENDED USE

One Step LH Test Strip is an in vitro diagnostic test device intended for the rapid determination of human Luteinizing Hormones (LH) in urine to predict when there is a LH surge, and in turn, when you are likely to ovulate. You can easily carry out yourself. It is intended for self-testing use. For in vitro diagnostic use only.

HOW DOES IT WORK

The LH (Luteinizing Hormone) which is in the urine of normal women will increase dramatically in the middle of the menstrual cycle. The LH increase triggers ovulation which is when the egg is released periodically from Normal fertile women. WHO experts state that LH testing is a reliable way to detect ovulation. Ovulation will occur after 24-48 hours following a positive test. The ovulation test contains antibodies which specifically react with LH. When the sample end of the test is dipped in the urine specimen, the urine specimen is applied to the strip and migrates along the strip. A colored line will appear at the Control Zone region if the test has been performed properly. The control line also works as a reference line for you to find the LH peak time when the test line is as dark or darker than the control line.

CONTENT OF THE TEST KIT

- One pouch containing a reaction strip.
- Leaflet with instructions for use.
- Urine Cups
- Time Tracking Chart

WHAT ELSE DO YOU NEED?

Timer or clock.

PRECAUTIONS

- This kit is for external use only. Do not swallow.
- Discard after first use. The test strip cannot be used more than once.
- Do not use the test past the expiration date printed on the foil pouch.
- Do not use the kit if the pouch is punctured or not well sealed.
- Keep out of the reach of children.

STORAGE AND STABILITY

- Store at 39°F to 86°F (4°C to 30°C) in the sealed pouch up to the expiration date.

- Keep away from direct sunlight, moisture and heat.
- DO NOT FREEZE.
- You should perform the test soon after opening the pouch. You should keep the test in a dry environment before use.

WHEN TO COLLECT URINE FOR THE TEST?

The length of the menstrual cycle is the duration from your first menstrual bleeding day to the day before the next bleeding begins. Determine the length of menstrual cycle before test. Please refer to the chart below to determine when you should start testing. If your cycle is shorter than 21 days or longer than 38 days, consult a physician.

If you do not know your cycle length, you may begin the test 11 days after your first period since the average cycle length is 28 days. Perform 1 test each day over a 5 days period, or until the LH surge has been detected.

Cycle Length	Day of Cycle (CD) to Begin Testing
21 days	day 5
22 days	day 6
23 days	day 7
24 days	day 8
25 days	day 9
26 days	day 10
27 days	day 11
28 days	day 12
29 days	day 13
30 days	day 14
31 days	day 15
32 days	day 16
33 days	day 17
34 days	day 18
35 days	day 19
36 days	day 20
37 days	day 21
38 days	day 22
39 days	day 23
40 days	day 24

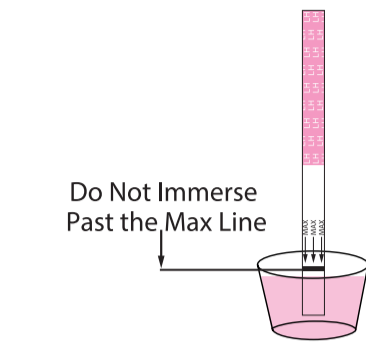
HOW TO COLLECT URINE?

Urine specimens may be collected in any clean, dry, plastic or glass container.

HOW TO DO THE TEST?

- The test strip and urine have to be at room temperature 59-86°F (15-30°C) for testing.
- Remove the test strip from the sealed pouch.

- Immerse the strip into the urine with the arrow pointing towards the urine. Take the strip out after 5 seconds and lay the strip flat on a clean, dry, non-absorbent surface (such as the mouth of the urine container). **IMPORTANT: Do not allow the urine level to exceed the MAX (marker line), otherwise the test will not perform correctly.**
- Read results in five minutes. **Do not read results after more than 5 minutes.**

HOW TO READ THE RESULTS?

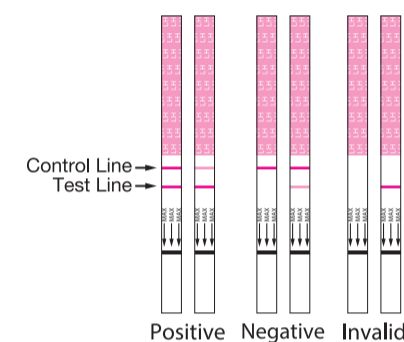
Negative (No LH Surge): Only one color band appears on the control region, or the test band appears but is lighter than the control band. This means there is no LH surge.

Positive (LH Surge): If two color bands are visible, and the test band is equal to or darker than the control band, one will probably ovulate in the next 24-48 hours. If trying to get pregnant, the best time to have intercourse is after 24 but before 48 hours.

Invalid: No visible band at all or there is a visible band only in the test region and not in the control region. Repeat with a new test kit.

UNDERSTANDING TEST RESULTS

- The test works only when the test procedures are precisely followed.
- This test may not be used as a form of birth control.
- Consult a doctor if irregular or unusually long cycles are experienced.

QUESTIONS & ANSWERS

- My cycle length is outside of the range show in the table. How will I know when to start testing? Simple start testing 17 days before you expect your next period.
- I do not know my usual cycle length. How will I know when to start testing? If you wish to start using the test before knowing your cycle length, we recommend you start testing on day 10.
- What time of day should I take the test? You may test any time of day, but you should test at approximately the time each day. You should not have urinated for at least 4 hours before testing.
- There is much difference between the control lines of two tests. Is this a concern? No. Variations in the color of the control band will not affect the test result. It is only effective to compare the color of the test band to that of the control band of the same device on the day the test is performed.
- Can test results be interpreted after more than five minutes? No. Test results must be read at 5 minutes. Though a positive result should not change for several days, a negative result may change to a false positive within minutes after the end of the testing period, which would not be an accurate reading. It is always best to read the results at the 5 minute testing period and then discard the test to avoid confusion.
- How long should I continue to perform the test? At least 5 days or until the LH surge has been detected.
- Do alcohol or common medications affect the test? No, but you should consult your physician if you are taking any medication.
- Once I see a positive result, when is the best time to have intercourse? Ovulation is likely to occur within 24-48 hours. This is your most fertile time. Sexual intercourse is advised within this time frame.
- Will the amount of liquid I drink affect the result? We suggest that you limit your fluid intake for about two hours before you collect your urine. Because heavy intake of fluids prior to testing will dilute the hormone in your urine.

PERFORMANCE CHARACTERISTICS

- Sensitivity:** One Step LH Test has a sensitivity of 25 mIU/ml.
- Specificity:** Cross reaction studies has been tested with known amounts of hormones. 200mIU/ml hFSH and 250uIU/ml hTSH were added to LH negative and positive specimens, no cross reaction were observed.
- Interfering Substances:** The following potentially interfering substances were added to LH negative and positive specimens, no interferences were observed. Acetaminophen, aspirin, ascorbic acid, atropine, caffeine, glucose, hemoglobin, tetracycline, ampicillin, albumin, bilirubin.

- Stability after first opening** The results of the package opened stability studies show that the products quality was stable and valid within 1.5 hours. Given variations in temperature and humidity that may occur we suggest to use the test immediately after opening.

Learn more information, please register on mommedtech.com to be our member of MomMed, you will get **free samples and big discount on next order**.

If you have any question, please contact us by email right away:

Support@mommedtech.com

Also if you are satisfied with our products, please kindly share your experience on Amazon by writing a review. As a small and growing business, your honest review helps and motivates us to constantly improve our products and customer service.

INDEX OF SYMBOLS

CE 0123	CE Mark	IVD	In vitro diagnostic medical device
Read instructions carefully before use			Contains sufficient for <n> tests
Temperature limitation 4-30 °C			Do not reuse
Manufacturer		EC REP	European Representative
keep away from sunlight			keep dry
Expiry date		LOT	Batch code

MANUFACTURED BY

Co-Innovation Biotech Co. Ltd
No. 9 Baihe 3 Street, Economic And Technological Development East Zone, Guangzhou 510530, Guangdong P.R. CHINA
Tel:+86-20-82109823 Fax:+86-20-82109823

MANUFACTURER	Co-Innovation Biotech Co. Ltd No. 9 Baihe 3 Street, Economic And Technological Development East Zone, Guangzhou 510530, Guangdong P.R. CHINA TEL:+86-20-82109823 FAX:+86-20-82109823
EC REP	Shanghai International Holding Corp. GmbH (Europe) Eiffestrasse 80, 20537Hamburg, GERMANY TEL: +49-40-2513175 FAX: +49-40-255726 P/N: 181204 Rev. 12.2018

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