



Moogco Silver Nursing Cups Manual





What Are Silver Nursing Cups?

Moogco Silver Nursing Cups are a natural, effective solution designed to support breastfeeding mothers. Made from high-quality silver, these nursing cups help soothe, heal, and protect the nipples, providing relief from common breastfeeding issues such as soreness, cracks, and inflammation.

Silver has natural antimicrobial and anti-inflammatory properties, making these cups an excellent choice for maintaining nipple health and enhancing the breastfeeding experience.

Safety and Security Information

Moogco Silver Nursing Cups are free from harmful chemicals and preservatives. The cups do not contain nickel. Unlike creams and ointments, they do not contain any potentially harmful substances that could affect you or your baby.

Silver is hypoallergenic and safe for use on sensitive skin. If you have a known allergy to silver, consult your healthcare provider before use.

Ensure your cups are clean before each use to maintain hygiene and prevent infections.



How to Use Silver Nursing Cups

Clean the Cups: Before the first use and after each subsequent use, clean your Silver Nursing Cups thoroughly (see the cleaning instructions below).

Position the Cups: Place the cups directly over the nipples after breastfeeding or whenever you need relief. Ensure the cups are centered on the nipples. Before placing the cups over your nipples, add a few drops of breast milk into each cup as breast milk contains natural healing properties to further aid recovery

Wear a Bra: To keep the cups in place, wear a well-fitting, comfortable bra. The cups should fit snugly without causing pressure or discomfort.

Duration of Use: You can wear the cups throughout the day and night, removing them only during breastfeeding sessions and cleaning them regularly. We recommend you start using Silver Nursing Cups 7-10 days before giving birth as a preventative measure.

If you use the optional silicone pads, place them over the cups to prevent slipping and to wear them discreetly under clothes.

How to Clean and Store



Initial Rinse: Rinse the cups with warm water immediately after use.

Mild Soap: Use a mild, fragrance-free soap to clean the cups. Avoid harsh chemicals and abrasive materials that could damage the silver.

Rinse Thoroughly: Rinse the cups thoroughly to remove all soap residue.

Dry Completely: Allow the cups to air dry completely before the next use. Do not use cloth towels or paper towels, as they can leave fibers or contaminants on the cups.

Store in a clean, dry place: Once dry, store your Moogco Silver Nursing Cups in a clean, dry place to prevent any contamination.





Tarnish Removal

Cold Water and Baking Soda: If tarnishing of the silver occurs, wash the cups by immersing them in cold water and baking soda (50 g per liter) until the stain has gone.

Sandpaper: For any persistent stain, use our sandpaper gently on the tarnished area.

Expiration

Moogco Silver Nursing Cups are durable and designed for long-term use. However, regular inspection is recommended:

Visual Check: Periodically check for any signs of wear or damage. If the cups become damaged or tarnished beyond cleaning, it is time to replace them.

Effectiveness: if you follow these storage guidelines, the product will not have an expiry date



Silver has been used in medicine for more than a century to treat various types of skin diseases. It has been found to release substances that perform antibacterial and healing actions in the presence of moisture.

The shape of Moogco Silver Nursing Cups allows them to fit perfectly on the breast, protecting the nipples from friction caused by clothing or external agents.

These cups offer significant savings in both financial and environmental terms. Once opened and stored properly, they can be reused for future pregnancies by the same mother. This means you don't have to purchase additional products, such as creams or single-use disposable nipple shields.

Thank you for choosing Moogco Silver Nursing Cups. We are honored to be part of your breastfeeding journey and are dedicated to supporting you and your baby every step of the way.



For more information
cs@moogco.com | www.moogcobaby.com