

Important Steps When Using the A1CNow® Self Check System

A1CNow®

Self Check


Review A1CNow Self Check Overview and Quick Reference Guide insert for full testing instructions before using this abbreviated guide.

1 Preparing to test — Match lot numbers and open Pouch 1*




IMPORTANT:
Do not open Pouch 2 until Step 3 as shown below

2 Collecting the right amount of blood (Wash hands or clean with alcohol and dry before testing.)

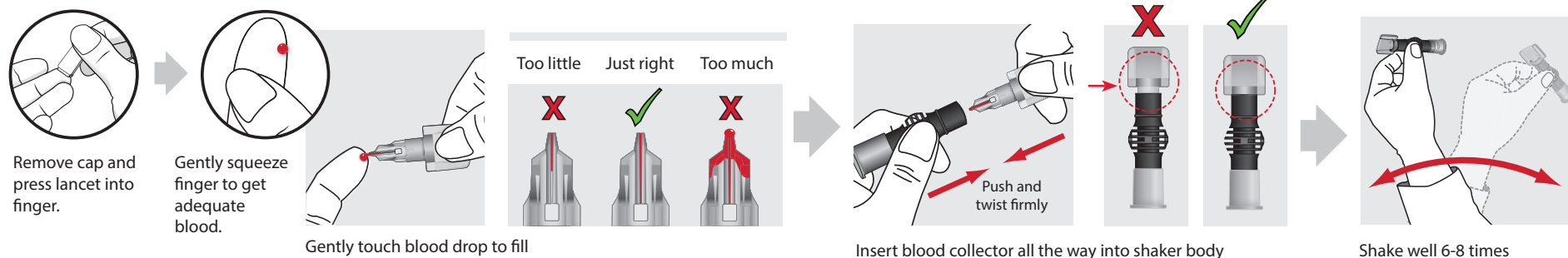

3 Open Pouch 2*. Insert cartridge, wait for SMPL to appear and dispense sample

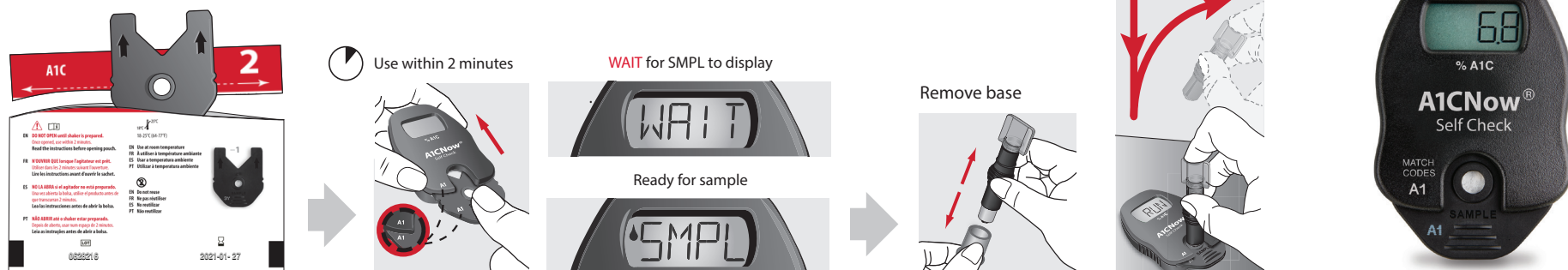

* Pouch illustrations may vary.

A1CNow is a trademark of Polymer Technology Systems, Inc. ©2021 Polymer Technology Systems, Inc. LIT 002020 Rev. 1 02/2021



Do not handle the analyzer until test is complete; results in 5 minutes