AUTOMATIC UPPER ARM BLOOD PRESSURE MONITOR M3 (HEM-7131-E)

All for Healthcare

IM-HEM-7131-E-01-12/2017
3125149-9A

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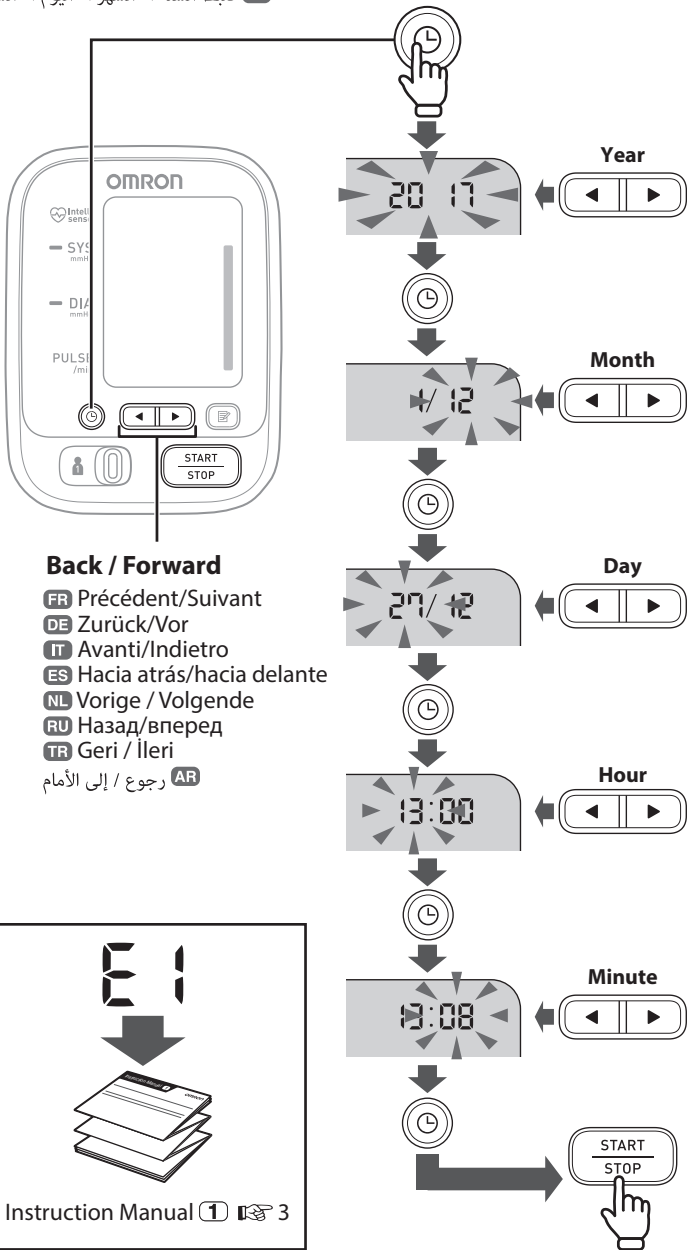
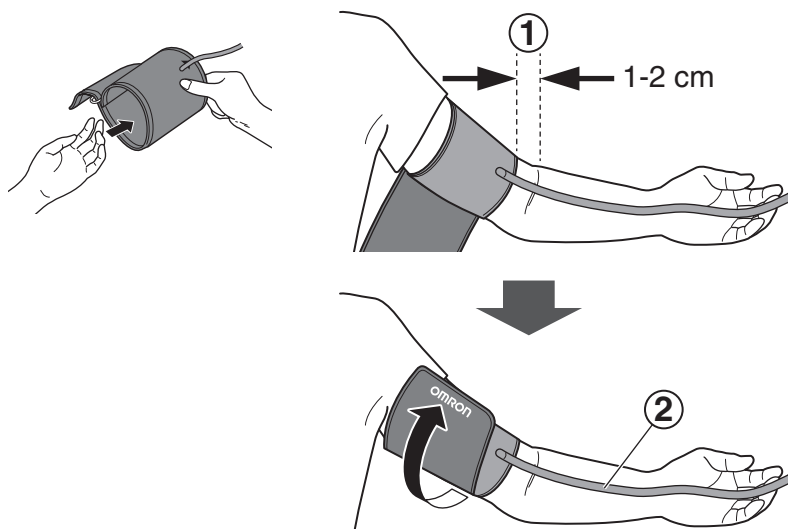
① Tube side of the cuff should be 1 - 2 cm above the inside elbow.

- FR Le côté du tuyau du brassard doit se situer de 1 à 2 cm au-dessus de l'intérieur du coude.
DE Die Schlauchseite der Manschette sollte sich 1 – 2 cm oberhalb der Ellenbogen-Innenseite befinden.
IT Il lato del bracciale con il tubo deve trovarsi da 1 a 2 cm sopra l'interno del gomito.
ES El lado del tubo del manguito debe estar de 1 a 2 cm por encima del codo.
NL De kant van de armmanschet waar zich de slang bevindt moet zich 1 tot 2 cm boven de binnenkant van de elleboog bevinden.
RU Сторона манжеты для подсоединения трубки должна располагаться на 1–2 см выше локтя.
TR Manşetin tüp tarafı dirsekten 1 ila 2 cm yukarıda olmalıdır.

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② Make sure that air tube is on the inside of your arm and wrap the cuff securely so it can no longer slip round.

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If taking measurements on the right arm, refer to:

- FR Si les mesures s'effectuent sur le bras droit, référez-vous à :
DE Bei Messung am rechten Arm, siehe:
IT Se si prendono le misurazioni sul braccio destro, fare riferimento a:
ES Si toma las mediciones en el brazo derecho, consulte:
NL Als u metingen uitvoert op de rechterarm, raadpleeg dan:
RU При выполнении измерений на правой руке см. следующий раздел руководства по эксплуатации:
TR Sağ kolda ölçüm yaparsanız, şunlara dikkat edin:
AR إذا أخذت قياسات على الذراع اليمنى، راجع:

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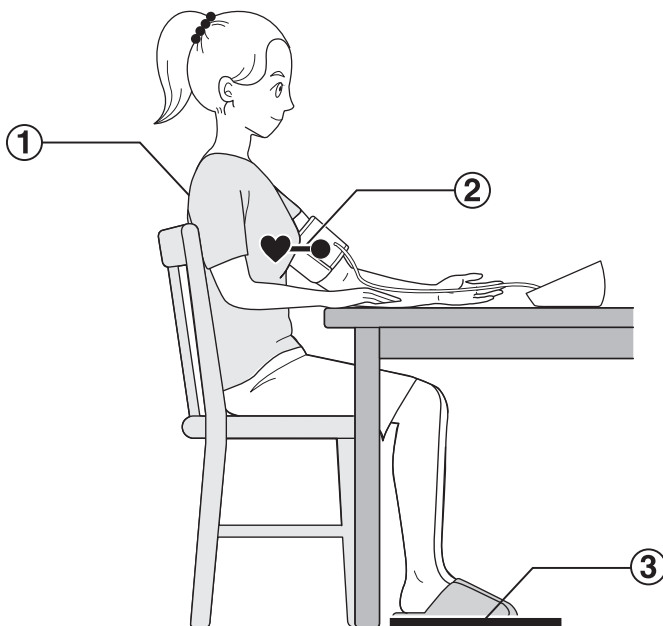
① Sit with your back and arm supported.

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② Place the arm cuff at the same level as your heart.

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③ Keep feet flat, legs uncrossed, remain still and do not talk.

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
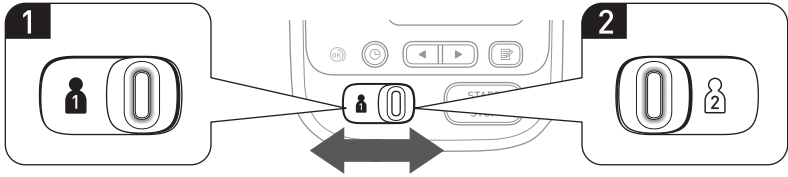
Switching user ID enables you to save readings for 2 people.

- FR Le changement d'identifiant d'utilisateur vous permet d'enregistrer les résultats de 2 personnes.
DE Durch Umschalten der Benutzer-ID können Sie Messwerte für 2 Personen speichern.
IT La modifica dell'ID utente consente di salvare i valori di 2 soggetti.
ES Cambiar la ID de usuario le permite guardar las lecturas de 2 personas.
NL Door van gebruikers-ID te veranderen, kunt u metingen van 2 personen opslaan.
RU Переключение идентификатора пользователя позволяет сохранять показания для 2 людей.
TR Kullanıcı ID'si değiştirme, 2 kişilik okumaları kaydetmenizi sağlar.
AR يمكنك التبديل بين معرفات المستخدم من حفظ قراءات شخصين.

8 Taking a Measurement

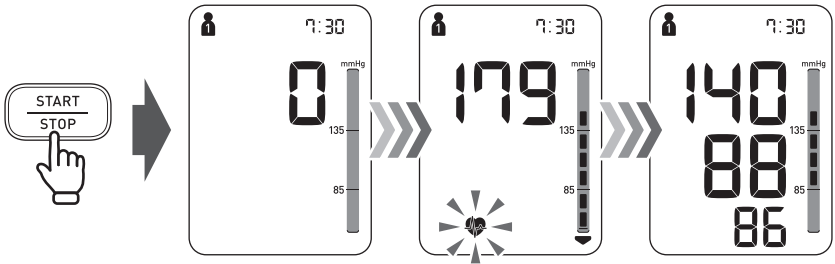- FR Prise d'une mesure NL Een meting uitvoeren
DE Durchführen einer Messung RU Выполнение измерения
IT Esecuzione della misurazione TR Ölçüm Yapmak
ES Tomar una medición AR أخذ قياس

When the START/STOP button is pressed, the measurement is taken and saved automatically.

- FR Lorsque le bouton START/STOP est enfoncé, la mesure est effectuée et enregistrée automatiquement.
DE Wenn die START/STOP-Taste gedrückt wird, wird die Messung durchgeführt und die Messwerte werden automatisch gespeichert.
IT Quando si preme il pulsante START/STOP, viene eseguita la misurazione e viene salvata automaticamente.
ES Al presionar el botón START/STOP se toma la medición y se guarda automáticamente.
NL Als de START/STOP-knop wordt ingedrukt, wordt de meting uitgevoerd en automatisch opgeslagen.
RU При нажатии кнопки START/STOP измерение выполняется и сохраняется автоматически.
TR START/STOP düğmesine basıldığında, ölçüm otomatik olarak alınır ve kaydedilir.
AR عند الضغط على زر START/STOP، يأخذ القياس ويحفظ تلقائياً.

9 Indication Lights and Symbol Explanations

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The high blood pressure definition is based on the JNC 7 standard.

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Irregular heartbeat symbol: Appears when an irregular rhythm is detected during a measurement. If it continues to appear, it is recommended to consult your physician. An irregular heart beat rhythm is defined as a rhythm that is 25 % less or 25 % more than the average rhythm detected during a measurement.

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Movement error symbol : Appears when your body moves during a measurement. Remove the arm cuff, wait 2-3 minutes and try again.

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E1

E2

Error messages or other problems? Refer to:

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