


SAFETY INFORMATION

The Sleeve Stars Ankle Brace is designed to support recovery from various foot-related conditions, such as plantar fasciitis, Achilles tendinitis, swelling, and sprains. Its adjustable design ensures a snug, secure fit for all-day comfort while promoting mobility and reducing discomfort.

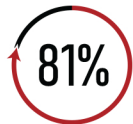

Polyester



Rubber



Features and Benefits

This ankle brace offers targeted compression to help reduce swelling, improve blood circulation, and alleviate pain. It is crafted from breathable, sweat-absorbent materials that keep feet dry and odor-free even during prolonged use. The adjustable straps provide customized support to stabilize weak joints and speed up recovery. The innovative 3D weaving technology ensures the brace is highly durable and stretchable, maintaining its shape and effectiveness over time.

Product Specifications

The Sleeve Stars Ankle Brace is made from a combination of 81% polyester and 19% rubber, ensuring both durability and flexibility. It is designed to fit most users, with an adjustable size range suitable for arch circumferences between 10 and 12 inches. The sleeve measures 10 inches (25 cm) in height and 3.5 inches (9 cm) in width, while the strap measures 20.5 inches (52 cm) in length and 3 inches (7.5 cm) in width. The lightweight design makes it convenient for both daily wear and travel.

Safety Information

Before using the Sleeve Stars Ankle Brace, it is recommended to consult a physician, especially if you plan to wear it for extended periods. Ensure that the brace is not overtightened, as this can restrict blood flow. Use the product only on clean, intact skin, and do not use it if you are allergic to rubber or polyester. If irritation, pain, or discomfort occurs, discontinue use immediately and consult a doctor. This product is not a toy and should be kept out of reach of children.

How to Use

To use the Sleeve Stars Ankle Brace effectively, first position the sleeve over your foot and ensure it fits securely. Then, wrap the strap around your ankle, adjusting it for the desired level of compression. Once adjusted, fasten the strap in place to ensure stability and comfort. Proper application ensures optimal support and enhances the product's effectiveness in reducing discomfort and promoting recovery.

Care Instructions

To maintain the quality and durability of the Sleeve Stars Ankle Brace, wash it in a machine on a warm setting. Avoid using bleach, tumble drying, or ironing, as these may damage the product. Air drying is recommended to preserve its structure and elasticity, ensuring that the brace remains effective for long-term use.

Certifications and Disclaimer

The Sleeve Stars Ankle Brace is FDA-cleared and made from high-quality materials, ensuring it is safe and effective for everyday use. The manufacturer is not responsible for improper use or application of the product. Always follow the recommended usage guidelines and consult a healthcare professional for advice tailored to your specific needs.