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EN TOE SPACER ENG Instructions for Use

Thank you very much for the trust you have placed in this medical product from the Restore Feet brand. Please read these usage instructions carefully. If you have any questions, please contact your doctor or our customer service at info@restorefeet.com.

Intended Use / Indications

Toe separators are designed to relieve tension and can be used to:

- Accompany a hallux valgus treatment
- Alleviate bunion discomfort while walking
- Prevent overlapping of the toes
- Ensure a gap between the toes

Contraindications / Important Instructions

Sensory and circulatory alterations in the treated body area, skin diseases in the treated area. Do not place the product on open wounds. Do not use it if you have diabetes or sensory disorders in the feet (neuropathy). In case of irritation, discomfort, or poor circulation in the toe, remove the product immediately and contact a doctor. Not suitable for children.

Side Effects

Rarely allergic reactions to the skin or sensitivities related to the materials may occur, as it is made with SEBS and talcum powder.

Instructions

To apply the toe separator, make sure to have clean and dry skin. With both hands, gently hold the big toe and slide the pad between the toes. A gentle rocking motion forward and backward facilitates placement and removal. It is important not to do it with one hand to avoid damaging the separator. This product fits perfectly to the shape of the toes and adheres firmly without slipping, can be used on both feet. For best results, it is recommended to wear toe separators during the day with flat and wide shoes, or at home with socks or barefoot. Start with 30 minutes daily and gradually increase the usage time.

Sizes

One size fits all, can be used on the left and right.

Material Composition

100% SEBS, talcum powder

Washing Instructions

Hand wash in mild soapy water at a maximum of 30°C. Let the product air dry; protect it from direct heat and do not dry it in the sun. For optimal comfort and increased durability, lightly dust with talcum powder afterward.

Customer Service / Contact

Do you have questions about the product and its application or are not satisfied with your purchase? Our team guarantees to find a satisfactory solution. Contact us at info@restorefeet.com.

Discover how to restore your feet

You can get our direct link from the QR code on the thank you card or directly visit RestoreFeet.com.

Storage

Store in a dry place, away from direct sunlight, out of the reach of children.

Usage Duration / Product Lifespan

The lifespan of the medical product is determined by natural wear when used correctly and according to the instructions. Regularly sprinkle with talcum powder to extend the lifespan.

Notification Obligation

Please note that all serious incidents (serious temporary or permanent health deterioration, or a serious risk to public health) related to the product must be reported to us and the competent authority in the country where you are established. info@restorefeet.com

Disposal

To be disposed of with household waste.

Version: 04/2024

Single patient - multiple use.

	CONSULTESE LAS INSTRUCCIONES DE USO	GEBRAUCHSANWEISUNG BEACHTEN	OBSERVER LA NOTICE EXPLICATIVE	LEGGERE ATTENTAMENTE E IL FOGLIO ILLUSTRATIVO	OBSERVES INSTRUCTIONS LEAFLET
	FECHA DE FABRICACIÓN	HERSTELLUNGSDATUM	DATE DE FABRICATION	DATA DI FABBRICAZIONE	DATE OF MANUFACTURED
	FABRICANTE	HERSTELLER	FABRICANT	PRODUTTORE	MANUFACTURER
	NUMERO LOT	LOT-NUMBER	NUMÉRO LOT	LOTTO-NUMERO	LOT NUMBER
	NUMERO DEL ARTICULO	ARTIKELNUMMERN	NUMÉRO DE L'ARTICLE	NUMERO DELL'ARTICOLO	ARTICLE NUMBER
	DISPOSITIVO MEDICO	MEDIZINPRODUKT	DISPOSITIF MEDICAL	DISPOSITIVO MEDICO	MEDICAL DEVICE

restorefeet

CUSTOMER SERVICE / CONTACT / KONTAKT / CONTATTO
[INFO@RESTOREFEET.COM](mailto:info@restorefeet.com)
TEL / WHATSAPP: +34 685 591 625

REF ZSA4 2023-04 LOT 2303-ZSA4 MADE IN CHINA

Shijiazhuang Aofeite Import&export Co., Ltd East and southwest side in the fourth floor in the NO.202 of Gree supporting industrial park, No.389 of Hengshan street, High-tech Zone, Shijiazhuang, Hebei Province, China, 050000.

GUIIMI EVOMEDICAL SL, Av de Sant Esteve 157, 08211 Castellar del Vallès, Spain.

SUNGO Europe B.V, Olympisch Stadion 24,
1076DE Amterdam, Netherlands.
Tel: (+31) (0) 2021 11106 Email: info@sungoglobal.com

WE TAKE CARE OF YOUR FEET

BUNION CORRECTOR

WITH CORRECTIVE ELASTIC BAND

ALIGN. STRENGTHEN. RESTORE.

FOLLOW THE ENCLOSED INSTRUCTIONS FOR USE AND KEEP THIS PRODUCT OUT OF THE REACH OF CHILDREN.

APPLICATION:

APPLY THE TOE SPREADER ONTO CLEAN AND DRY SKIN. WITH BOTH HANDS, GENTLY GUIDE IT OVER THE BIG TOE, ENSURING THE PAD SITS SNUGLY BETWEEN THE TOES. A SUBTLE ROCKING MOTION BACK AND FORTH FACILITATES THE EASE OF BOTH APPLICATION AND REMOVAL. IT'S IMPORTANT TO AVOID USING JUST ONE HAND, AS THIS MAY LEAD TO POTENTIAL DAMAGE TO THE TOE SPREADER.

POSITION THE BUNION CORRECTOR ON THE INNER SIDE OF THE FOOT, TARGETING THE MOST STRESSED AREA. THE TOE SPREADER SEAMLESSLY ADJUSTS TO THE NATURAL SHAPE OF THE TOES, PROVIDING A SECURE GRIP WITHOUT ANY RISK OF SLIPPING. THIS VERSATILE PRODUCT IS SUITABLE FOR USE ON EITHER FOOT.

FOR OPTIMAL RESULTS, WEAR THE TOE SPREADERS THROUGHOUT THE DAY, EITHER IN FLAT, WIDE SHOES OR AT HOME WITH SOCKS OR BAREFOOT. BEGIN WITH A DAILY ROUTINE OF 30 MINUTES AND GRADUALLY EXTEND THE DURATION OVER TIME.

MATERIAL COMPOSITION:

100% SEBS, TALCUM POWDER.

CONTACT:

INFO@RESTOREFEET.COM

@RESTOREFEET_

MYRESTOREFEET@GMAIL.COM

WE HAVE CREATED A VIDEO-GUIDE AVAILABLE IN MULTIPLE LANGUAGES. SCAN THE QR CODE TO ACCESS IT.

WWW.RESTOREFEET.COM

HERE YOU WILL FIND OUR VIDEO-GUIDE WITH EASY EXERCISES TO RESTORE THE NATURAL SHAPE OF THE FOOT.

SEE YOU INSIDE!

Medical Device
Made in China

2034-01

ZST6

ZSN6

ZSR6

2024-01

22055 - ZST6

Shijiazhuang Aofeite Import&export Co.,Ltd East and southwest side in the fourth floor in the NO.202 of Gree supporting industrial park, No.389 of Hengshan street, High-tech Zone, Shijiazhuang, Hebei Province, China, 050000.

GUIMI EVOMEDICAL SL, Av de Sant Esteve 157, 08211 Castellar del Vallès, Spain.

SUNGO Europe B.V, Olympisch Stadion 24, 1076DE Amsterdam, Netherlands.
Tel: (+31) (0) 2021 11106 Email: info@sungoglobal.com